MILFORD PERSON LIFT

(SV)	Bruksanvisning	03
(EN)	User manual	19
(DE)	Benutzerhandbuch	35
(FR)	Manuel d'utilisation.....	51
(ES)	Modo de empleo.....	67
(IT)	Manuale delle istruzioni...	83
(NO)	Bruksanvisning	99
(PT)	Manuel do usuário.....	115

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Congratulations on your purchase of an adaptation solution from Autoadapt!

Thank you for choosing us as the supplier for your customer's car adaptation. An adaptation from Autoadapt is designed to make it easier for your customer to drive their car, and we have focused particularly on installation-friendliness, safety, quality and functionality. This manual is an integral and important part of the product, providing you with information on how to perform accurate and safe installation of the product. Please read through the manual carefully before starting your work. If you have any concerns about your customer's adaptation, please feel free to contact Autoadapt. Your feedback is important to us. You can find more information about us and our range of car adaptation solutions by visiting www.autoadapt.co.uk

Once again, thank you for placing your confidence in our products!

Autoadapt

1. Safety	20
2. Operation	21
2.1 Belt adjustment, harness position	21
2.2 Lifting in and out of the car	22
3. Emergency-mode operation	30
4. Maintenance	31
5. Problem solving	32
6. Technical specification	33

1. Safety

Ensure that all belt straps are threaded correctly through their buckles to ensure that they do not slide during a lifting operation, see page 21.

Ensure that all belts are suspended correctly from the lifting yoke before lifting commences. Do not raise and lower the lift at the same time as rotating the person in and out of the car.

Ensure that hands, clothing and cables are not in the way of the lift's movement.

Place the manual control onto the top arm of the lift during use. You will then have both hands free to help the person in and out of the car.

Avoid allowing electrical components to get wet when it is raining.

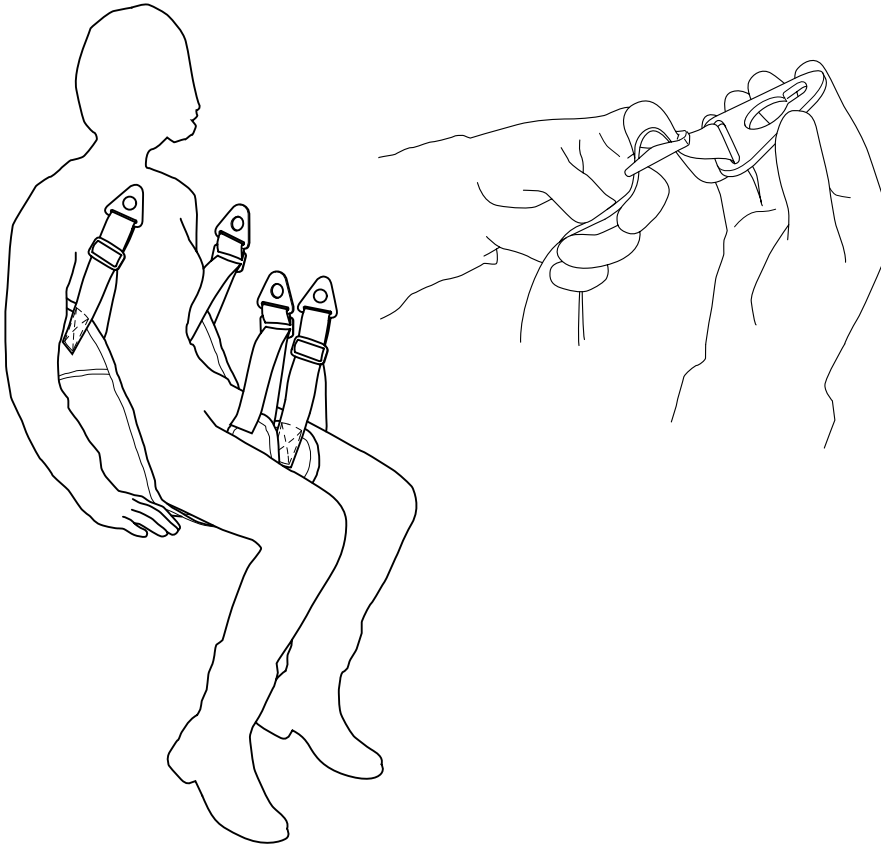
Ensure that the power cable and the cord for the manual control do not become pinched while the lift is being used or stored. Damage to electrical components could mean that the lift stops working and has to be repaired.

Always remove the harness and disassemble the lift before you commence your journey by car.

2. Operation

2.1 Belt adjustment, harness position

Thread the straps through the buckle as shown in the picture.

Adjust the length of the belt as required. They should be as short as possible. The correct length is when the person can be lifted on a level with the lower part of the chest.

A belt of the correct length permits easier lifting and rotating of the person in and out of the car.

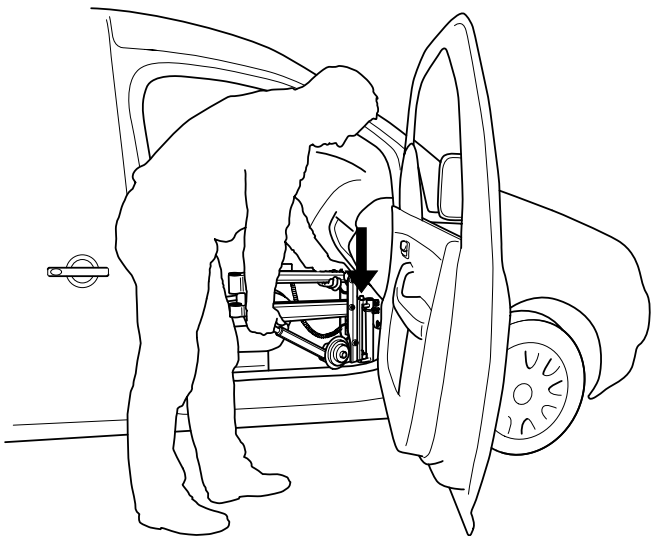
Place the harness with the belts as shown in the picture.

It is important that the harness should be placed under the armpits and as close to the groin as possible.


The correct positioning of the harness permits easier lifting and rotating of the person in and out of the car.

2.2 Lifting in and out of the car

1


2

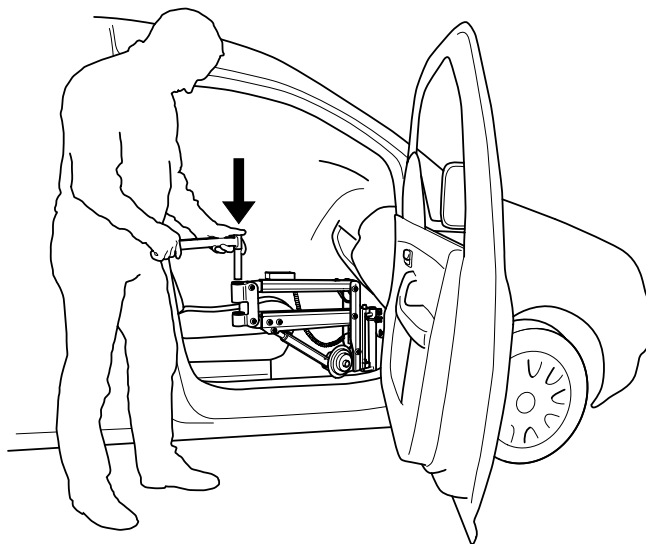1 Hook the lift onto the mounting post.


2 Assemble the extension arm onto the lift.

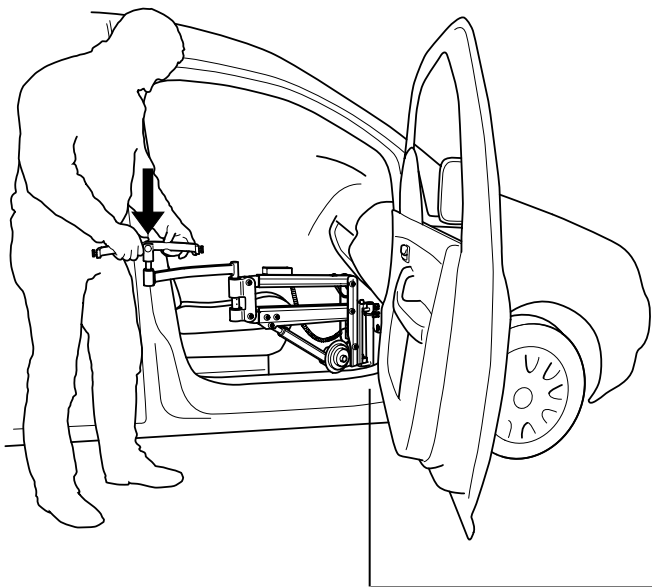3


3 Assemble the lifting yoke onto the extension arm.

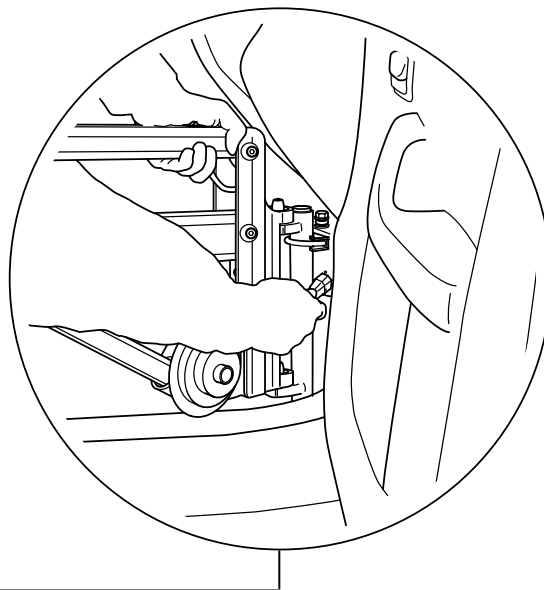4


4 Insert the plug into the contact piece.

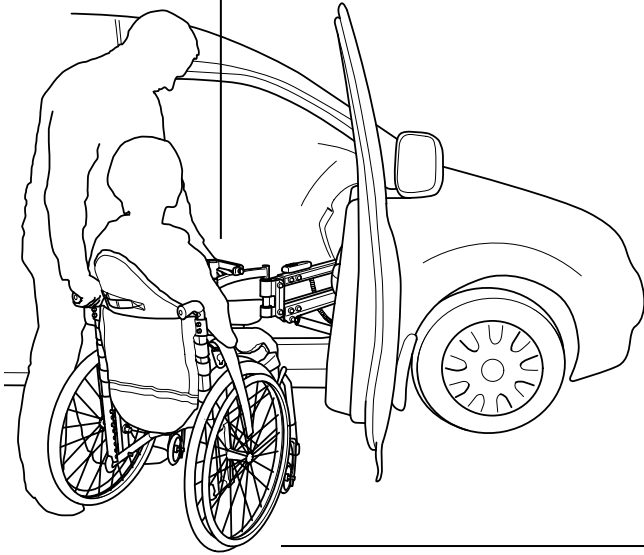5


6

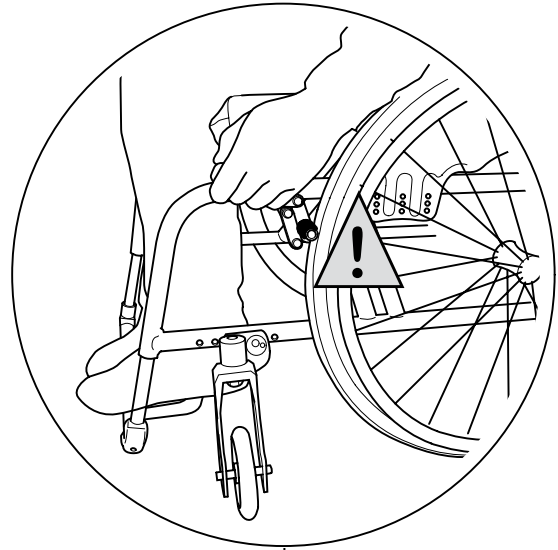5 Position the person in the wheelchair in front of the lifting yoke.


6 Apply the wheelchair brake.

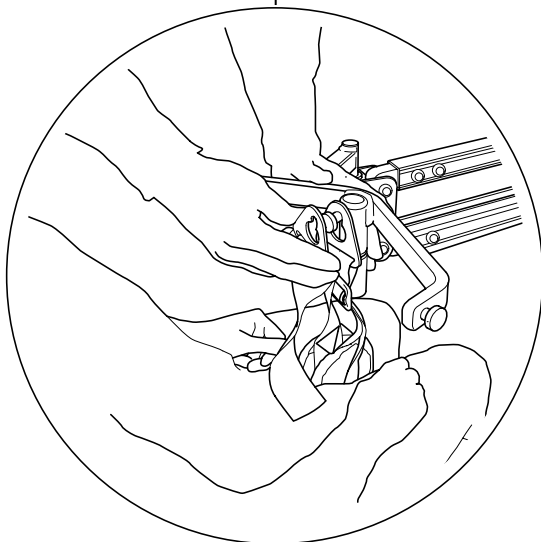7


7 Hook the thigh belts onto the centre of the lifting yoke. The belts should be under tension. For adjustment, see page 21.

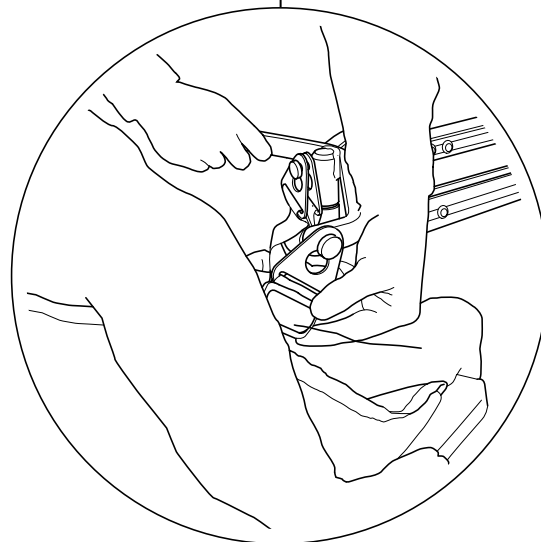8


8 Hook the chest belts onto the sides of the lifting yoke. The belts should be under tension. For adjustment, see page 21.

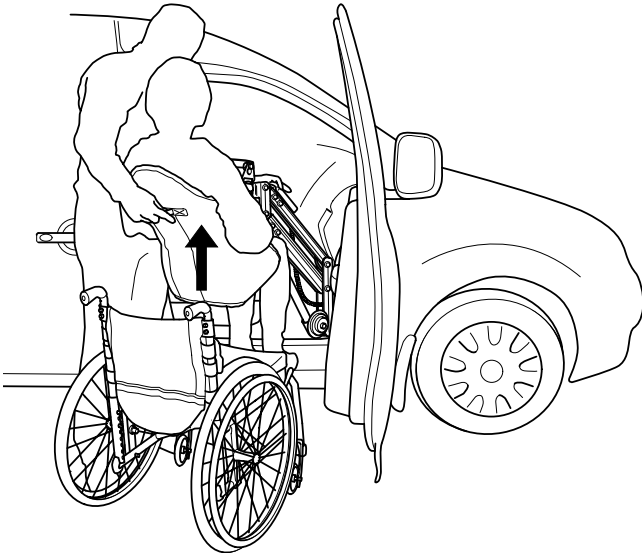
9

10

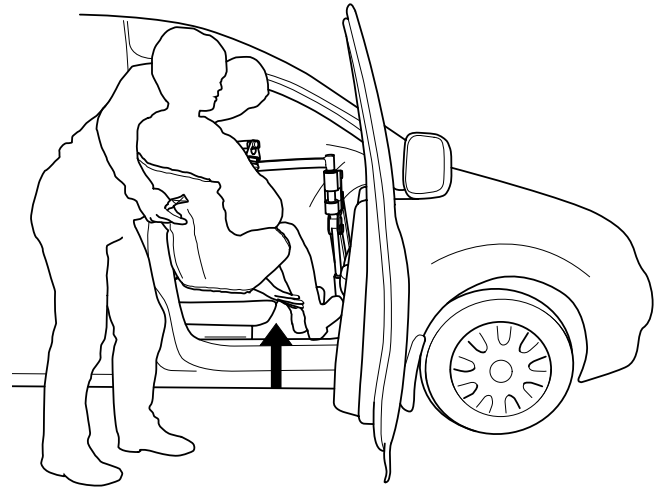9

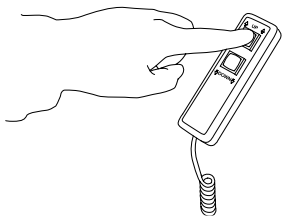
Press UP


10

Place the arms of the lift along the door for maximum lifting room. Lift in the feet.

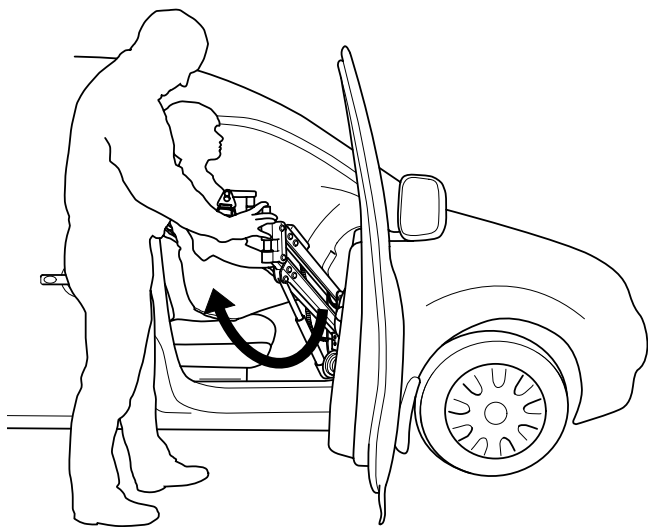11


12

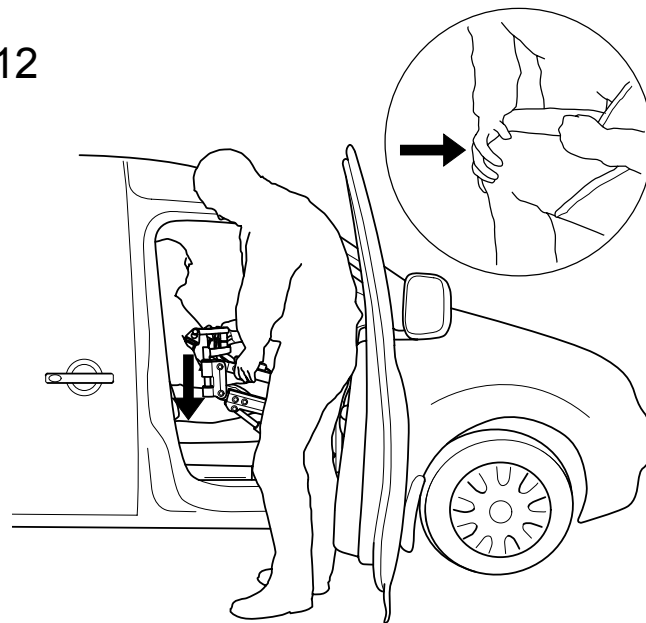
11 Rotate and move the person into the car.

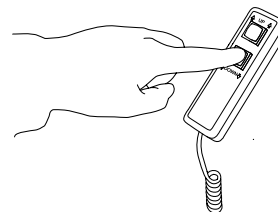12

Press DOWN
Press the hand against the
knees and move the person
as far back as possible in
the seat.

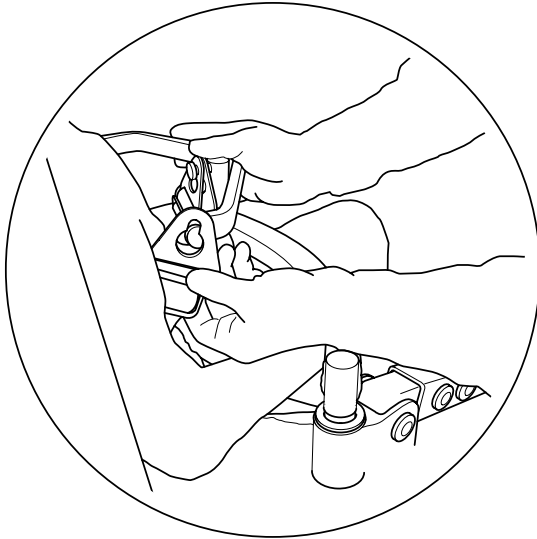13


14

13 Unhook the chest belts from the sides of the lifting yoke.


14 Unhook the thigh belts from the centre of the lifting yoke.

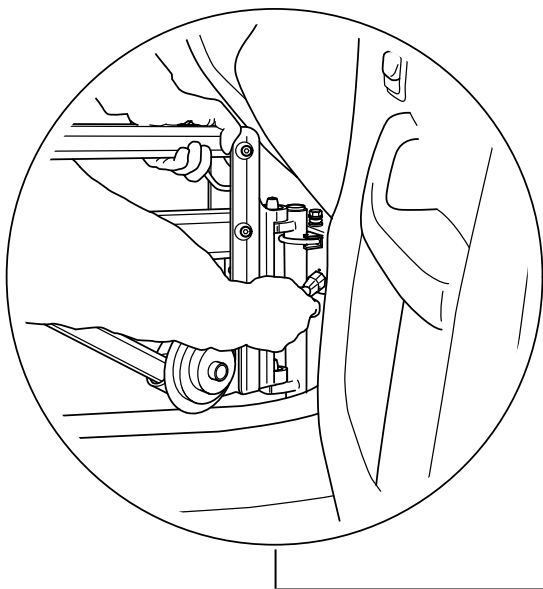15

15 Pull the plug out of the contact piece.

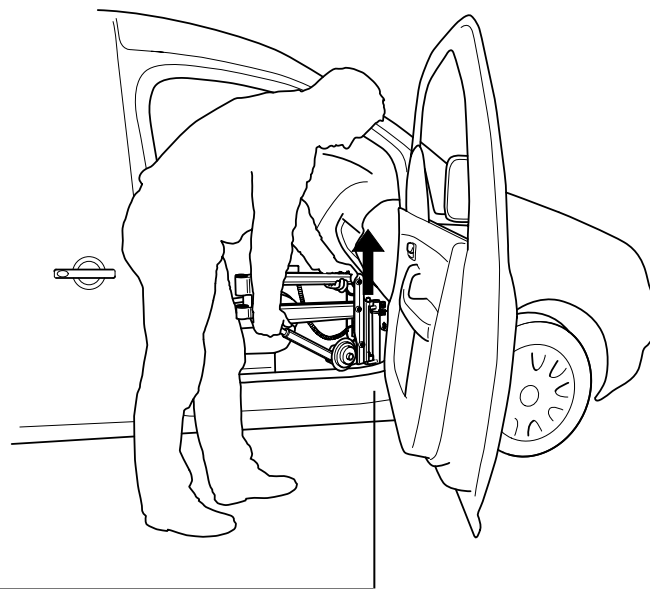
16

16 Disassemble the lift.

To lift the person out of the car, perform steps 5 to 14 in reverse order.

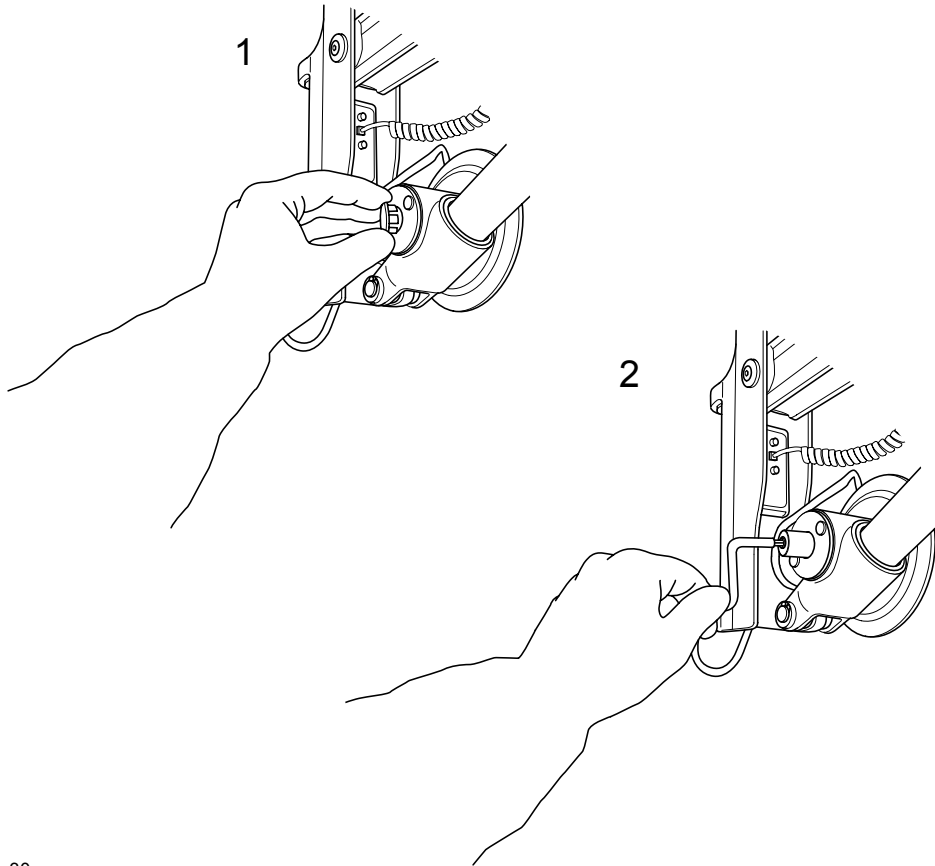3. Emergency-mode operation

When manual operation of the Milford is required, follow the paragraphs below.

1. Remove the plastic shield on the rear of the lift's motor.
2. Use the emergency operation sleeve supplied. Rotate the sleeve to operate the lift up or down.
Replace the plastic shield.
3. Contact your dealer to have the product overhauled.

4. Maintenance

RECOMMENDED FREQUENCY	MAINTENANCE
When necessary	* Clean the lift, extension arms and lifting yoke. * Wash the harness, see the instructions on the back of the harness.

- The product documentation must always be taken along when the vehicle is being inspected.
- When disassembling, always contact your dealer.


5. Problem solving

PROBLEM	SOLVING
The lift will not move.....	<ul style="list-style-type: none"> * Check that the contacts for the lift and the manual controller are connected. * Check that the circuit-breaker on the lift is switched on * Check that the car battery is fully charged. * Check that the fuse close to the car battery has not blown. * Check that none of the cables have been damaged. In the event of any damage, contact your dealer.
The person is not being lifted high enough or feels uncomfortable during the lift.	<ul style="list-style-type: none"> * Check that the harness fits correctly under the arms and legs, and that the belts on the harness are as short as possible.
The lift is moving extremely slowly. ...	<ul style="list-style-type: none"> * Check that the maximum weight of 111 kg has not been exceeded. * Check that the car's battery is fully charged.


For further assistance or answers to queries, please contact your dealer.
 We recommend you have the following information to hand:

Model.....	Installer contact information.....
Serial number
Year of manufacture
Date of purchase

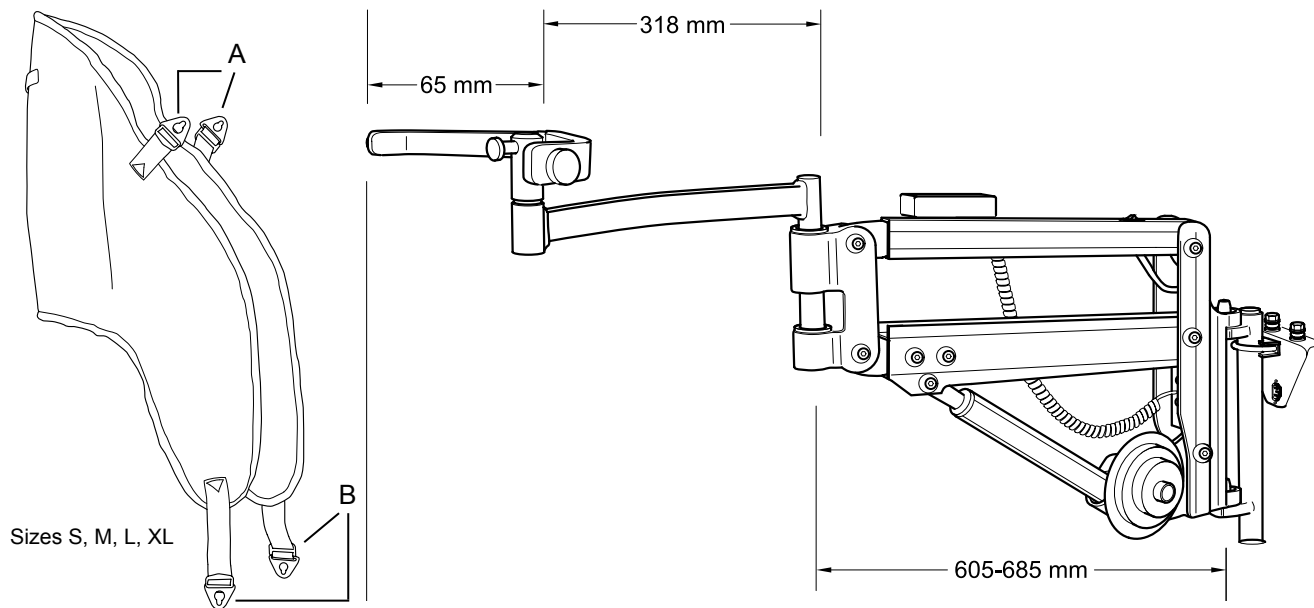6. Technical specification

Manufacturer	Autochair Ltd / UK	Power consumption	12 V
Model	Milford electric person lift	Fuse	25 A
Max width	988 mm	Maximum lifting weight	111 kg
Max height	580 mm	A / chest belts
Lift weight	8 kg	B / thigh belts

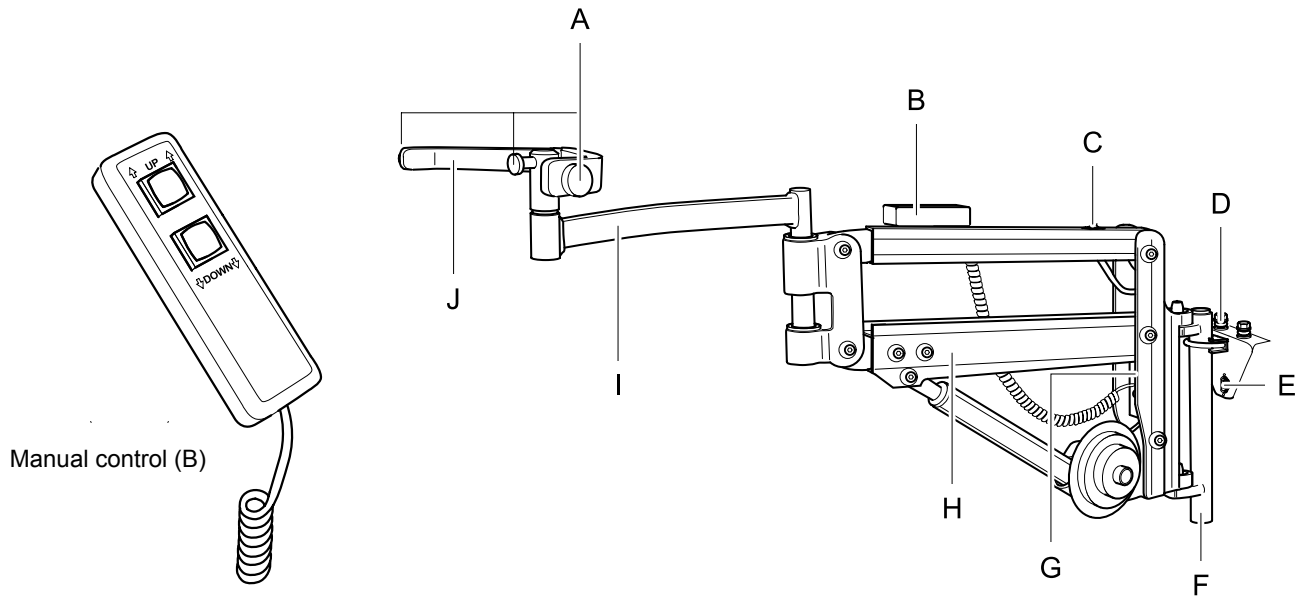-
- | | | | |
|---------|--------------------------|---------|---|
| A | Belt brackets | F | Mounting post |
| B | Manual control | G | Fixed set of buttons for lifting function |
| C | ON/OFF button | H | Lift |
| D | Bracket for support arms | I | Extension arm |
| E | Contact piece | J | Lifting yoke |

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© 2010 Autoadapt AB
Autoadapt AB
Autoadapt UK Ltd