# Steering knob adapter

Simplifies one-handed steering

SV	4	Installationsanvisning / Bruksanvisning
EN	8	Installation instructions / User manual
DE	12	Einbauanleitung / Benutzerhandbuch
FR	16	Manuel d'installation / Manuel d'utilisation
ES	20	Instrucciones de instalación / Modo de empleo
IT	24	Istruzioni per l'installazione / Manuale delle istruzioni

SV

EN

DE

FR

ES

IT

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Thank you for choosing a

# Steering knob adapter from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit [www.braunability.eu](http://www.braunability.eu).

Once again, thank you for placing your confidence in our products!

---

**Mounting**

|0

---

**Operation**

|1

EN

Index

“Product certificate” at the end of the manual.

## Mounting

For warranty conditions the adapter shall be installed by a workshop authorized by BraunAbility. Make sure not to damage the steering wheel while removing the adapter.

The adapter can leave impression marks on the steering wheel.

### Determine a suitable position

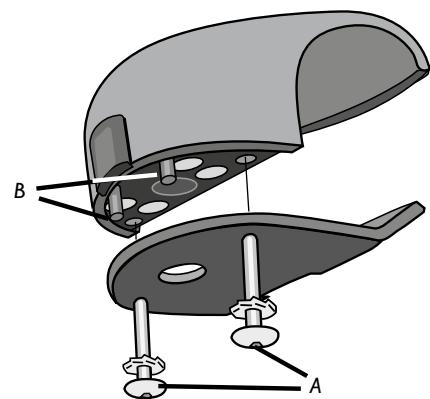
Make sure that the adapter and the steering knob will not interfere with other vehicle controls. The adapter shall be placed with the release button facing towards the steering wheel centre. For individuals using a hand control for accelerator and brake with their right hand, a suitable position for the steering knob is usually on the left side (10 o'clock), in left side driven vehicles. In right side driven vehicles a suitable position, under the same conditions, would be reversed (2 o'clock).

### Mounting

Remove the fastening plate (bottom part) by unscrewing the two screws (A). Check that the adapter fits the steering wheel. If there is a gap, place some rubber material in between to achieve a snug fit. Adjust the two spacing screws (B) to create a hold-on against the fastening plate. Mount the fastening plate using the two screws (A) and the locking washers. Be careful to screw correctly into the threads of the holes.

### Check-up

Check that the adapter is properly fastened by mounting the steering knob and trying to move the adapter inwards and out. The adapter shall stay put.

# Operation

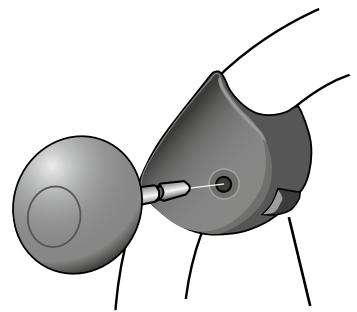
## Mounting and removal of steering knob

EN

Mount the steering knob by pushing the peg into the quick-fastening of the adapter, until a click-sound is heard. Check that the knob is properly fastened!

Remove the steering knob by pressing the release button on the side of the adapter, while at the same time pulling out the steering knob.

Check regularly (for example monthly) that the adapter is properly fastened to the steering wheel. If the quick-fastening for the steering knob is not functioning properly, contact the adaptation workshop.

*Button for releasing the steering knob from the quick-fastening.*

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## Declares that the product /

Steering knob and adapter

## Conforms to following directives, standards and regulations /

2006/42/EEC  
ISO/TC22/WG4

BraunAbility Europe AB, Stenkullen  
2019-04-02

Jan M Jensen  
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005  
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.

BraunAbility Europe AB reserves the right to make alterations without previous notice.

© 2019 BraunAbility Europe AB

CE

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)