# Carony Go

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Thank you for choosing a

# Carony Go from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit [www.braunability.eu](http://www.braunability.eu).

Once again, thank you for placing your confidence in our products!

---

**Safety**

---

28 Safety measures

**Operation**

---

30 Adjustments

32 Reflectors

33 The control box

36 From wheelchair to vehicle

38 From vehicle to wheelchair

**Care instructions**

---

40

**Technical specification**

---

46 Technical data

**Introduction for users**

---

47

“Product certificate” at the end of the manual.

# Safety

## Safety measures

*Carony Go is tested and approved for a maximum user weight of 120 kg.*

*Before using Carony Go for the first time, the batteries must be charged.*

*When Carony Go is not in use always connect to battery charger.*

*Carony Go shall not be driven over curbs higher than 6 cm.*

*Carony Go shall not be driven in steeper slopes than 6 degrees.*

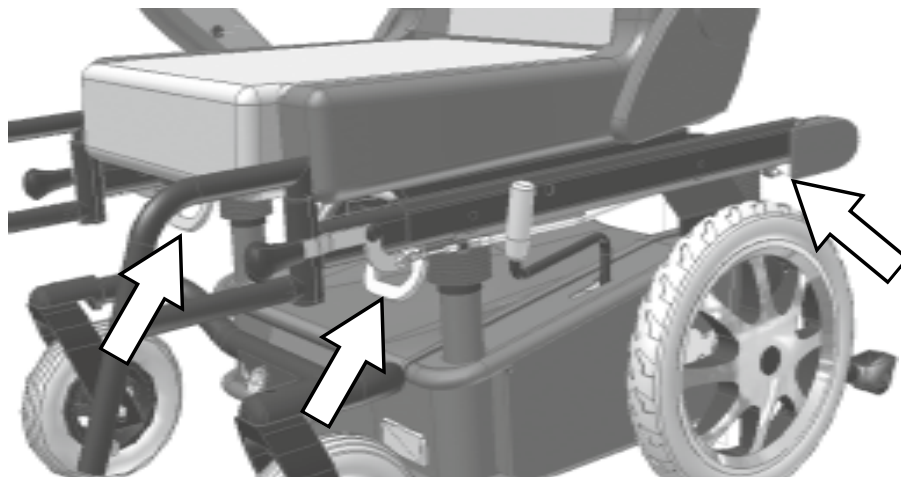
*Driving in slopes and on supple ground (gravel, grass, ice and snow etc.) must be done with caution and in slow speed.*

*Make sure to engage the motors again after they have been disengaged.  
Transportation in a slope can be hazardous if the motors are disengaged.*

*The Carony Go wheel unit must be well secured in the luggage compartment when transported. Secure the wheel unit by the securing loops, two at the front and two at the back.*


# Operation

## Adjustments

### Legrests


The foot rests are length adjusted by first loosening the wing nut.

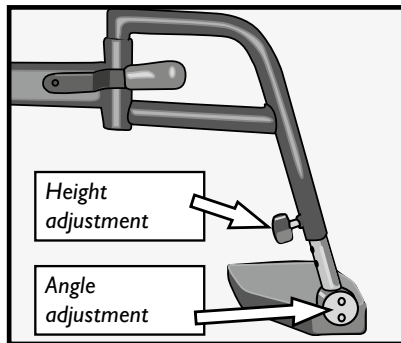Angle adjustment, loosen the screw of the angle cylinder by using a 6 mm Allen key.


### Headrest

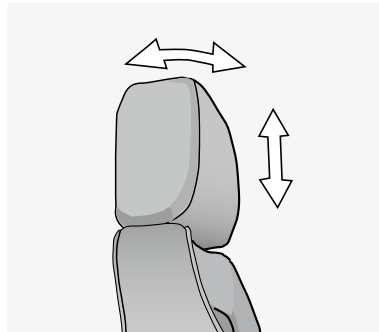Push the headrest by hand, to adjust it forward or backward. The height adjustment is done by pulling up-wards, or pushing downwards.

### Backrest

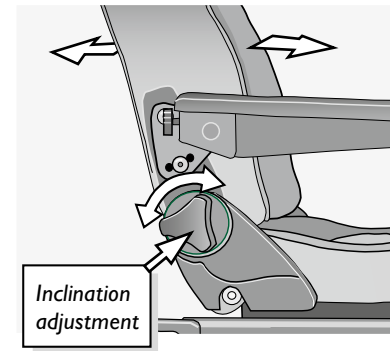The adjustment of backrest is limited to avoid risk for tilting. Adjust the backrest by turning the control, placed on the right hand side of the seat. Turn clockwise for raising the seat forward and counter-clockwise for leaning backwards.


## Armrest

Height adjustment /

Unscrew the height adjustment to adjust the position of the foot plate.

Angle adjustment /

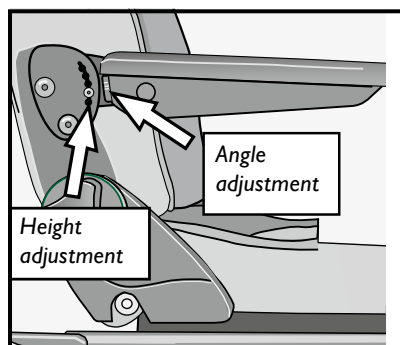Adjust the angle by loosening the socket head screws, using an 5 mm.


## Seat lift

The seat lift is used for adjusting the seat height to the vehicle swivel base. The seat height adjustment is also of advantage when making transfers to another seat or when wanting to reach higher up. The seat height can be adjusted from 42 to 55 cm.

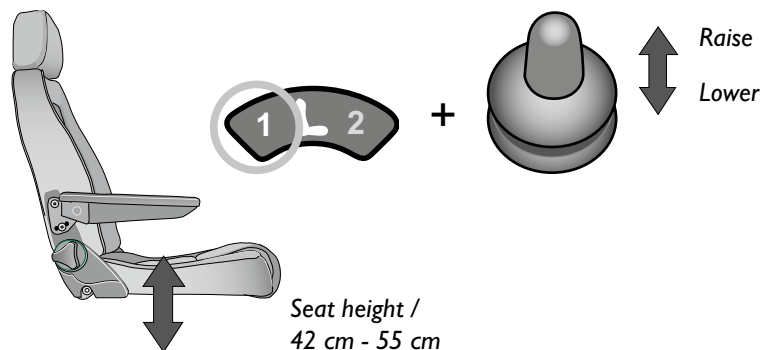*When driving the wheelchair the seat must be in lowest position. The maximum speed is, due to safety reasons, reduced if the seat is in raised position.*


## Reflectors


### Seat

The seat length is adjusted by moving the cushion forwards or backwards.

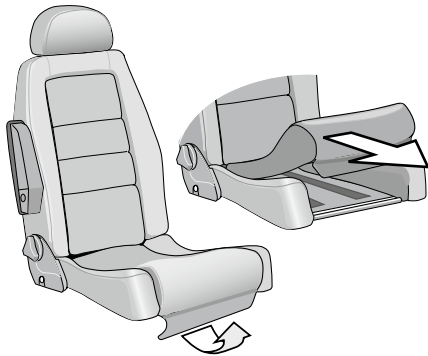
Carony Go is equipped, as standard, with two red reflectors at the back, one yellow reflector on each side and two white reflectors in front.

*At the front*

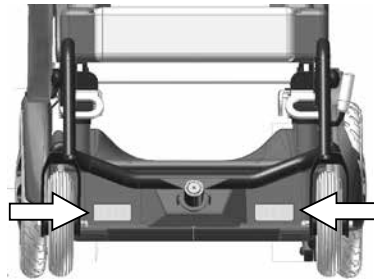*At the back*

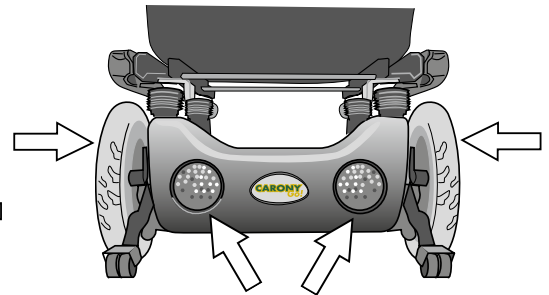
## The control box

### Memorized settings

All settings, such as speed, seat height etc., are saved when the wheelchair is turned off and remain memorized as it's turned on again.


### Lock mode

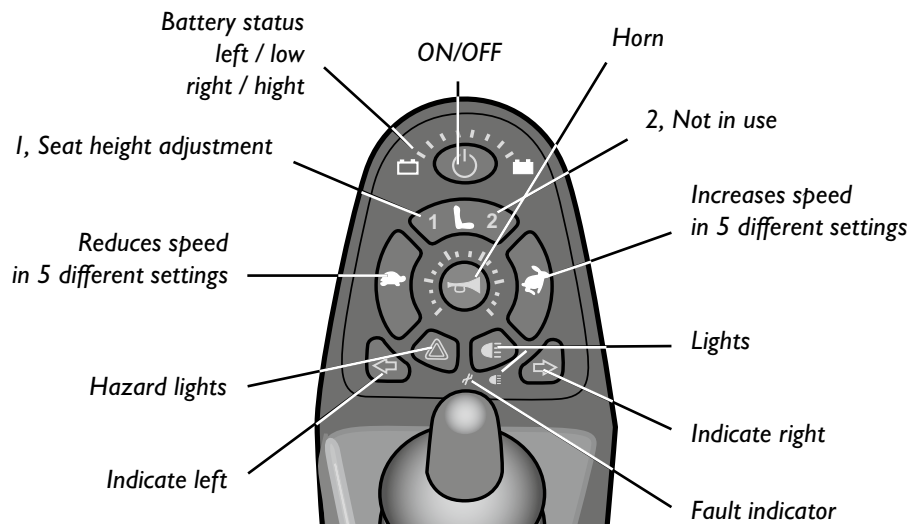When keeping the ON/OFF button pressed for 4 seconds the control box will enter lock mode. To unlock, press the horn button twice.


**The control box arm is lowered as follows**

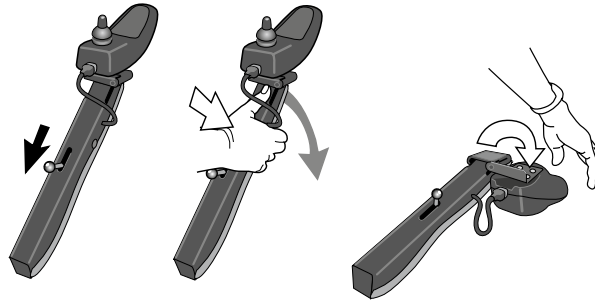Pull the release button, placed on top of the arm, towards the back and lower the arm. At this stage the control box has also been released and can be folded to the side.


**The control box arm is raised as follows**

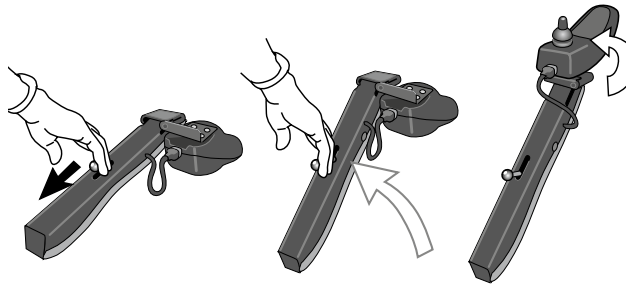Pull the release button towards the back and the arm will rise by itself. Return the control box to operating position.

**The lengthwise position of the control box arm is adjusted as follows**

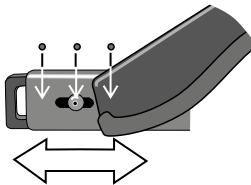Unscrew the socket head cap screw placed in the groove by the arm attachment in the wheelchair frame. Adjust and tighten the screw.

## Driving

Activate the wheelchair by pressing 

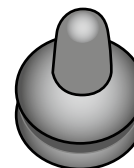
Adjust the speed setting by pressing the  for higher speed or the  for slower.

There are five different speed settings, displayed on the circle shaped speed indicator of the control box.

Move the joystick in the direction where you wish to drive.


The speed transmission is variable depending on how far the joystick is moved.

NOTE! The maximum speed is, due to safety reasons, reduced if the seat is in raised position.


## From wheelchair to vehicle
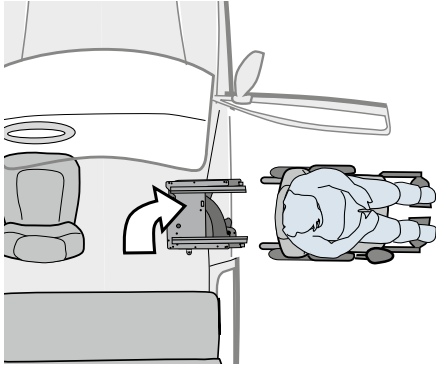
### 1. Reverse the wheelchair


Reverse the wheelchair towards the turned out swivel base Turnout or the lowered car seat lift Turny.

### 2. Disengage the motors

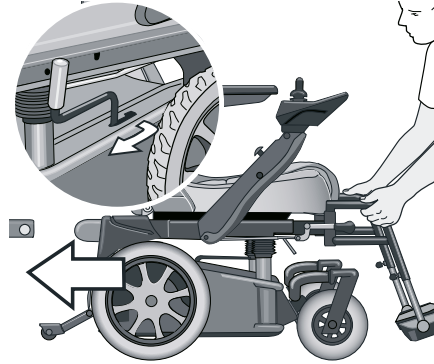
Disengage the motors by pulling the handle, placed on the left side of the wheelchair, forward. Move the wheelchair towards the swivel base.

### 3. Adjust the seat height

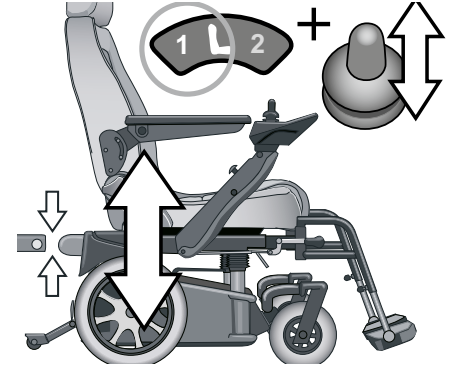
Level the seat height to the rails in the vehicle. The height is adjusted by first pressing the seat lift button, marked 1, and then by moving the joystick forwards or backwards.

## 4. Lock

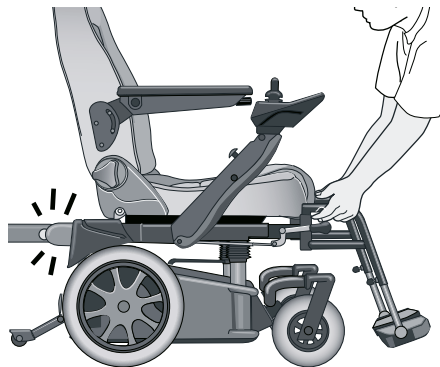
Push the wheelchair towards the rails in the vehicle. The covers of the clutch open automatically. Push until it connects properly to the vehicle rails. Make sure the units have been properly connected by trying to pull them apart.

## 5. Transfer the seat

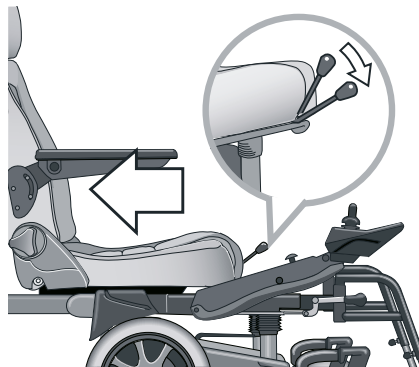Release the seat from the wheel unit by pulling down the seat release placed at the front of the seat. Slide the seat over to the vehicle until it locks in the inner position. Make sure that the seat release handle has returned to original position and that the seat is unable to slide forward. Check this!


## 6. Lossa underredet

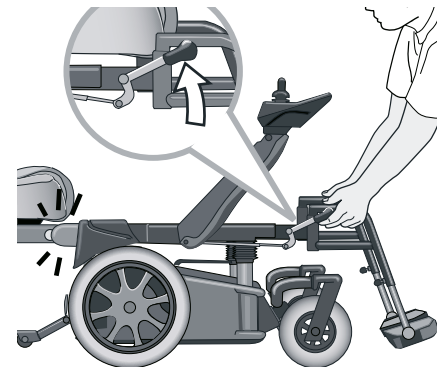Height adjust to compensate movement of weight. Release the wheel unit from the vehicle by pulling up both lock handles. Swivel the seat inside the vehicle/ drive the seat lift up and inside. Engage the motors. Turn off the power and load the wheel unit in the vehicle luggage compartment and secure it.


## From vehicle to wheelchair
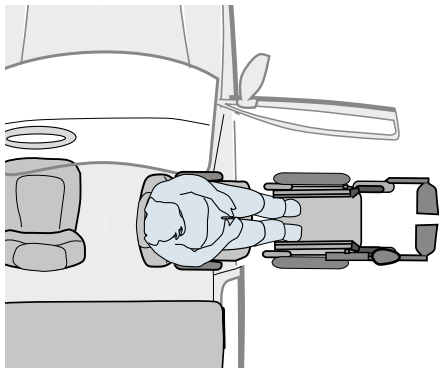
### 1. Drive up the wheel unit


Reverse the wheel unit towards the swivelled out seat.

### 2. Disengage the motors

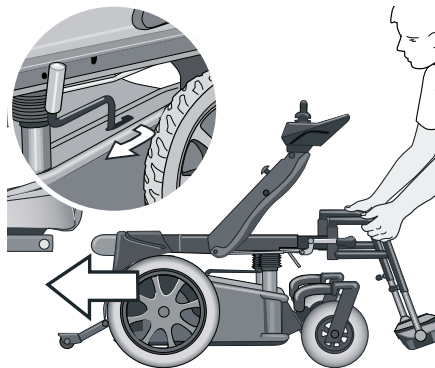
Disengage the motors by pulling the handle, placed on the side of the wheel unit, forward. Push the wheel unit towards the seat rails.

### 3. Adjust the seat height

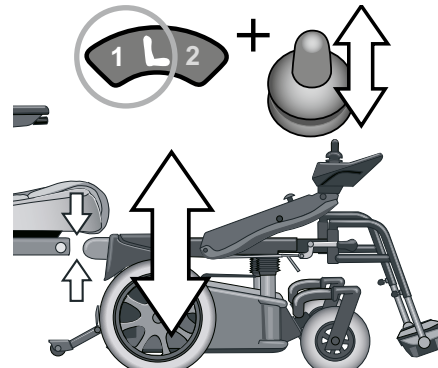
Adjust the height of the wheel unit and level it to the rails in the vehicle. Adjust by pressing the seat lift button, marked 1, and then moving the joystick back or forth.

## 4. Lock

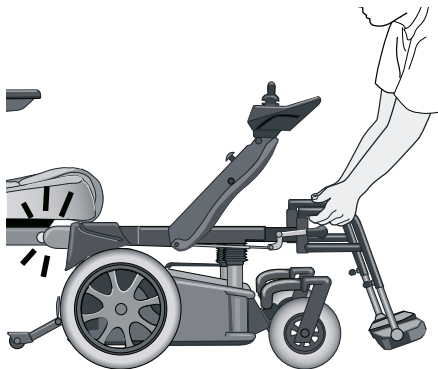
Push the wheel unit towards the rails in the vehicle. The covers of the clutch open automatically. Push until it connects properly to the vehicle rails. Make sure the units have been properly connected by trying to pull them apart.

## 5. Transfer the seat

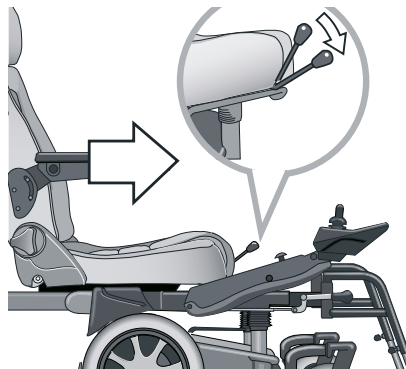Release the seat from the vehicle by pulling down the seat release handle at the front of the seat. Slide the seat over to the wheel unit until it locks at end position. Make sure that the seat release handle has returned to original position and that the seat is unable to slide backwards!

## 6. Prepare for drive

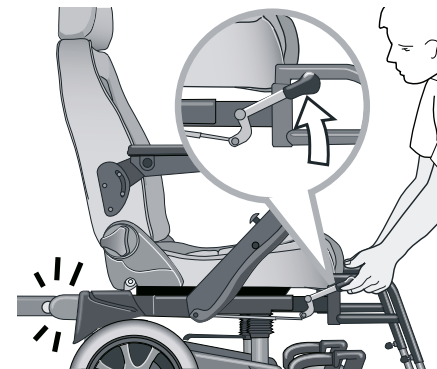Height adjust to compensate movement of weight. Release the wheelchair from the vehicle by pulling both lock handles placed at the front. Adjust the seat height to lowest position. This is important for the balance of the chair. Engage the motors!

## Care instructions

If necessary.....

### **Clean**

Do not use a high-pressure cleaner and do not flush water on the wheelchair. This might cause damage to the electricity system etc. Clean the wheelchair with a moist towel and a gentle cleaning agent. For upholstery use vacuum cleaner and upholstery cleaner. The seat cushion is detachable and can be machine washed in 40 degrees C. Do not dry in tumble-drier or drying cupboard!

### **Inflate tires**

The wheel pressure needs to be checked regularly. The driving wheels (rear wheels) shall have a pressure of 2.8 Bar. The front wheels shall have 2.6 Bar. Low air pressure can cause extra wear and tear to the wheels.

### **Replace tubes**

Place pieces of board under all wheels except the one that shall be amended. Remove the wheel using a 15 mm wrench. Let the air out of the tube and remove the tyre from the rim. Replace the old tube with a new one. Restore and mount the wheel. Don't forget to tighten the nut. Inflate the wheel to specified air pressure.

### **Replace fuses**

The fuse is placed under the plastic cover of the wheel unit. The cover is Velcro fastened and is easy to pull off. The main fuse of 50 Ampere is placed on the red cable. A defective fuse might be caused by a major fault of the electric system. Take the wheelchair to an Authorized workshop.

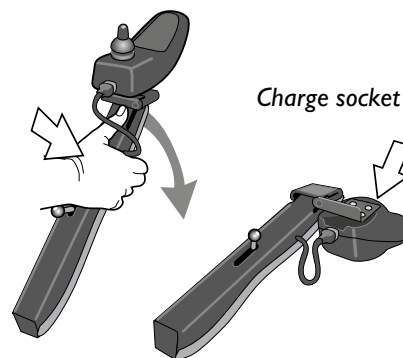
If necessary.....

### Charge batteries

The Batteries of the wheelchair are of gel type and therefore don't need any liquid check-up. Regular charging prolongs the lifetime of the batteries. Batteries shall be charged in a ventilated area, using a charger of type IVO 24/6 IP65. When the battery is charging the charger lamp flashes. When the battery is fully charged the lamp shines constantly. The charge socket is placed in the front edge of the control box.

BEFORE using Carony Go for the first time the batteries must be charged.  
When Carony Go is not in use - connect to battery charger.

**Battery status**  
left / low  
right / high


If necessary.....

### Replace batteries

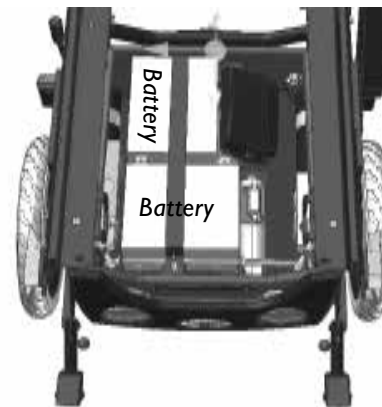
Remove the plastic cover, fastened by Velcro. Loosen the wing nut holding the battery bracket and remove this. Raise the rear battery upright. Disconnect the cables and remove the battery. Push the remaining battery as far back as possible, raise it up and remove it.

When new batteries are connected it is very important that they are correctly placed. First connect the minus then the plus poles.

*Under the plastic cover of the wheel unit*

*This diagram is placed on the inside of the plastic cover and shows how the batteries are connected.*

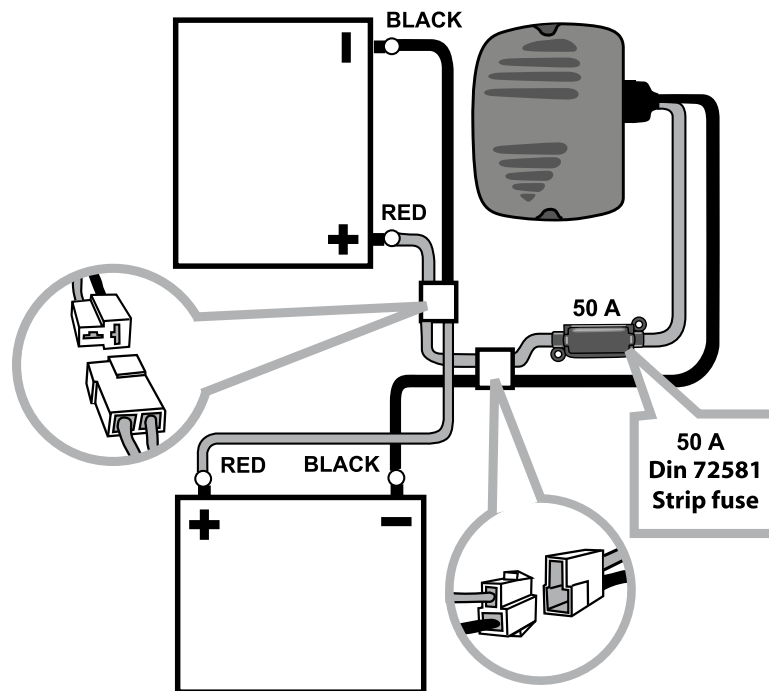
**Battery connection of Carony Go.**

If necessary.....

**Replace headlight bulb**

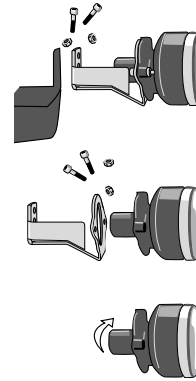Remove the lamp body from the control box arm, by unscrewing the two screws with a size 3 hexagon key. Unscrew the two screws that fix the lamp body to its holder. Unscrew the black socket from the lamp body. Remove the bayonet fastened bulb and replace it with a new. Remount and tighten all screws.

**Replace bulbs of rear light and rear indicator**

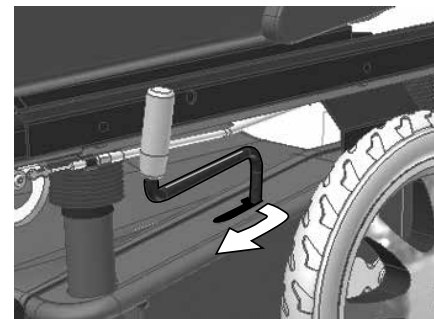
Unscrew and remove the glass from the rear light. Change bulb and remount the glass.

**Replace bulb of front indicators**

Use a blunt knife or other tool to remove the glass. Replace the bulb and remount the glass.

When Carony Go is used.....

Make sure that the handle for disengaging and engaging the motors is functioning.

- The product documentation must always be available during inspection of the vehicle.
- Always contact your dealer for disassembly.

# Technical specification

## Technical data

Total width.....	65 cm
Length.....	75 cm
Length incl. foot rest and anti-tipping device.....	108 cm
Height excl. seat.....	38 cm
Height incl. seat.....	115 cm
Total weight inc. seat and batteries.....	93 kg
Weight of wheel unit incl. batteries.....	75 kg
Max clearance beneath wheelchair.....	65 mm
Seat height, adjustable.....	42-55 cm
Range.....	25 km
Max speed.....	7 km/h
Max reverse speed.....	3 km/h
Braking distance.....	0,8 m
Obstacle clearance.....	6 cm
Turning space 180 degrees.....	1500 mm
Hill-climbing ability.....	6 grader
Max user weight.....	120 kg

Battery type.....	AGM: DJM 1245
Battery max dimensions.....	220x130x80 mm
Main fuse.....	50A
Lights.....	10A
Front wheel dimensions.....	200x50
Front wheel tyre pressure.....	2,6 bar
Back wheel dimensions.....	315x50
Back wheel tyre pressure.....	2,8 bar
Headlight, BA 15 st.....	24V/21W
Indicators front, Ba 15 st.....	24V/5W
Indicators back, Ba 15 f.....	24V/5W
Rear light, BAY 15 d.....	24V/3W

## Introduction for users

BraunAbility Europe AB would like you, as a user, to feel secure and comfortable with your new product. For that reason, it is your installer's duty to go through the following with you /

Safety  
Operation

If any questions about the product should come up later on, you are of course very welcome to contact your installation engineer again.

The above chapters have been gone through /

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## Declares that the product /

Carony Go

## Conforms to following directives, standards and regulations /

2006/42/EEC, 2007/47/EEC  
LVFS 2003:11  
ECE R-10  
ECE R-17 rev 5 (incl supplement 08)  
ECE R-21  
ECE R-25  
EN 12182  
EN 12184:1999  
ISO 3795  
ISO 7176: 1-19  
ISO 10542  
ISO 8191-1  
ISO 8191-2

BraunAbility Europe AB, Stenkullen  
2019-04-02

Jan M Jensen  
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005  
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.  
BraunAbility Europe AB reserves the right to make alterations without previous notice.  
© 2019 BraunAbility Europe AB

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)