Turnout

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Thank you for choosing a

Turnout from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit www.braunability.eu.

Once again, thank you for placing your confidence in our products!

Safety

16

Operation

16 Turnout 2 and 4

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Care instructions

20

Troubleshooting

21

Introduction for users

23

“Product certificate” at the end of the manual.

Safety

Max user weight is 150 kg.

Ensure that the car is standing on a level surface and that the parking brake is set before using Turnout.

Ensure that nothing can fasten or get caught when manoeuvring Turnout. Make sure that the cable to the hand control hangs freely to avoid it becoming damaged.

If possible, recline the back support to minimise the risk of the user's head bumping against the door opening whilst being moved.

If an obstacle is in the way the swivel movement will stop immediately and reverse a little bit. Remove the obstacle and press the button again.


Any modifications, adjustments or repairs to a Turnout may only be performed by authorised technicians.


Operation

Turnout 2 and 4

Rotate outwards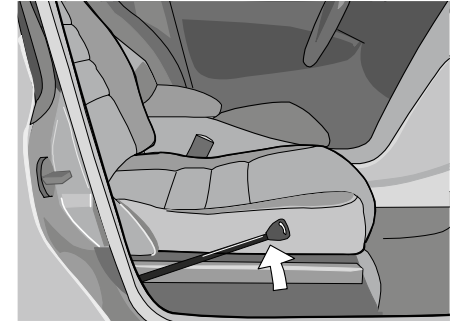

1. Adjust the seat lengthwise to optimize space for rotating the seat. Lift the release lever and start the rotating movement of the seat.

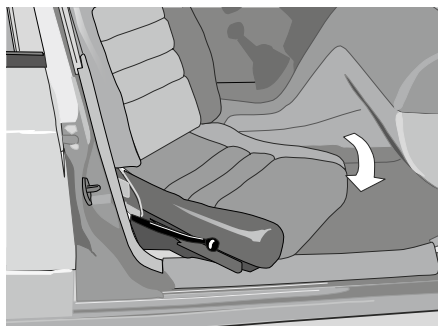
2. Let go of the release lever. Rotate the seat by holding on to the seat and not by pulling the lever. When the ultimate outer position is reached it locks automatically.

3. If needed, the seat can be adjusted further out by pulling the release for lengthwise adjustment.

Rotate inwards

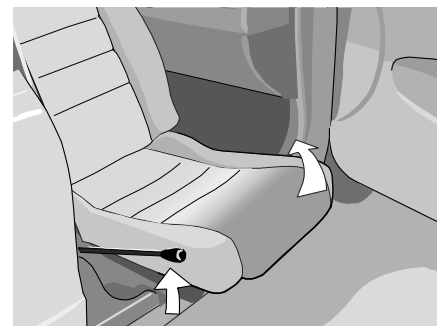Adjust the seat if needed, to an optimal position for rotating it inside. Lift the release lever and start the rotating movement of the seat. Rotate by pushing the seat and let go of the release lever. Rotate the seat until it locks in its ultimate inner position!

NOTE! Watch the head!

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The hand control is universal. The LEDs on the hand control is not used for Turnout.

- The standard model Turnout is supplied with a cable connected hand control unit. Turnout can be manoeuvred via a wireless hand control unit, available as an optional extra.
- For safety reasons the Turnout can only be manoeuvred when the vehicle is stopped and/or the door is open.
- When not in use place the hand control in its holder.


OUT /

In order to rotate outwards keep the left-hand button in. To interrupt the movement, release the button.


IN /

In order to rotate inwards keep the right-hand button in. To interrupt the movement, release the button.

Rotate outwards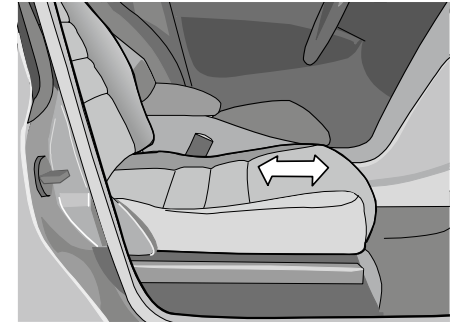

1. Adjust the seat lengthwise to optimize space for rotating the seat. Lift the release lever and start the rotating movement of the seat.

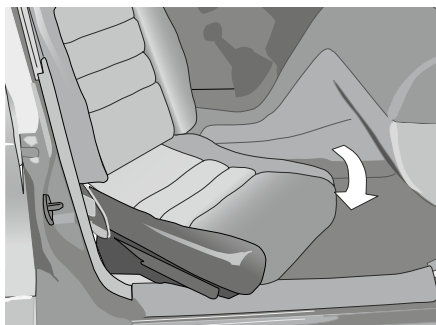
2. The seat rotates towards the outside as long as the left hand side button of the control is pressed down. At ultimate outer position the movement stops automatically.

3. If needed, the seat can be adjusted further out by pulling the release for lengthwise adjustment.

Rotate inwards

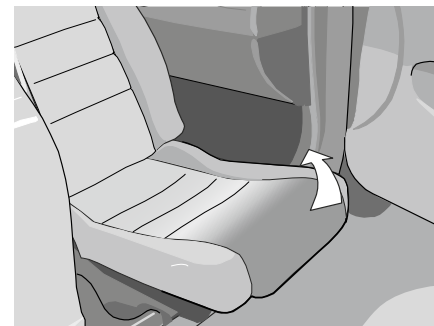Adjust the seat if needed, to an optimal position for rotating it inside. The seat rotates towards the inside as long as the right hand side button of the control is pressed down. At ultimate inner position the movement stops automatically.

NOTE! Watch the head!

Care instructions

If necessary.....

Carefully vacuum the Turnout. Do not use any liquids.

Once a year.....

Contact your installer to book the annual service.

Always contact your dealer for disassembly.

Any modifications, adjustments or repairs to a Turnout may only be performed by authorised technicians.

Product data

Fuse, Turnout 2E and 4E 20A

Troubleshooting

Turnout 2 an 4

The swivel end position does.....
not lock properly.

Make sure nothing is in the way of Turnouts movement. Try again.
Contact your installation engineer if the problem remains.

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Nothing happens when control
button is pressed.

Check that /

- the fuse 20A, placed in the fuse holder near the vehicle battery plus pole.
- no object has got stuck in the Turnout, and has released the over current protection.


Start the engine of the vehicle to ensure that there is enough power in the battery.
Contact your installation engineer if the problem remains.

Turnout does not turn as far as usual.....

Check if there is something blocking the turning movement, something stuck in the gear drive.
Start the engine of the vehicle to ensure that there is enough power in the battery.

Turnout has stopped functioning
in the turned out position.

If none of the above suggested measures help out, it will be necessary to loosen the motor from the swivel base, to be able to turn in the seat manually. Loosen the screw, marked in the picture, and take the motor with the gear drive away from the gear plate. Turn in the swivel base. Remount the motor.

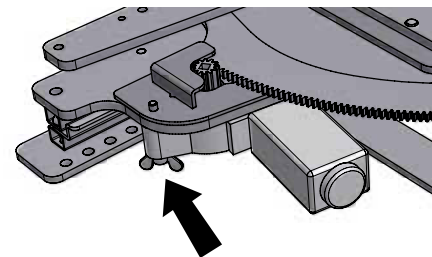Please contact your installation engineer if you require further assistance or need to ask questions about warranty. We recommend keeping the following information to hand at all times /

Contact details for the installation engineer /

Model /

Serial number /

Year of manufacture /

Date of purchase /

Notes /

Introduction for users

BraunAbility Europe AB would like you, as a user, to feel secure and comfortable with your new product. For that reason, it is your installer's duty to go through the following with you /

Safety
Operation

If any questions about the product should come up later on, you are of course very welcome to contact your installation engineer again.

The above chapters have been gone through /

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Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu

Declares that the product /

Turnout, Turnout II, model Left/Right, 2/4
and E2/E4

Conforms to following directives, standards and regulations /

2006/42/EEC, 2007/46 EEC, 2007/47/EEC,
LVFS 2013:II
ECE R-10, ECE R-14, ECE R-16
ECE R-17 rev 5 (incl supplement 08)
ECE R- 21
ISO 3795
FMVSS 201
FMVSS 207
FMVSS 208
FMVSS 209
FMVSS 210
FMVSS 225
FMVSS 302

BraunAbility Europe AB, Stenkullen
2019-04-02

Jan M Jensen
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.
BraunAbility Europe AB reserves the right to make alterations without previous notice.
© 2019 BraunAbility Europe AB

BraunAbility Europe AB
Åkerivägen 7
443 61 Stenkullen
Sweden

Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu