Turny 6-Way

SV Bruksanvisning
 EN User manual
 DE Gebrauchsanweisung
 FR Instructions d'utilisation
 ES Instrucciones de uso
 IT Istruzioni per l'uso

SV


EN

DE

FR

ES

IT

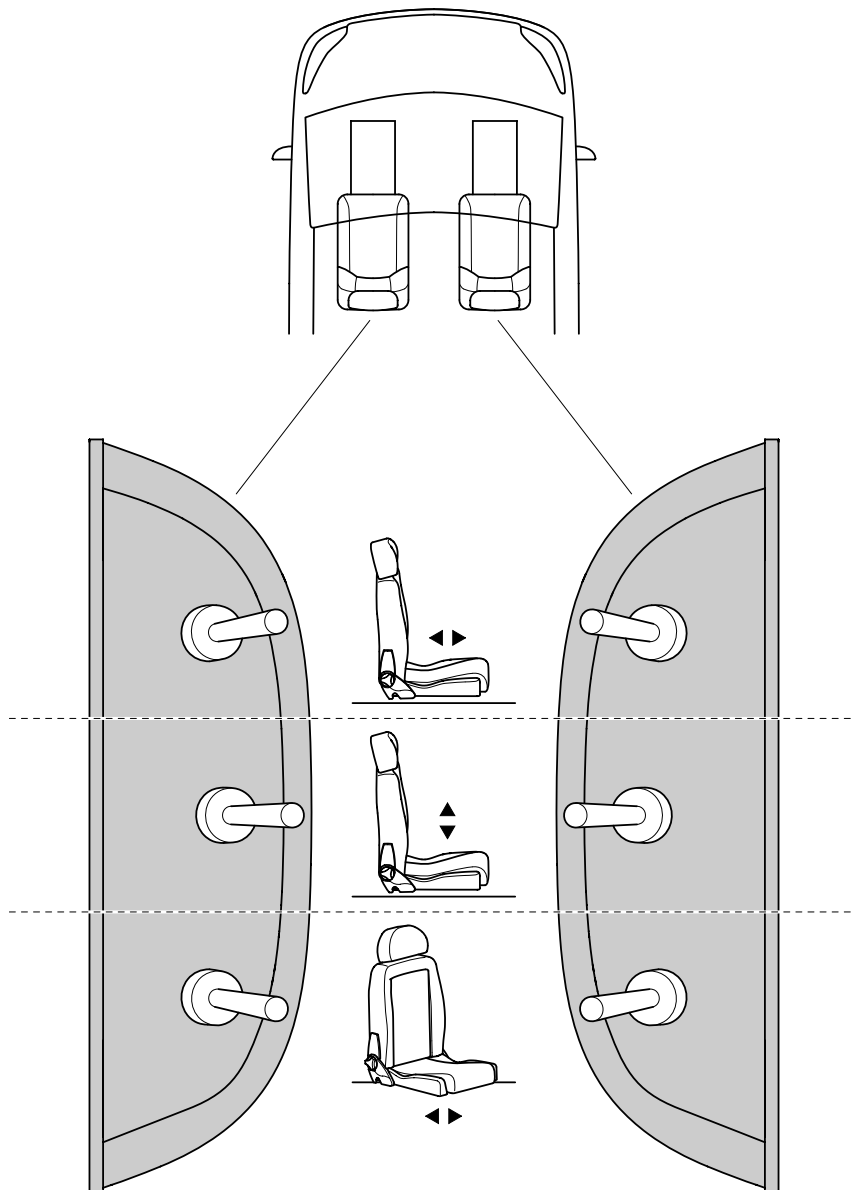427615-3 2023/10

BraunAbility Europe AB
 Åkerivägen 7
 443 61 Stenkullen
 Sweden

Phone: +46 302 254 00
 E-mail: info@braunability.eu
www.braunability.eu

 **BraunAbility**

SV

Viktkapacitet max 180 kg.

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EN

-  *Weight capacity max 180 kg.*
-  *The Forward/Backwards and Up/Down toggle switches are always on. Ensure that they are not accidentally activated while driving. The Left/Right toggle switch is off when the vehicle's ignition is turned on.*
-  *Using the Turny 6-Way without the optional footrest causes a potential pinch risk.*
-  *Any modifications, adjustments or repairs to a Turny 6-Way must be performed by an authorised technician. Always contact your dealer for disassembly.*
-  *The control unit and the toggle switch cover shall not be used as support when transferring to and from the seat.*

Product data

| | |
|---|-----------------|
| Length* | 1150 mm |
| Width | 430 mm |
| Height | 170-347 mm |
| Comfort adjustment | 704 mm |
| Vertical travel | 177 mm |
| Swivel range** | 0-140° / 0-120° |
| Weight capacity*** | 180 kg |
| Unit weight, without seat and accessories | 54 kg |
| Operating voltage | 12V |
| Fuse**** | 20A |

* can be reduced 45-190 mm by installer if needed.
 ** differs between left (140°) and right (120°) side installation.
 *** refers to the total weight of the seat, its accessories and the user.
 **** when replacing, use a 20 A fuse.

Care instructions

| | |
|---------------------------|--|
| As necessary..... | Carefully wipe off and vacuum clean the Turny 6-Way. Do not use any liquids. |
| Once every two years..... | We recommend to contact your dealer for an overhaul. |

Troubleshooting

| | |
|---|---|
| Turny 6-Way is not working..... | Check that / - the fuse has not blown and that the vehicle's battery is charged up. - there is no obstruction blocking the selected movement. If the problem remains, contact your dealer. |
| Forward/backward movement is..... working slower than usually. | Probably time for re-lubrication of the aluminum profiles. Contact BraunAbility Support for further help (lubrication art no 407611). |
| Left/right toggle switch is working..... despite the vehicle's ignition is turned on. | Contact your dealer. |
| Discordant sounds are heard during operation. | Contact your dealer. |

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