JOEY™ VSL-4400

**OPERATOR'S
MANUAL**

**MANUAL
DEL OPERADOR**

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Thank you for purchasing a VSL-4400.

The information contained in this manual is provided with your safety in mind. Please read and follow the instructions to ensure safe and trouble-free operation. Throughout this manual, **safety precautions** are provided to identify potentially hazardous situations and to advise how to avoid them.

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Authorized UK Agent:
BRUNO UK LTD.
Unit 5, Millennium Court
Clayhill Industrial Park
Neston, Wirral CH64 3UZ
UNITED KINGDOM

WARNING: Indicates a potentially hazardous situation that, if not avoided, could cause serious bodily injury and/or property damage.

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CAUTION: Indicates a potentially hazardous situation that, if not avoided, could cause damage to the lift, vehicle and/or moderate injury.

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CAUTION: Indicates a potentially hazardous electrical situation that, if not avoided, could cause damage to the lift, vehicle and/or moderate injury.

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NOTE: Directions that make it easier to install or operate the lift.

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NOTICE TO: Customers in the State of California

WARNING: Electrical cords, cables, product cords, wire assemblies, and other parts made with PVC can expose you to chemicals including DEHP, which are known to the State of California to cause cancer and birth defects or other reproductive harm. WASH HANDS AFTER HANDLING. For more information, go to www.P65Warnings.ca.gov.

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We are pleased to provide you with this mobility-enhancing product.

The Product Registration Form is shipped in a plastic bag with this manual. It is very important that you fill out and return this form at your earliest convenience so that we may complete the warranty registration process for your unit.

The **serial number** is found on labels affixed to the manual, as well as on the unit itself.

Serial and model numbers must be provided when filing a warranty claim, requesting service or ordering parts. We encourage you to keep this information readily available at all times.

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 **Product Registration Form**
Please print in CAPITAL letters.

Online registration available at <http://www.bruno.com/warranty.html>

Please be assured that Bruno **does not** share or sell the information you provide.

* = required field

*First Name _____ M.I. _____

*Last Name _____

*Street Address _____

*City _____ *State/Province _____

*Zip/Postal Code _____

Email Address _____

Telephone No. _____

Date of Purchase (mm/dd/yy) _____

Dealer: Locate the Model No. and Serial No. on the dealer sheet shipped with the unit and place it here.

End-User: Register registration, please go to the website shown above.

Model No. _____

Serial No. _____

Please rate your satisfaction with your Bruno dealer:

excellent good average poor

Please rate your satisfaction with your Bruno product:

excellent good average poor

Bruno reserves the right to use information indicated on this form in its online, video, audio and printed materials. Names will be abbreviated to ensure the privacy and anonymity of the individual.

©2013,2005 Bruno Independent Living Aids, Inc.® Rev. 02-18-2013
P/N PROD REG STD

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NOTE: The VSL-4400 is designed to operate at full rated capacity on a relatively flat, level surface.

It is designed to operate in temperatures above -25°F (-32°C). If operating in extreme heat (over 120°F/49°C) or extreme cold (below -25°F/-32°C), starting the vehicle prior to operating the VSL-4400 is recommended.

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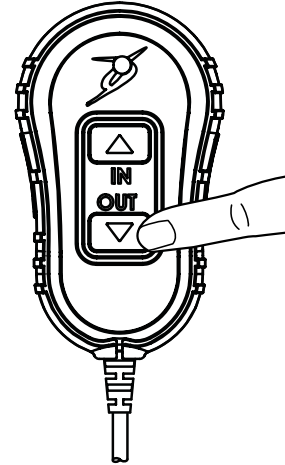
■ PASO 3.1

■ ÉTAPE 3.1

Press the OUT switch and hold until the lift travels out and down to the ground.

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■ PASO 3.2

■ ÉTAPE 3.2

Release the switch when the platform rests flat on the ground.

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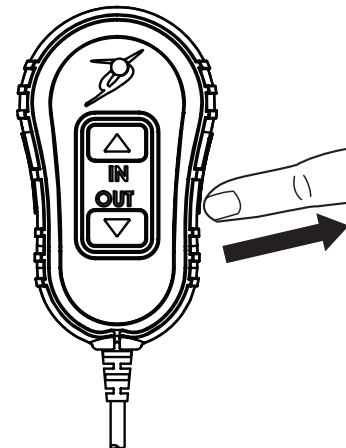
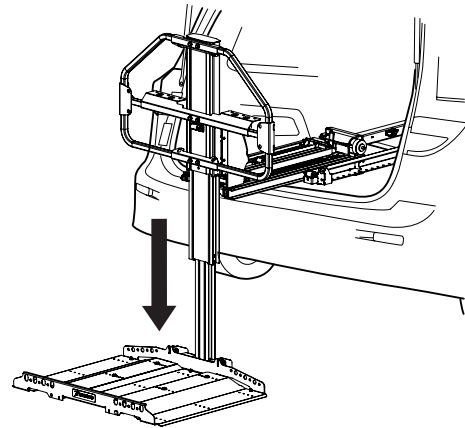
NOTE: Continuing to press the switch after the platform is flat on the ground could cause the lift belt to unwind and reverse the direction of Up/Down.

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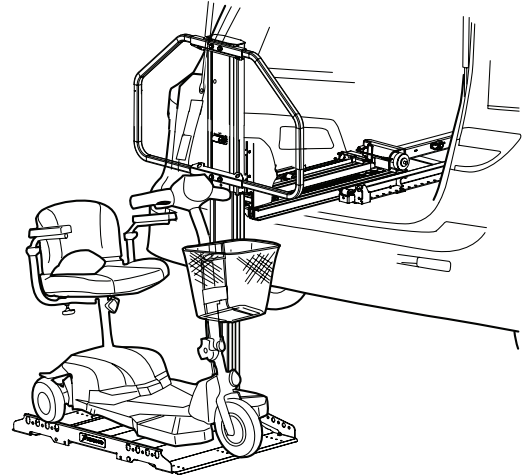
■ PASO 3.3

■ ÉTAPE 3.3

Position the mobility device on the platform.

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■ PASO 3.4

■ ÉTAPE 3.4

Press the IN switch until the lift reaches end of up travel. Release switch.

If equipped with optional barrier belts, place (2) belts across mobility device and hook onto platform. See next page for belt operation.

Press IN switch to power lift into the vehicle. The lift will stop automatically and the mast light will shut off.

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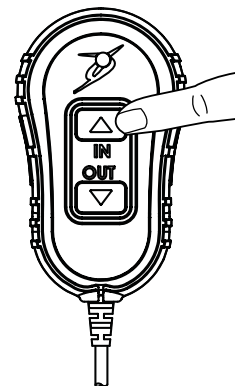
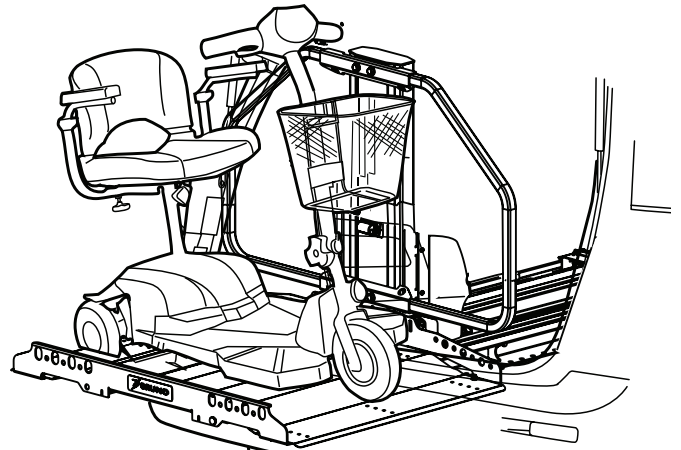
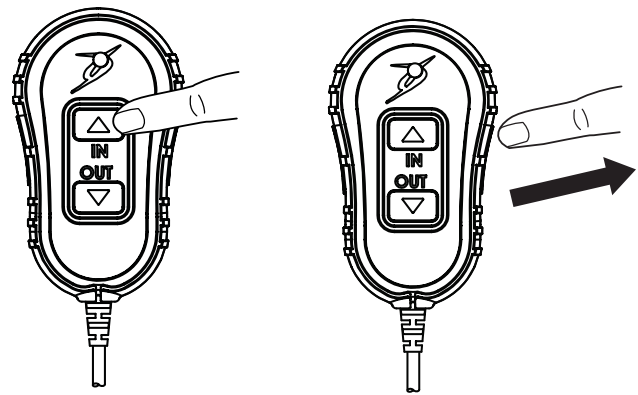
■ PASO 3.4.1

■ ÉTAPE 3.4.1

- Press small tab (A).
- Push up large tab (B).
- Pull strap and hook to platform.

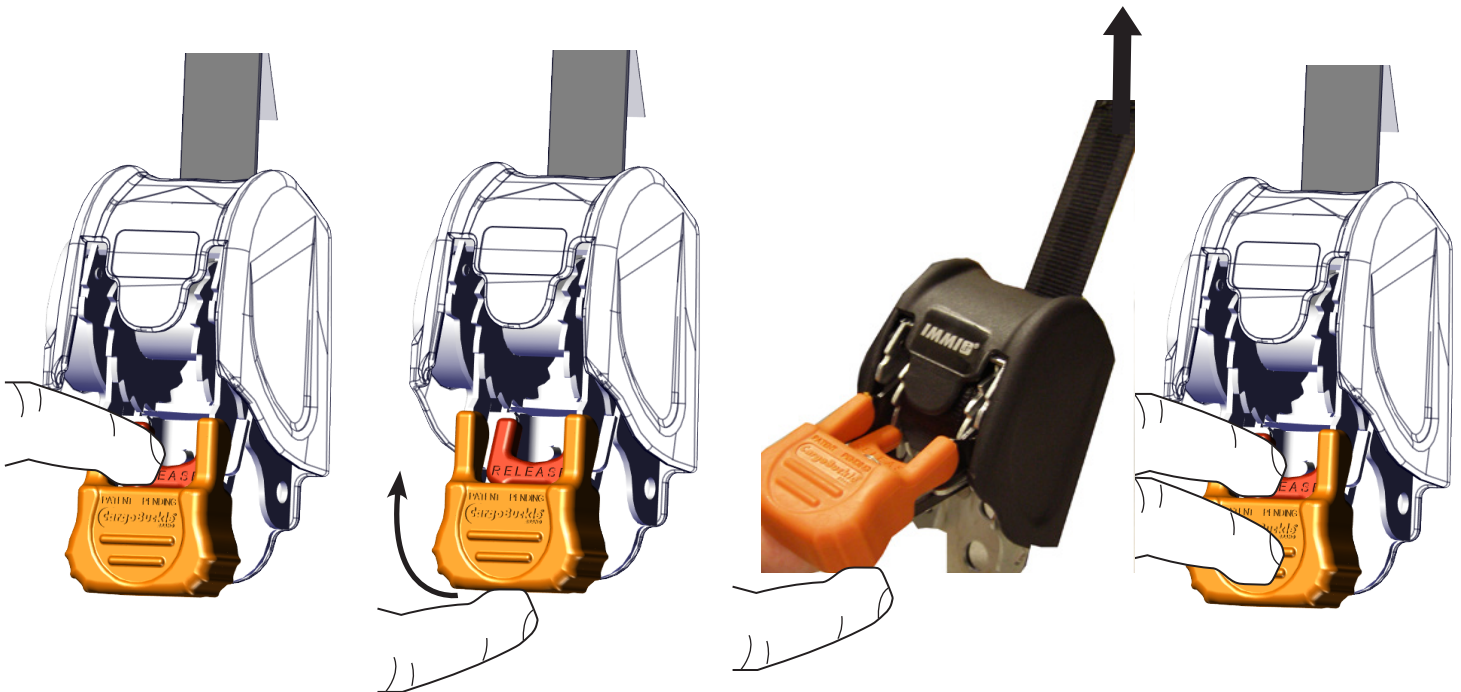
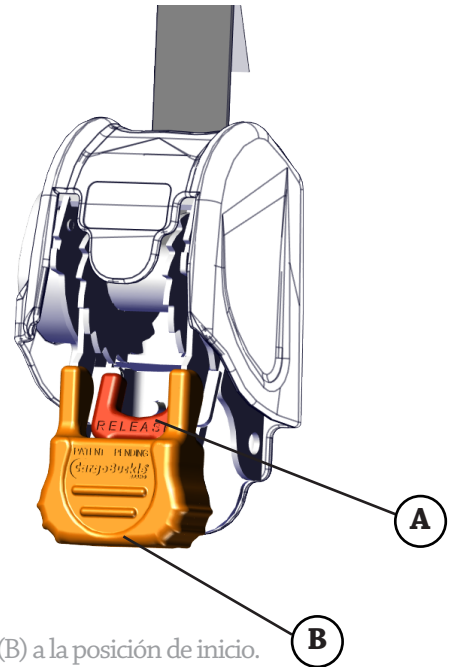
NOTE: You will hear the tab click into its home position. The strap will “auto ratchet” to secure the mobility device.

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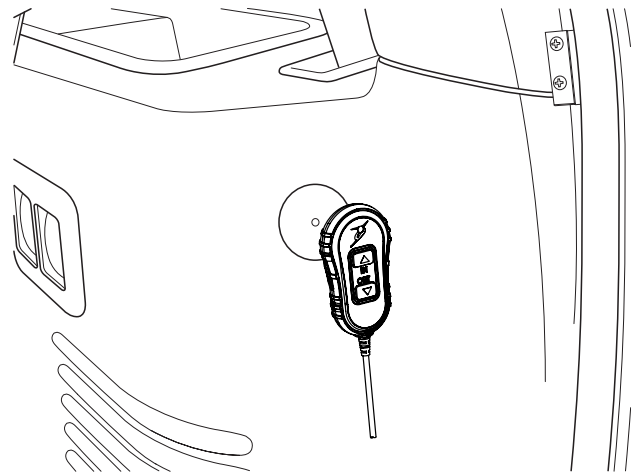
■ PASO 3.5

■ ÉTAPE 3.5

Store the remote on its mounting plate on the vehicle wall.

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
CAUTION: Make sure remote cord is clear from rear hatch to prevent cord from becoming pinched or severed.

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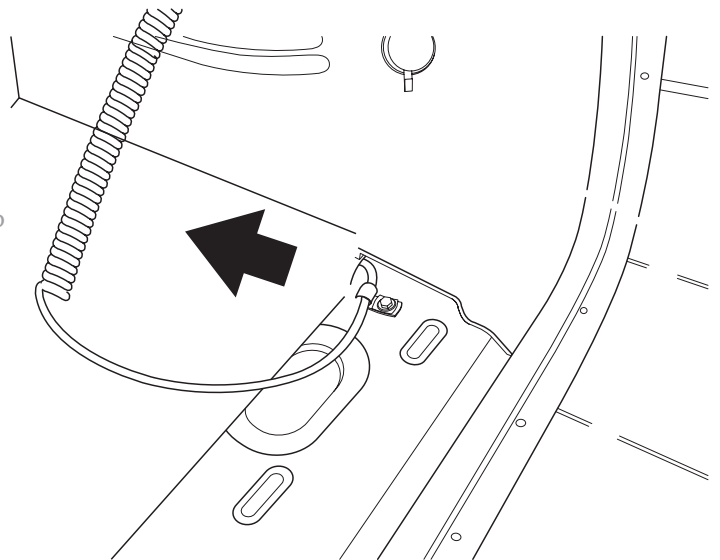■ PASO 4.1

■ ÉTAPE 4.1

Press and hold OUT switch until lift reaches end of outward travel. Release switch.

If equipped with optional barrier belts, unhook (2) belts securing mobility device to platform. See page 9 for belt operation.

Press and hold OUT switch to lower lift to ground. Release switch. Lift will stop automatically and mast light will shut off.

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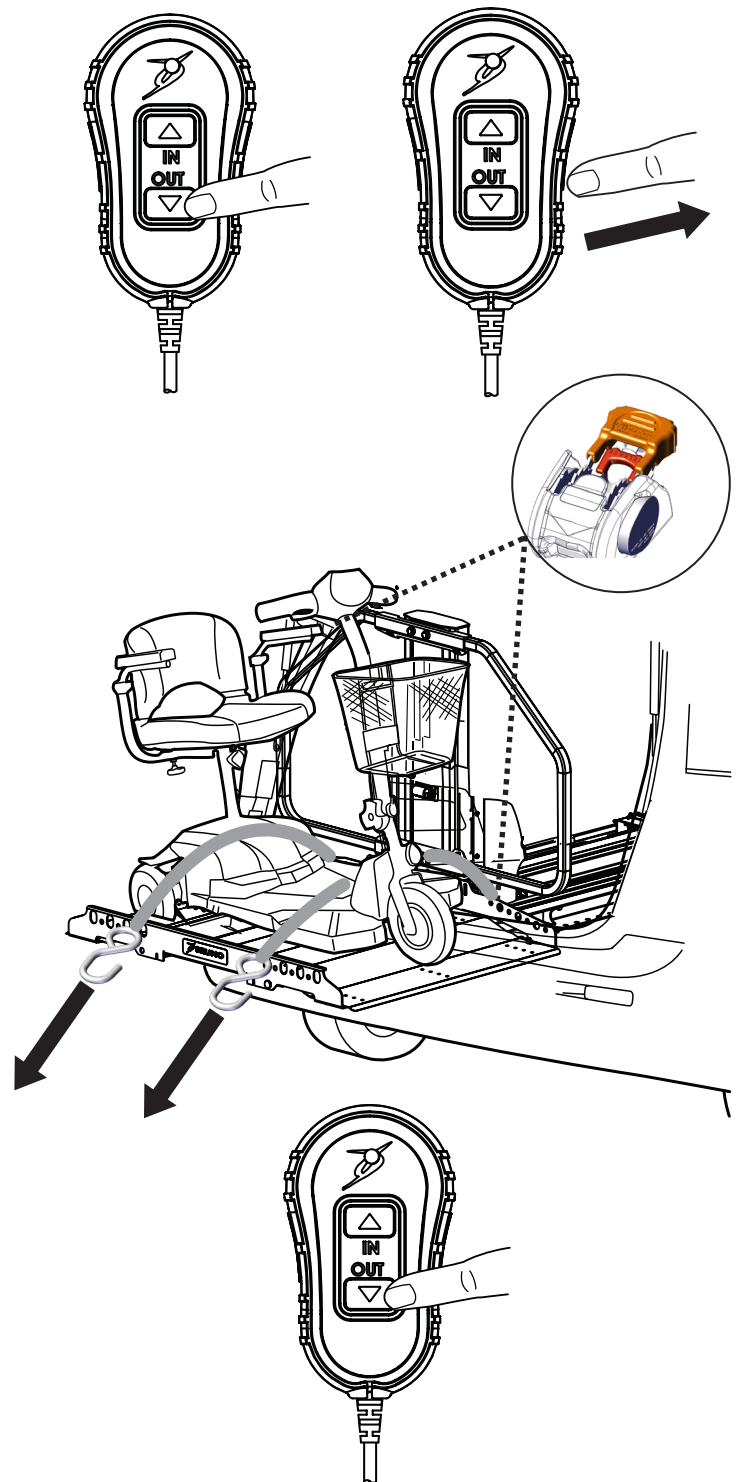
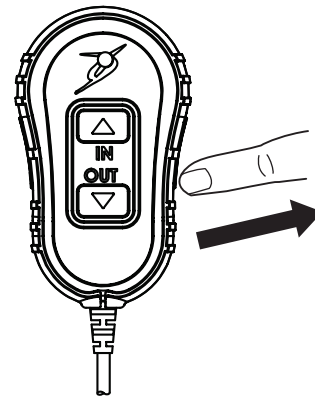
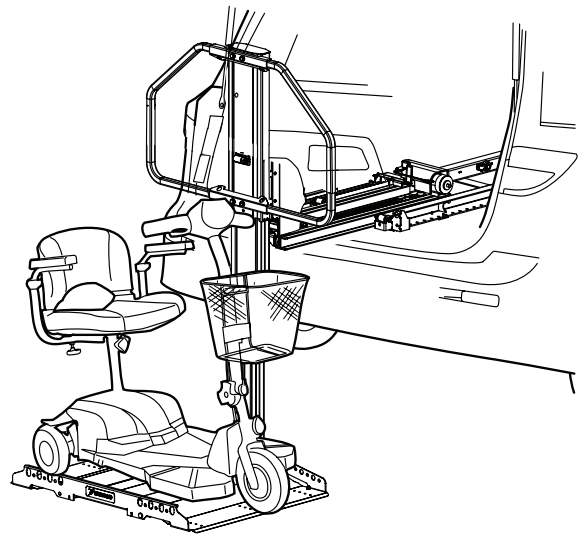
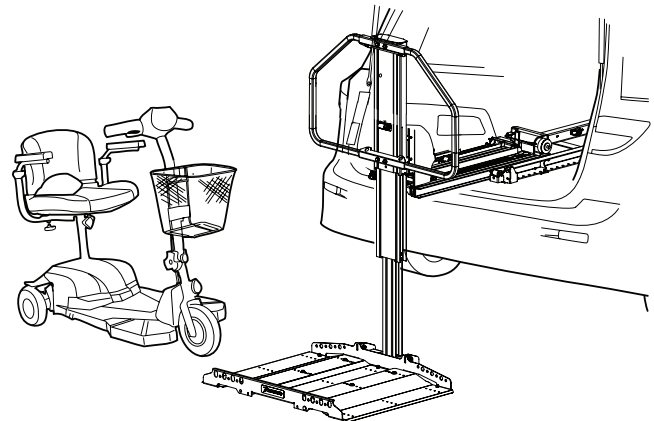
■ PASO 4.3

■ ÉTAPE 4.3

Remove the mobility device a safe distance from the platform.

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■ PASO 4.4

■ ÉTAPE 4.4

Press the IN switch until the lift travels up and into the vehicle. The lift will stop automatically and the mast light will shut off.

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NOTE: The vehicle bumper may raise and/or lower slightly during lift operation with the mobility device on board. Do not be alarmed. This is normal.

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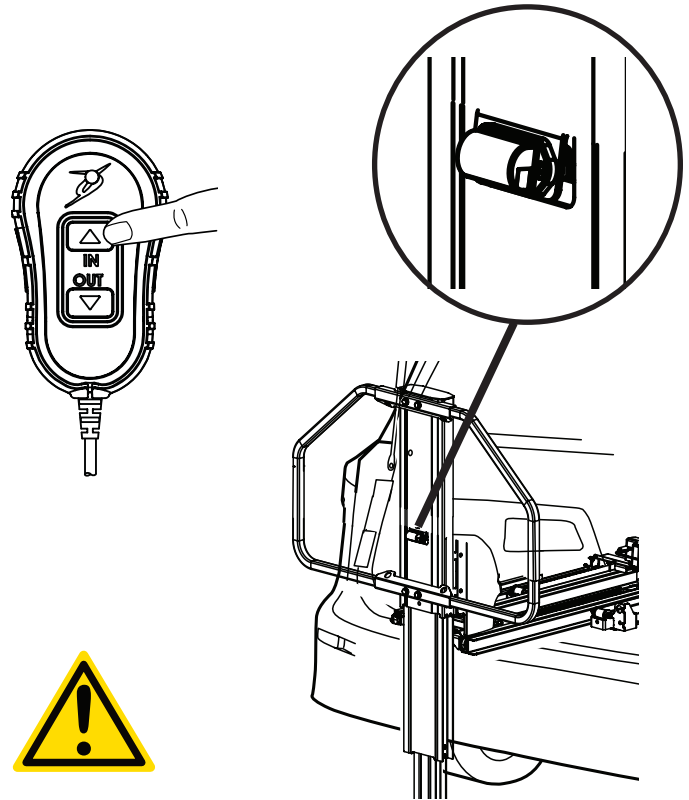
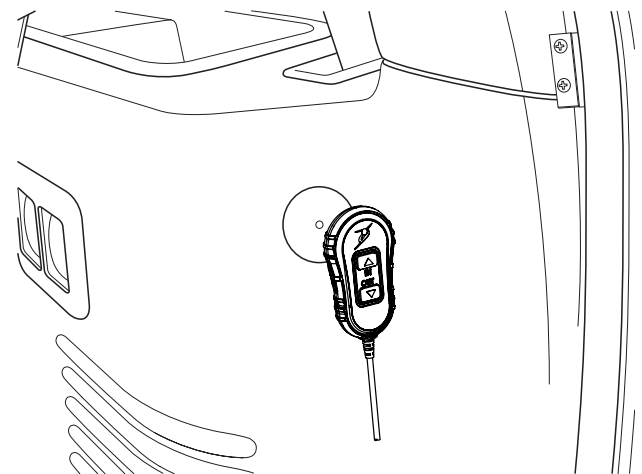
5.0 PINCH POINTS / PUNTOS DE PELLIZCO / POINTS DE PINCEMENT

Keep clear of pinch points.

Up/Down: Where the stages come together as the platform is lifted (A).

Between the barrier and platform (B)

In/Out: Where the inner stage meets up with the rear plate as the unit is retracted to home position (C).

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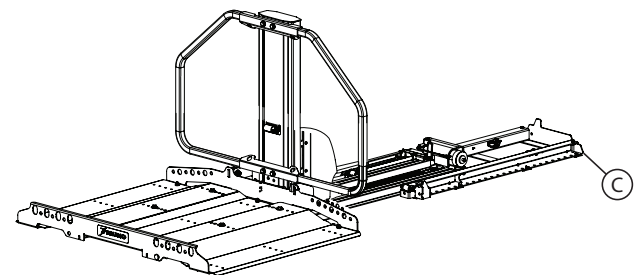
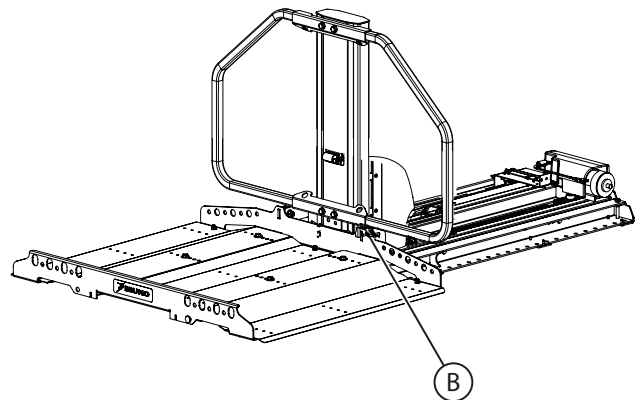
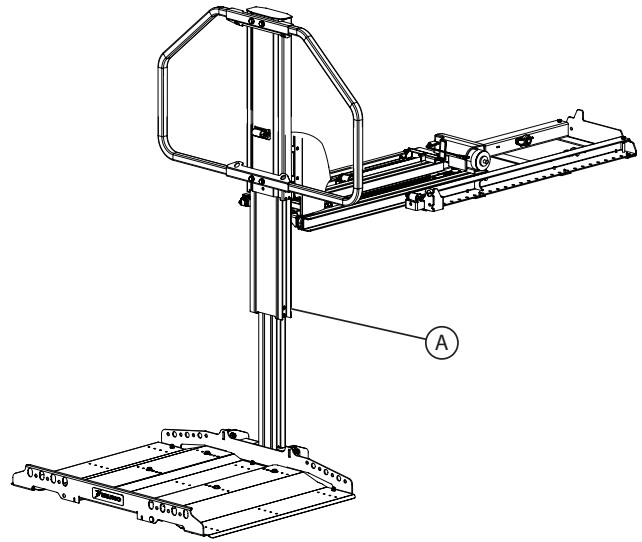
■ PASO 6.1

■ ÉTAPE 6.1

Pull in-line fuse (under hood) before proceeding with emergency back-up.

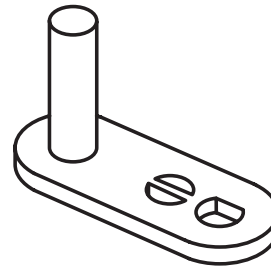
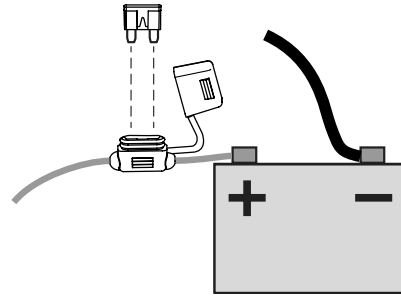
Locate back-up tool from bag containing operator's manual.

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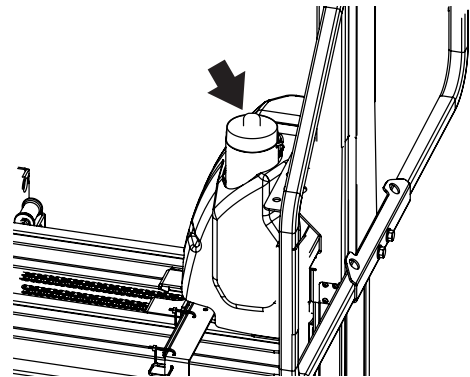
■ PASO 6.2

■ ÉTAPE 6.2

Remove black plastic caps from motor shafts.

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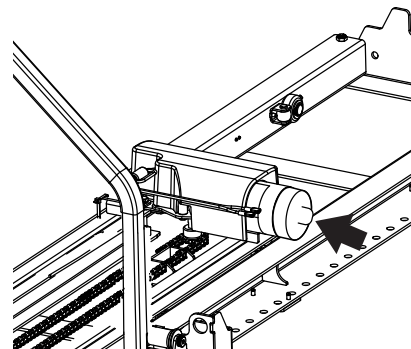
■ PASO 6.3

■ ÉTAPE 6.3

For scooters, secure one chock in front of front wheels and one chock behind rear wheels.

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■ PASO 6.4

■ ÉTAPE 6.4

Place crank on motor #1. Turn crank clockwise to move platform all the way up.

If equipped with optional barrier belts, place (2) belts across mobility device and hook onto platform. See page 9 for belt operation.

Move crank to motor #2.

Turn crank clockwise to move platform all the way in.

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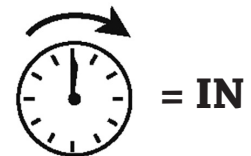
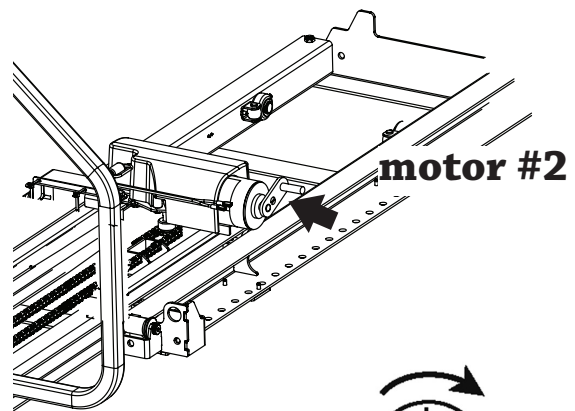
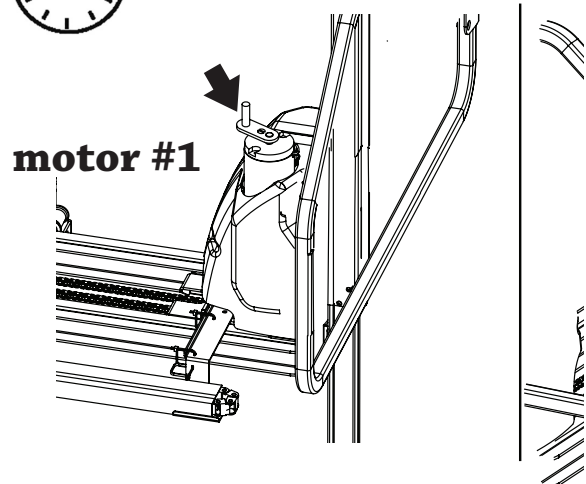
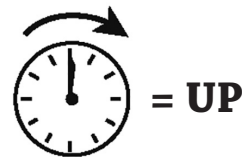
■ PASO 6.5

■ ÉTAPE 6.5

Once stowed, have the lift inspected and repaired by a qualified Bruno dealer before further use.

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7.0 MAINTENANCE AND TROUBLESHOOTING

Take the vehicle and lift to an authorized Bruno dealer for complete inspection and any necessary repairs/replacements once a year (more frequently if the lift is used under strenuous conditions).

PROBLEM	POSSIBLE CAUSE / SOLUTION
Lift does not operate.	+/- power connection to unit is loose. Verify that remote backlight is on. If unit is not in stowed position, verify that mast light is on.
Lift does not operate (will not run with rear vehicle door open)	Verify that the remote backlight is on.
	Dead vehicle battery. Charge, test and replace if necessary.
Lift operates slowly, lacks power or stalls	Weak vehicle battery. Attempt to start vehicle to determine if battery is weak. Charge battery before attempting to operate lift.
	Load is too heavy. Do not exceed maximum rated lift capacity of 350 lbs (159 kg).
	Vehicle parked on uneven ground. Move vehicle to level ground.
	Temperature in vehicle too hot or too cold. Start vehicle engine to lower/raise temperature.
Platform is not moving in direction indicated on remote.	Limit switch accidentally pressed. Reset circuit board by passing back of remote over door safety switch. Remote light should turn off, then back on. Top remote button should move platform UP and IN.
Platform has stopped moving but is not at top or bottom	Have an Bruno dealer inspect the lift and repair if necessary.
Lift travels out but not down	Check the OUT limit switch. Have unit adjusted and repaired if necessary by an Bruno dealer.
Lift stops at end of travel	To reset the printed circuit board (PCB), wave the remote by the door safety switch. The magnet in the remote will reset the PCB.

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DECLARATION OF CONFORMITY

Manufacturer: Bruno Independent Living Aids, Inc.
Address: Headquartered at:
1780 Executive Drive
Oconomowoc, WI 53066 USA
An ISO-9001:2011 registered company
Email: service@bruno.com
Internet: www.bruno.com

Declare that our manufactured Product:

Product Name: Joey **Model:** VSL-4400

Conforms to the following Directives, Standards and Regulations:

ECE R-17
FMVSS 302 and ISO 3795 and 95/28/EG
CISPR 12:2007 A1:2009
ISO 10605:2008 AC1:2008
72/245/EEC, 89/491/EEC, 78/632/EEC, 2000/4/EG
95/54/EC, 74/60/EEC, 2006/42/EC, EN 55 022
EN 12182,

When installed in accordance to the manufacturer's specifications and operated in accordance with manufacturer's operating instructions.

We hereby appoint:
ASCIER SAS
11-13 Rue Charles Cordier
ZA Parc de Bel Air
77164 Ferrières-en-Brie
FRANCE

To act as European Responsible Person as stipulated in 93/42/EEC.

Date of Issue: April 15, 2019
Location: Oconomowoc, WI USA

BY: 
William W. Belson III
Director of Engineering