# Carobrake 2.0

Electric parking brake

## Carobrake 2.0 1600N | Carobrake 2.0 2200N

SV	4	Installationsanvisning / Bruksanvisning
EN	12	Installation instructions / User manual
DE	20	Einbauanleitung / Benutzerhandbuch
FR	28	Manuel d'installation / Manuel d'utilisation
ES	36	Instrucciones de instalación / Modo de empleo
IT	44	Istruzioni per l'installazione / Manuale delle istruzioni

SV

EN

DE

FR

ES

IT

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Thank you for choosing a

# Carobrake 2.0 from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit [www.braunability.eu](http://www.braunability.eu).

Once again, thank you for placing your confidence in our products!

## Disclaimer!

The Carobrake 2.0 1600N and 2200N electrical parking brake system do not in its current state conform to the "Uniform provisions concerning the approval of passenger cars with regard to braking: Regulation No. 13-H revision 3".

The Carobrake System is only permitted to be utilized in vehicles fitted for invalid drivers according to the ECE-R13-H Scope 1.2.2.

### Installation

---

- 14 Presentation
- 15 Three normal scenarios
- 16 Critical elements

### Operation

---

18

Wiring diagram can be found at the end of the manual.

# Installation

## Presentation

Carobrake tensions the vehicle's original parking brake cable. Note that performance deteriorates if the product's cable is installed bent. Carobrake replaces the function of the handbrake lever and should not be used in combination with it. The actuator itself can be fitted in or under the vehicle, depending on the prevailing circumstances.


Carobrake has been tested and approved for handling 30,000 cycles with a 5-minute rest between each cycle.

Ensure the force is sufficient for the intended type of vehicle, fitting and use before installation.

	<i>1600N</i>	<i>2200N</i>
<i>Stroke length</i>	50	50
<i>Cable length</i>	700	700
<i>Optimum force with straight cable</i>	*1600N / 4A	*2200N / 4A
<i>Fuse</i>	10A	10A

\*Deviation: +/- 10% at +25 °C

## Three normal scenarios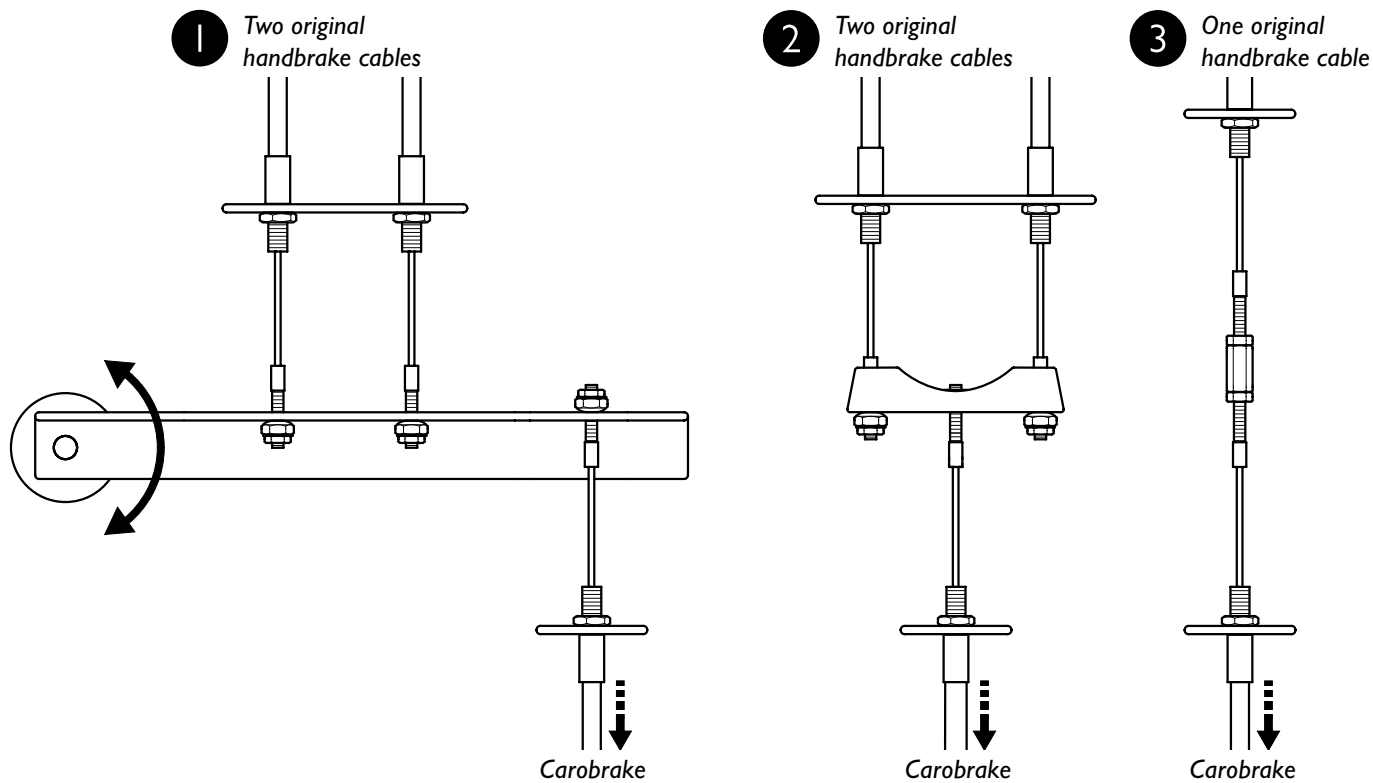

## Critical elements

### For a stable and sustainable installation /

**A,** Position all cables as much as possible in parallel to each other. The straighter the installation, the better the performance and the lesser the wear.

**B,** Install the cable in some form of console (not included with the product) in order to keep a straight and stable installation when the construction moves.



**C,** Use nuts with rounded heads on all cable attachments. This will allow the attachment to roll against the nuts when the construction moves. Nuts with flat heads will not allow any flexibility, whereby the cables will bend and risk breaking.

**D,** Do not tighten the nuts on any of the cable attachments. This will allow the cables to move freely as the construction moves. Tightened nuts will not allow any flexibility, whereby the cables will bend and risk breaking.


**E,** Avoid bending the cable as this will impair performance and increase wear on the cable. If the cable despite this must be bent, the radius of the bend must not be less than 300 mm.

**F,** Place the actuator so that no mechanical impact is exerted on the pin. This would involve a risk of the pin coming loose and falling out. The actuator should not be placed in the vicinity of hot components either, such as the exhaust pipe.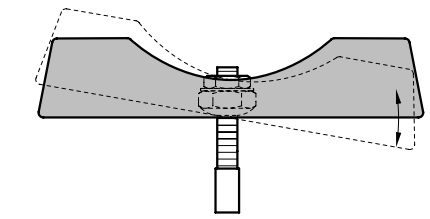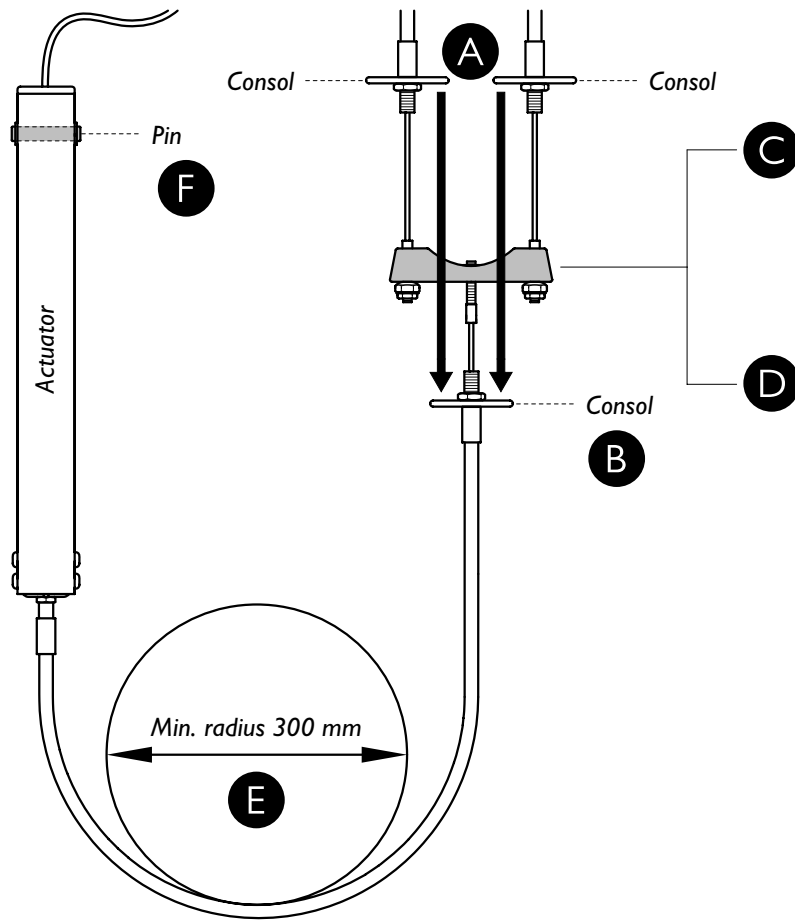


Items C and D apply to all cable attachments.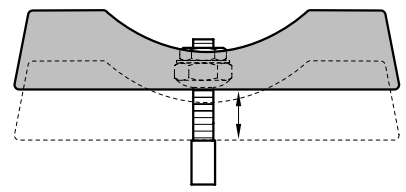

Items C and D give the construction flexibility and ensure the cables remain straight when it moves.

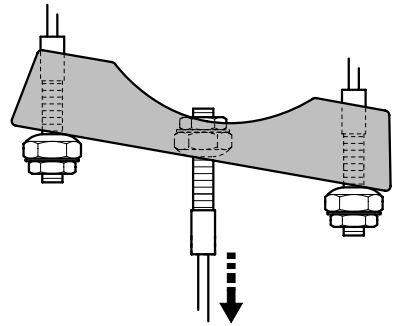## Operation

Carobrake replaces the function of the handbrake lever and should not be used in combination with it.

### To apply the parking brake

Press the switch to activate Carobrake. The lamp in the switch lights up straight away but there will be a 5 - 15 second delay before the parking brake is applied.

### To release the parking brake

Press the switch to deactivate Carobrake. The lamp in the switch goes out straight away but there will be a 5 - 15 second delay before the parking brake is released. Do **not** put the vehicle in motion before the parking brake is released.

*The Carobrake switch is located in the driver's area.*

- The product documentation shall always accompany the vehicle for an MOT inspection.
- Always contact your installer when dismantling.

Please contact your installation engineer if you require further assistance or need to ask questions about warranty. We recommend keeping the following information to hand at all times /

Contact details for the installation engineer /

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

---

## Wiring diagram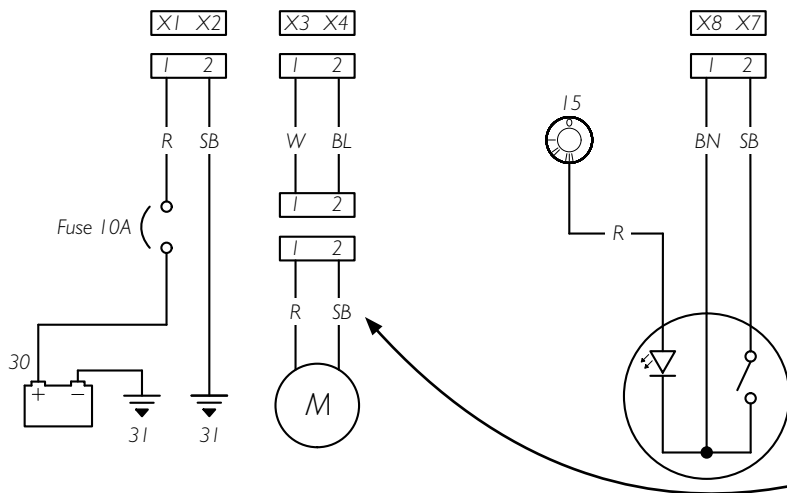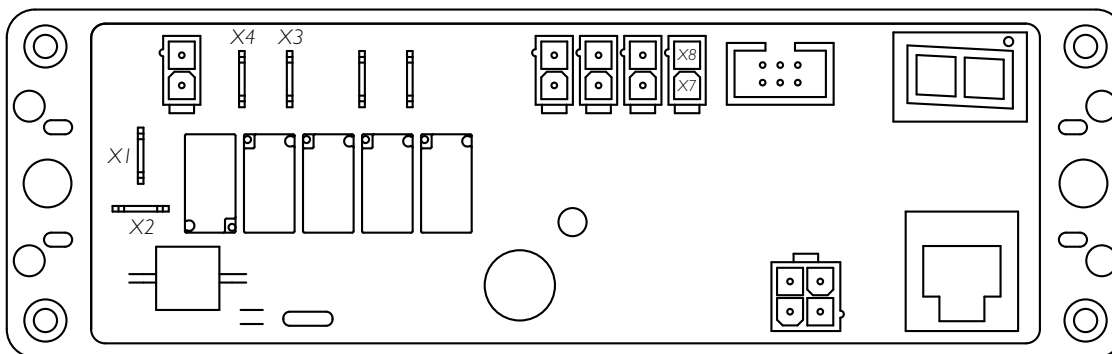

If you need to shorten the cable to the motor:  
 Make sure you do not switch polarity. Cable colours may vary and will not always follow the standardized colouring scheme. If polarity is switched Carobrake will still be operational but only at half the power.

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2018  
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.  
BraunAbility Europe AB reserves the right to make alterations without previous notice.  
© 2022 BraunAbility Europe AB

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)