# 6-Way Base

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Thank you for choosing a

# 6-Way Base from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit [www.braunability.eu](http://www.braunability.eu).

Once again, thank you for placing your confidence in our products!

---

**Safety**

---

18 Safety measures

**Operation**

---

19 Left hand side

20 Right hand side

**Care instructions**

---

21

**Troubleshooting**

---

22

**Technical specification**

---

24 Technical data

25 Components

**Introduction for users**

---

26

“Product certificate” at the end of the manual.

## Safety

### Safety measures

*Max user weight is 150 kg.*

*6-Way Base cannot be manoeuvred while the car's ignition is on.*

*The footplate should be used if the user requires foot support, to avoid any risk of pinching.*


*The wheelchair/scooter must be anchored to the floor of the car while it is moving.*


*Any modifications, adjustments or repairs to a 6-Way Base may only be performed by authorised technicians.*

# Operation


## Left hand side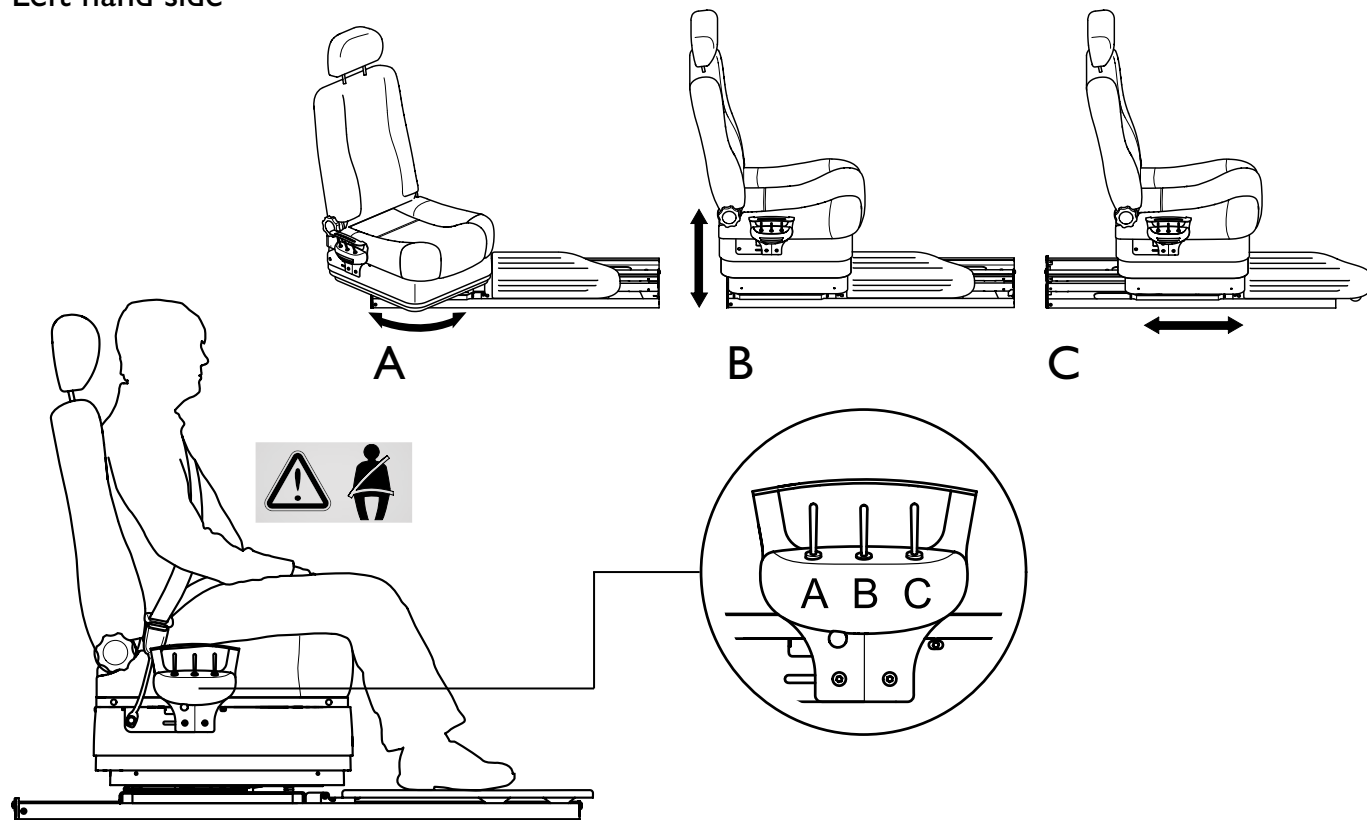


Right hand side


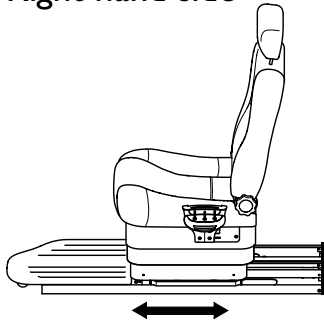
C

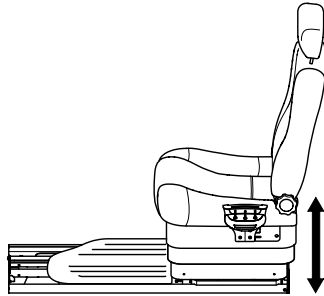B

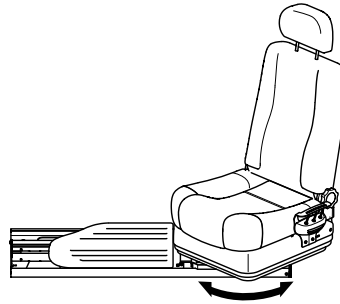A
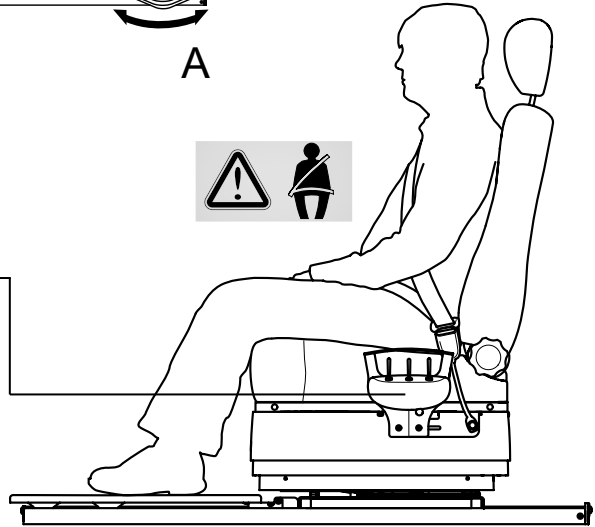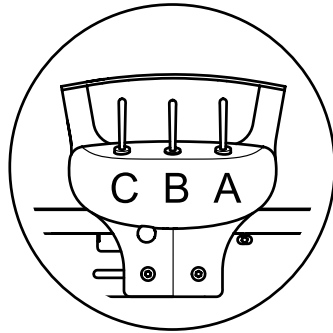
# Care instructions

When necessary .....	Clean the control unit, footplate and base plate, see page 25.
Once every two years .....	Contact your installer, see the servicing information at the end of the manual.

- The product documentation must always be available during inspection of the vehicle.
- Always contact your dealer for disassembly.

## Troubleshooting

6-Way Base is not working .....

Check that /

- the car's ignition is switched off.
- the fuse has not blown and that the car battery is charged up.
- there is no obstruction blocking the selected movement.

6-Way Base is working .....  
despite the car's ignition being on.

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Discordant sounds are heard .....  
during movement.

Contact your installation engineer.

Please contact your installation engineer if you require further assistance or need to ask questions about warranty. We recommend keeping the following information to hand at all times /

Contact details for the installation engineer /

Model /

Serial number /


Year of manufacture /

Date of purchase /

Notes /

# Technical specification

## Technical data

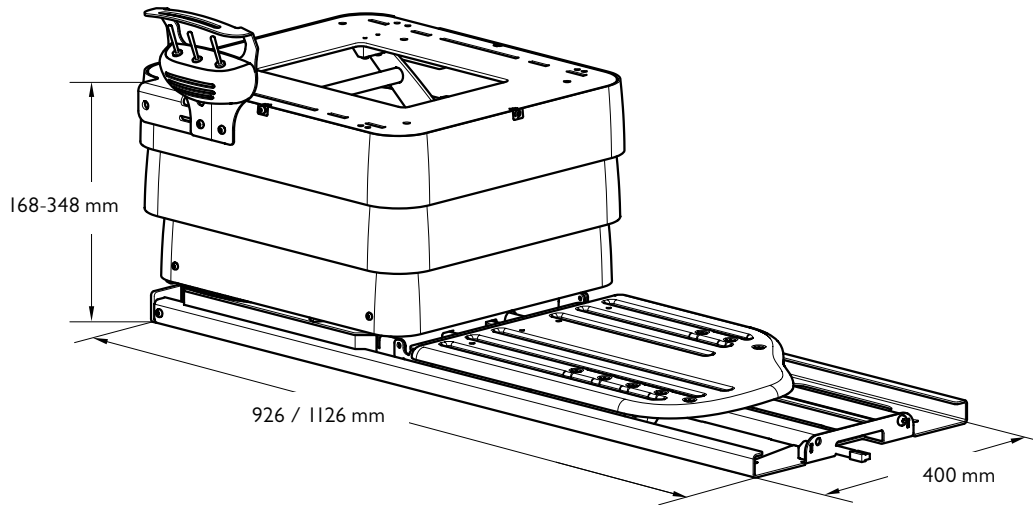
Height..... 168 mm  
 Heightwise transport ..... From 168 to 348 mm  
 Width..... 400 mm  
 Base plate short..... 926 mm  
 Horizontal transport, base plate short ..... 514 mm

Base plate long ..... 1126 mm  
 Horizontal transport, base plate long..... 714 mm  
 Angle of rotation ..... 140 °  
 Max user weight..... 150 kg  
 Fuse..... 30A

## Components

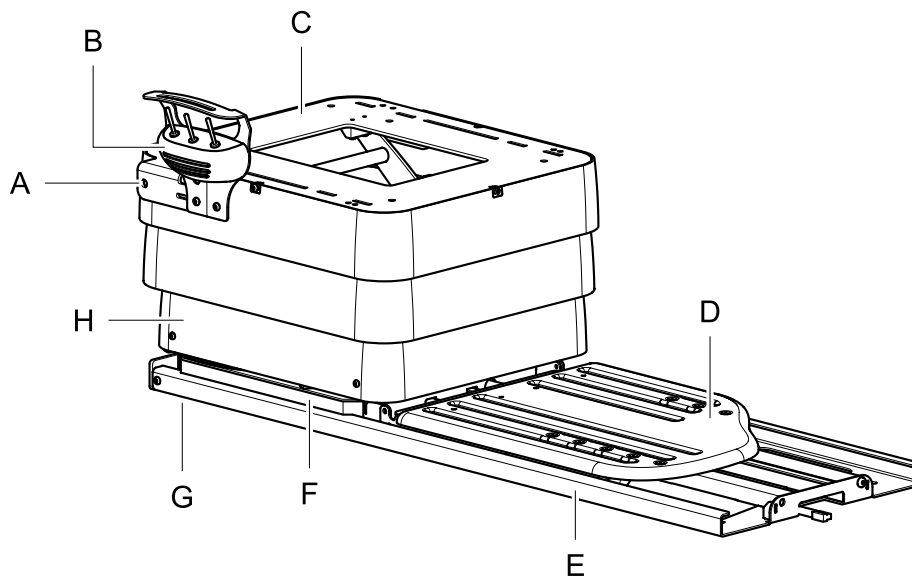A.....Crash tested seatbelt mounting  
 B.....Control unit  
 C.....Plate for seat attachment  
 D.....Footplate (option)  
 E.....Base plate

F.....Slide rails  
 G.....CE label  
 H.....Protective covers

## Introduction for users

BraunAbility Europe AB would like you, as a user, to feel secure and comfortable with your new product. For that reason, it is your installer's duty to go through the following with you /

Safety  
Operation

If any questions about the product should come up later on, you are of course very welcome to contact your installation engineer again.

The above chapters have been gone through /

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## Declares that the product /

6-Way Base,  
model right/left, 1000/1200

## Conforms to following directives, standards and regulations /

2006/42/EEC  
2007/46 EEC  
2007/47/EEC,  
LVFS 2013:11  
ECE R-10  
ECE R-14  
ECE R-16  
ECE R-17 rev 5 (incl supplement 08)  
ECE R- 21  
ISO 3795

BraunAbility Europe AB, Stenkullen  
2019-04-02

Jan M Jensen  
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005  
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.  
BraunAbility Europe AB reserves the right to make alterations without previous notice.  
© 2019 BraunAbility Europe AB

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)