


## How to set up the BraunAbility Remote app

1. Download the BraunAbility Remote app from Google Play or App Store.
2. Activate Bluetooth on your smartphone.
3. Run the Turny Evo to a position where the Teach-in button (A) on the receiver is accessible. You have to lift the curtain to access the button.
4. Press the teach-in button x3 and follow the setup wizard in the app.

*Turny Evo from above, swiveled out to the right.*

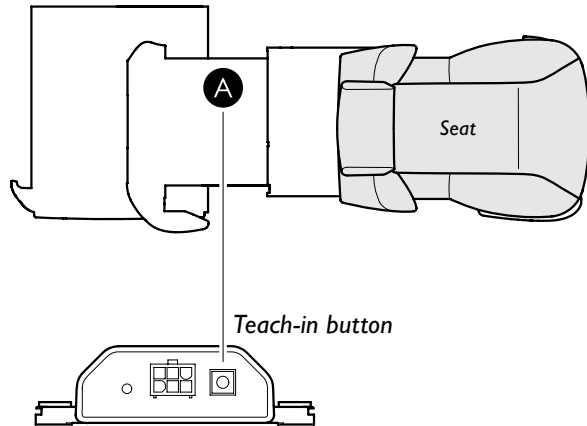


*Before using this device, download and read the user manual available at: [www.braunability.eu](http://www.braunability.eu)*

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)


427332-5 2019/04

 **BraunAbility®**

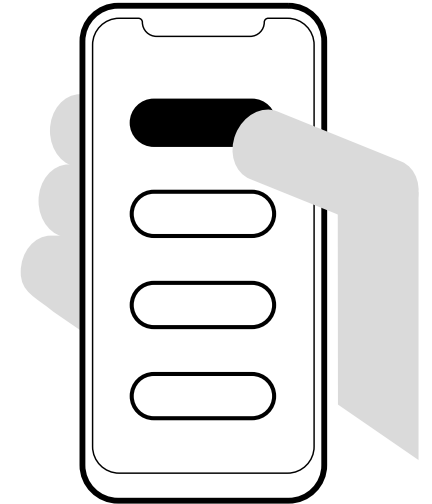

## BRAUNABILITY REMOTE

Quick guide for Turny Evo

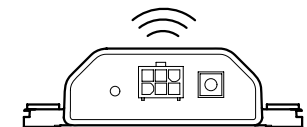

## Install the receiver to a Turny with six wires

1. Swivel and extend the Turny Evo as far out as possible.
2. Unscrew the curtain and feed it back in its roll. This will keep it from getting caught during the rest of the installation.
3. Attach the supplied cable to the Turny Evo (A) by connecting the coloured cables to their matching counterpart using the splices.
4. Connect the receiver to the supplied cable (B). Attach the receiver at the position (C) using the double sided adhesive pads.
5. Carefully loop the cables and attach them underneath the swivel motor using the cable tie. Ensure that no cables are pinched when the unit is in motion.
6. Run the Turny Evo to a position where the protective cover of the control board (D) is fully visible.
7. Remove the protective cover for the control board.
8. Locate the cable (E) for the Turny Evo remote interface next to the swivel motor and attach the connector (F) to the control board. Reattach the protective cover.
9. Pull out the curtain and reattach with the screw.

## Install the receiver to a Turny with one socket

1. Swivel and extend the Turny Evo as far out as possible.
2. Unscrew the curtain retaining screw and feed curtain back in its roll. This will keep it from getting caught during the rest of the installation.
3. Connect the receiver to the socket (B) on the Turny Evo. Align the receiver with the holes at the position (C). Fasten using the supplied screws and spring washers.

4. Carefully route the cables and secure them underneath the swivel motor using the cable tie. Ensure that no cables are pinched when the unit is in motion.
5. Pull out the curtain and reattach with the screw.

