


## How to set up the BraunAbility Remote app

1. Run the Chair Topper until the Teach-in button on the receiver is accessible.
2. Download the BraunAbility Remote app from Google Play or App Store.
3. Activate Bluetooth on your smartphone.
4. Press the Teach-in button (A) quickly three times in a row and follow the setup wizard in the app.

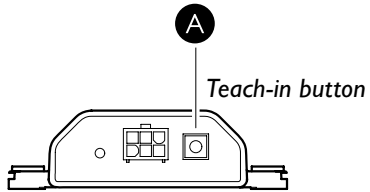

*Before using this device, download and read the user manual at one of the following websites:*

[www.braunability.com](http://www.braunability.com)  
[www.braunability.eu](http://www.braunability.eu)

**BraunAbility**  
631 West 11th Street  
Winamac, IN 46996  
USA.


Phone: +1 574 946 6153  
Fax: +1 574 946 4670  
[www.braunability.com](http://www.braunability.com)

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)


427414-2, 2020/12

 **BraunAbility**<sup>®</sup>

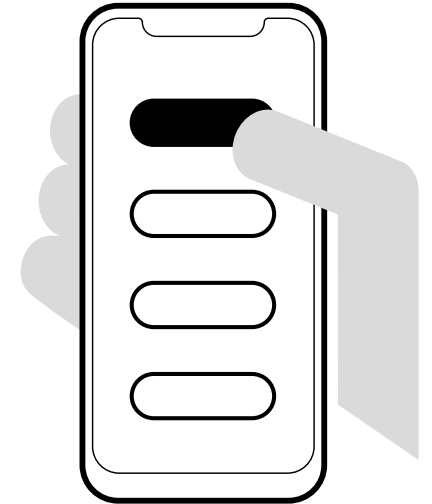

## BRAUNABILITY REMOTE

Quick guide for Chair Topper

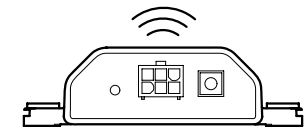

## How to install the BraunAbility remote receiver

1. Run the Chair Topper far enough to be able to mount the receiver next to the Control board box located behind the cover.
2. Place the bracket flush against the Control board box, according to the illustration, and mark the position of the three holes. Drill 5.2 mm (13/64) holes through the plastic bottom of the Chair Topper.



*Be careful. Check for obstructions to ensure free space before drilling.*

3. Mount the receiver using the supplied adhesive pads and clips. Fit the supplied rubber trims to protect existing cables from damage.
4. Disconnect the power supply cable and the standard hand held control from the Control board box. Connect the supplied adapter cable to the control box and reconnect the two cables. Depending on the Chair Topper's edition it might be necessary to disassemble the Control board box to be able to perform this step.
5. Mount the bracket in the holes drilled in step 2, using the supplied screws, washers and nuts.
6. Connect the adapter cable to the receiver. Route the cable as illustrated depending if it's left or right model. For the left model secure the adapter cable in the cable clip together with the other cables.

