# TURNY<sup>®</sup> evo

(SV) Bruksanvisning .....	03
(EN) User manual .....	23
(DE) Benutzerhandbuch .....	43
(FR) Manuel d'utilisation.....	63
(ES) Modo de empleo.....	83
(IT) Manuale delle istruzioni...	103
(NO) Bruksanvisning .....	123
(PT) Manual do usuário.....	143

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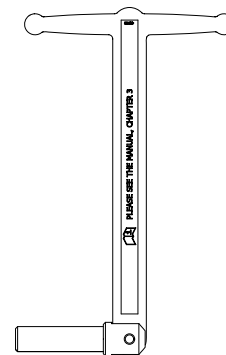
.....

.....

## **Congratulations on your purchase of an adaptation solution from Autoadapt!**

*Thank you for choosing us as the supplier of your car adaptation. This product is designed to make it easier for you to drive your car, and we have focused particularly on safety, quality and functionality. The following manual is an integral and important part of the product, providing you with information on how to achieve maximum performance and safe operation. So, keep the manual so that you can refer to it when necessary.*

*If you have any questions about your adaptation, please feel free to contact your installer. If you wish to share your opinions about the product or the company with us there is the possibility of registering this on [www.autoadapt.com/register](http://www.autoadapt.com/register). That way, you will be helping us to become a better supplier. By way of thanks, we are offering you a subscription to our electronic newsletter, free of charge.*

*Once again, thank you for placing your confidence in our products!*

*Autoadapt*

<b>1 Safety</b> .....	24
<b>2 Operation</b> .....	25
2.1 The manual control's indicators.....	25
2.2 Operating out and in.....	30
2.3 Lengthwise adjustment.....	31
<b>3 Emergency-mode operation</b> .....	32
3.1 Electrical.....	33
3.1 Manual.....	34
<b>4 Care instructions</b> .....	37
<b>5 Troubleshooter</b> .....	38
<b>6 Technical specification</b> .....	39
6.1 Technical data.....	39
6.2 Components.....	40
<b>7 Introduction for users</b> .....	41

**Servicing information at the end of the manual**

# 1 Safety

*Ensure that the car is standing on a level surface and that the parking brake is actuated before Turny is used.*

*For information about max user weight, see page 39.*

*Please note that there is still a pinching risk despite Turny's safety functions.*

*Ensure that the cable for the manual control is freely suspended to minimise the risk of injury.*

*If possible, use the footrest to make operation in and out of the car easier.*

*After a manual emergency-mode operation, an authorised workshop must carry out an overhaul of Turny. Any faults must first be remedied before the product is brought back into use.*

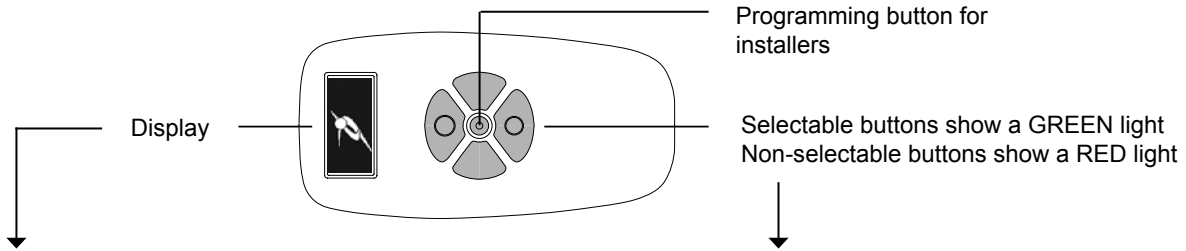
*Ensure that no one is sitting in the seat behind Turny when operating into or out of the car.*

## 2 Operation

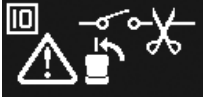
### 2.1 The manual control's indicators

Messages during normal operation		Possible button selections
	Welcome message.	
	Open door. Chair adjusted forwards.	Left – move backwards. Up – move backwards. Right – drive forward. Down – rotate and lower.
	Open door. Rotation out commenced.	Up – raise and rotate in. Down – rotate and lower.
	Closed door. Chair adjusted forwards.	Left – move backwards. Up – move backwards. Right – drive forward.

	<p>Open door. All movements are in the zero position, i.e. the chair has been lifted max up and in, rotated max in and moved max back.</p>	<p>Right – drive forward. Down – rotate and lower.</p>
	<p>Open door. Chair max down.</p>	<p>Up – raise and rotate in.</p>

<b>Messages on error, numbered 1-18.</b>		<b>Remedial actions.</b>
	<p>Low battery voltage.</p>	<p>Charge the battery.</p>
	<p>Cable brake in the manual control.</p>	<p>Change cable</p>
	<p>The back support is leaning too far back.</p>	<p>Raise the back support.</p>
	<p>1 Lengthwise adjustment motor overloaded. 2 Fault on pulse sensor for longitudinal adjustment motor.</p>	<p>1 Check that nothing is obstructing the movement. 2 Drive in the emergency-mode operating position, then contact the installer.</p>
	<p>1 Rotation motor overloaded. 2 Fault on pulse sensor at rotation motor.</p>	<p>1 Check that nothing is obstructing the movement. 2 Drive in the emergency-mode operating position, then contact the installer.</p>

	<p>1 Lifting motor overloaded. 2 Fault on pulse sensor for lifting motor.</p>	<p>1 Check that nothing is obstructing the movement. 2 Drive in the emergency-mode operating position, then contact the installer.</p>
	<p>Pinch protection actuated for rotation motor. In the event of any obstruction, Turny will stop and reverse a short distance.</p>	<p>Remove the obstruction and try again.</p>
	<p>Pinch protection actuated for lifting motor. In the event of any obstruction, Turny will stop and reverse a short distance.</p>	<p>Remove the obstruction and try again.</p>
	<p>Pinch protection actuated for lengthwise adjustment motor. In the event of any obstruction, Turny will stop and reverse a short distance.</p>	<p>Remove the obstruction and try again.</p>
	<p>Cable break or limit sensor for rotation motor out of service.</p>	<p>Drive in the emergency-mode operating position, then contact the installer.</p>
	<p>Cable break or limit sensor for longitudinal adjustment motor out of service.</p>	<p>Drive in the emergency-mode operating position, then contact the installer.</p>
	<p>Cable break or limit sensor for lifting motor out of service.</p>	<p>Drive in the emergency-mode operating position, then contact the installer.</p>
	<p>Overloading on rotation motor.</p>	<p>1 Check that nothing is obstructing the movement. 2 Test without anyone in the chair. Contact the installer if the problem remains.</p>

	Overloading on lengthwise adjustment motor.	1 Check that nothing is obstructing the movement. 2 Test without anyone in the chair. Contact the installer if the problem remains.
	Overloading on lifting motor.	1 Check that nothing is obstructing the movement. 2 Test without anyone in the chair. Contact the installer if the problem remains.
	Rotation cannot be actuated until the door has been closed.	Open the door.
	Lengthwise adjustment cannot be actuated until rotation into the car has been completed.	Press the green button to rotate into the car and complete the step.
	The door is closed but rotation into the car is not complete.	Press the green button to rotate into the car.

Messages during emergency-mode operation	Possible button selections
--	----------------------------

	Warning message.	Down – step down through the text. Right – confirm that you have read it.
	Emergency-mode operating position for lifting.	Left – lift up. Right – lower. Up – change to another movement.

Emergency-mode operating position for rotation.

Left – rotate in.  
Right – rotate out.  
Up – change to another movement.

Emergency-mode operating position for lengthwise adjustment.


Left – reverse backwards.  
Right – drive forward.  
Up – change to another movement.

## 2.2 Operation out and in
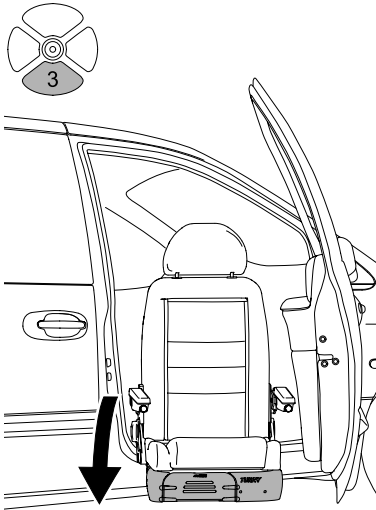
### Operation out.


*Mind your head! Ensure that no clothing, etc, is hanging down, risking becoming caught in Turny during operation into, or out of the car.*

### Sit in the chair.

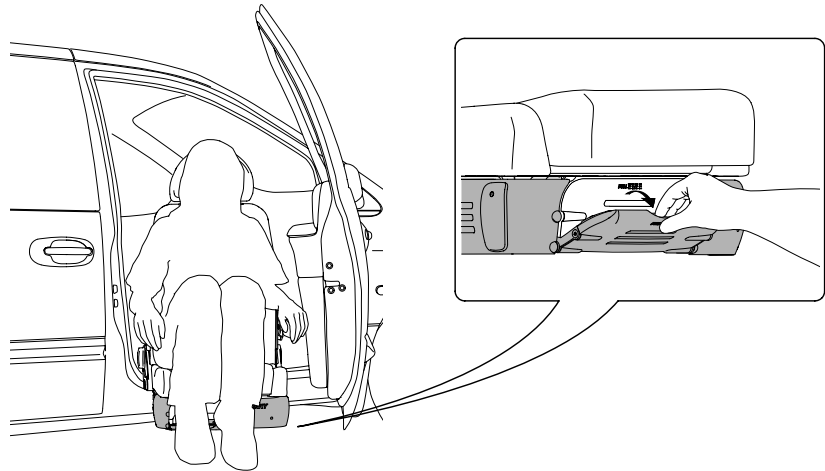If possible, use the footrest to make operation in and out of the car easier.



## 2.3 Lengthwise adjustment

### Operate in

Selectable buttons show a GREEN light.


Keep the button pressed down. When the chair reaches its end position, it automatically stops. To interrupt the movement, release the button.

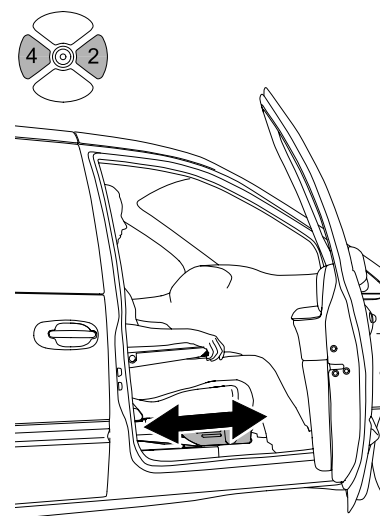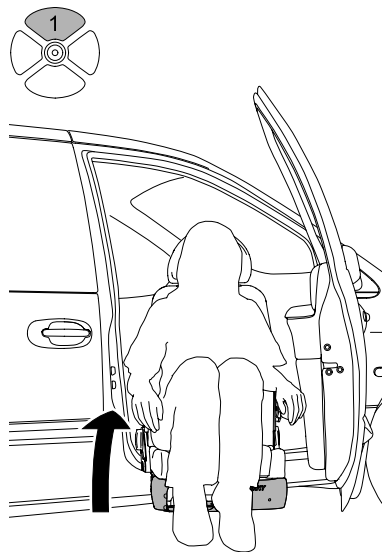To get out of the car again, look under the heading “Operating out”.

### 3. Emergency-mode operation

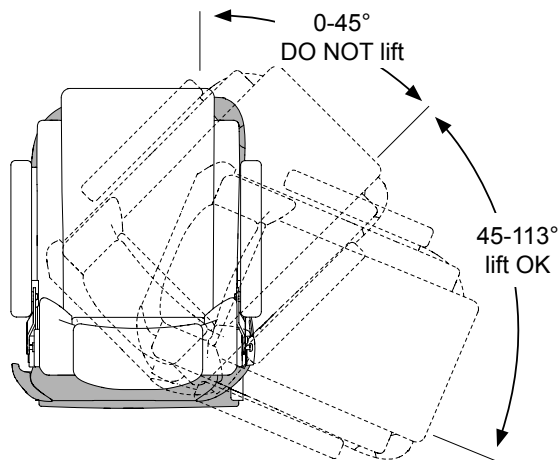1

*Lifting in the chair, max up and in, must be carried out when it is rotated out by 45-113°. If this lift is done when it is rotated out 0-45° serious damage may occur to Turny.*

2

#### Electrical

##### The chair shall:

- be lifted max up and in
- be rotated max in.
- be moved max back.

Zero positions for the chair's three movements.

*Contact your installer if the problem persists.*

3

#### Manual

##### Repeat the lift when the chair:

- can be moved forward and backward when the rotation in is complete.
- is difficult to rotate into the car.
- is not possible to rotate the full way back into the car.

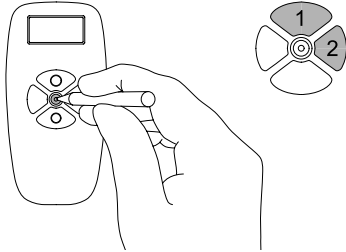
*Contact your installer to book an overhaul. Any faults must first be remedied before the product is brought back into use.*

## 3.1 Electrical

4

Use a pen to press carefully to actuate the emergency-mode operation position. Press code 1, 1, 1, 2.

5

A warning text is displayed on the display.

Step down through the text.

Confirm that you have read it.

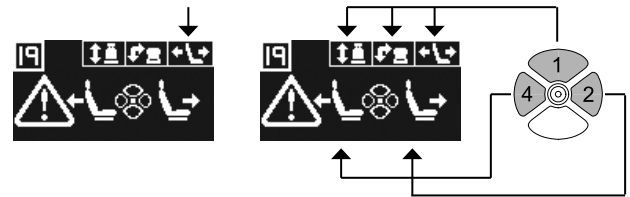
6

*During emergency-mode operation using the manual control, Turny is operated manually. Operate carefully to avoid damage to the product, see illustration on page 32.*

The three movements of the chair are shown at top right, the lengthwise adjustment is highlighted.

Press 1 until the desired movement is highlighted.

Press 2 or 4 for the desired direction of the selected movement. Once the movement has reached its zero position, the symbol will be toned down. When the seat will be driven forward, rotated out or lowered, stop each movement before it comes to a mechanical stop, to avoid damaging Turny.

Alternate between 1, 2 and 4 until all three of the chair's movements have reached their zero positions.


## 3.2 Manual

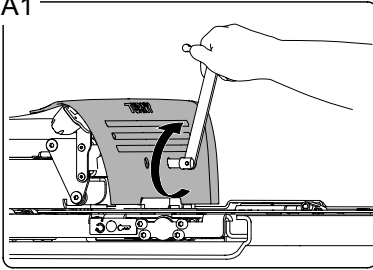7

### A - easy but slow

Use the enclosed emergency crank

Back of Turny  
A1


The emergency crank is placed in the hole to the right of a Turny located on the right and in the hole to the left on a Turny located on the left. Crank clockwise to raise the chair.

Select the option for lifting

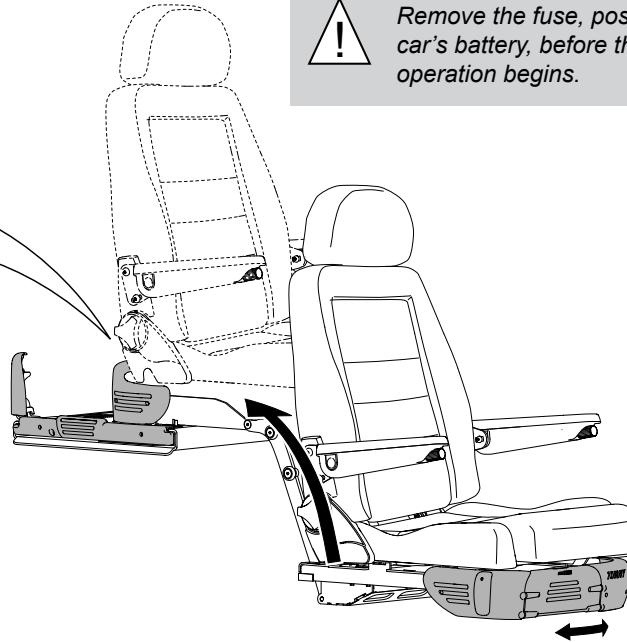*Remove the fuse, positioned close to the car's battery, before the emergency-mode operation begins.*

*Control operation carefully to avoid damage to Turny, see illustration on page 32.*


## B - difficult but quick

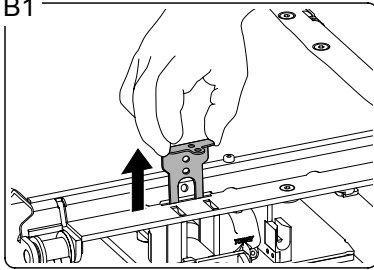Use manual power

Pull across the protective curtain


B1

### Tip!

Release the rotation if the space is limited, see the next page. Then, small rotation adjustments can be made in combination with the lift to facilitate operation into the car.

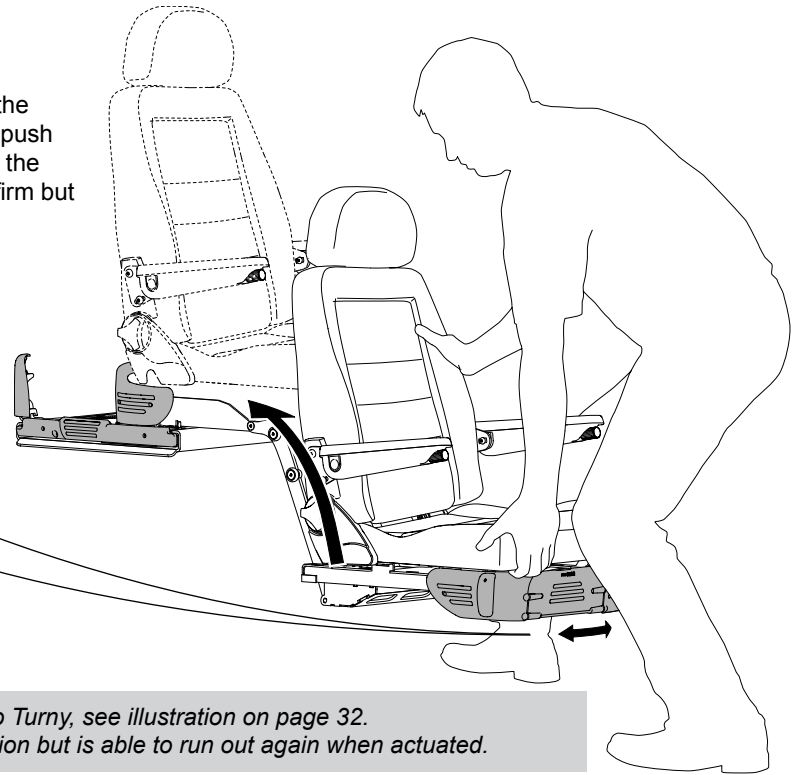B2 Lift and, at the same time, push the chair all the way in. Be firm but careful.



*Control operation carefully to avoid damage to Turny, see illustration on page 32.  
Note that the chair has not locked in this position but is able to run out again when actuated.*

## 8

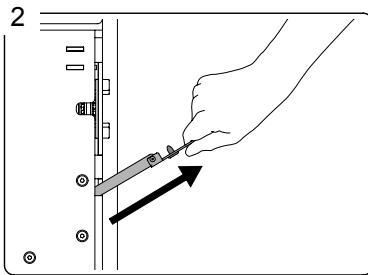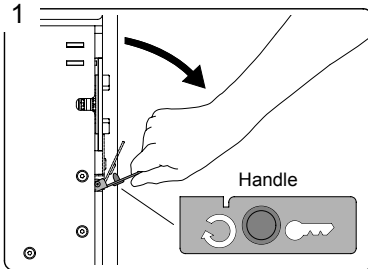**Rotation and lengthwise adjustment**



Use manual power and the enclosed emergency crank

Rotation stop at the front on base board

To be able to rotate into the car, a longitudinal adjustment may be required. Put the emergency crank into the hole furthest down on the left. Crank anticlockwise to move the chair backwards.

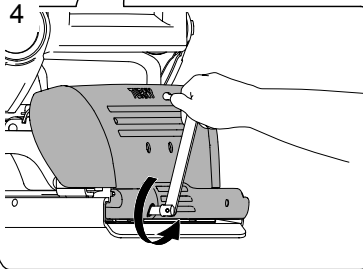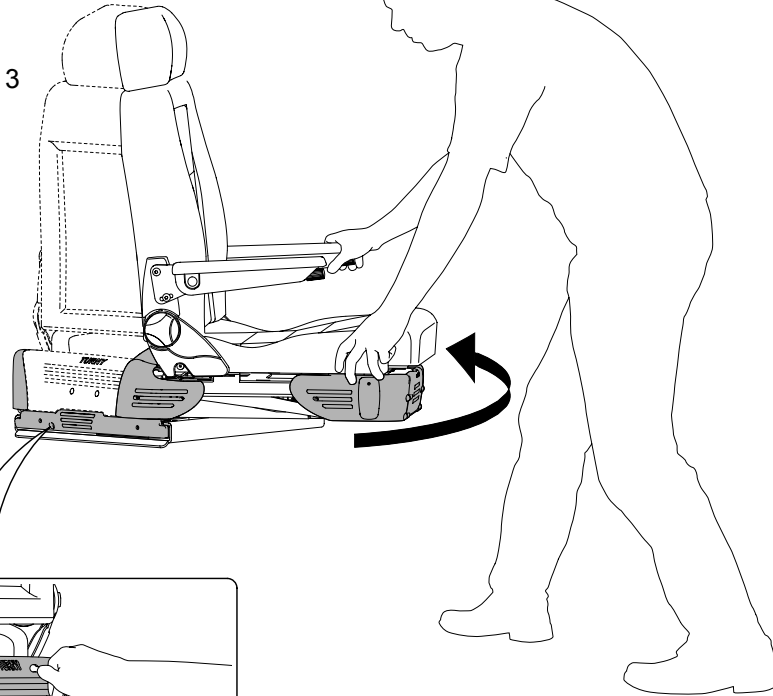3

- 5 To lock the chair in the inner position, when rotation in is complete, push back and fold the handle in at the front on the base board. It may need to be nudged a little.

## 4 Care instructions

Recommended frequency	Care
When necessary .....	* Vacuum clean Turny carefully. Do not use any liquids.
At $\approx$ 4 operations/day of use ..... or less, is service after one year of use.	* Contact your installer, see the servicing information at the end of the manual.
At more than 4 operations/day ..... or less, is service based on average use. Please contact Autoadapt support for service intervals.	* Contact your installer, see the servicing information at the end of the manual.

- The product documentation must always be available during any inspection of the vehicle.
- Always contact your dealer when disassembling.

# 5 Troubleshooter

Symptom	Action
The manual control: - does not light up .....	<ul style="list-style-type: none"> <li>* The manual control may be in “sleep mode”, i.e. resting. Press any button to actuate the display.</li> <li>* Check that the fuse has not blown and that the car battery is fully charged.</li> <li>* Check that the cable to the control is connected and free of any damage.</li> <li>* Chapter 3, items 1+3+7 and 8.</li> </ul>
- indicates a warning .....	<ul style="list-style-type: none"> <li>* Alternative 1: chapter 3, items 1+2+5 and 6.</li> <li>* Alternative 2: chapter 3, items 1+3+7 and 8.</li> </ul>
- indicates normal operating..... position but cannot be operated	<ul style="list-style-type: none"> <li>* Alternative 1: chapter 3, items 1+2+4 och 6.</li> <li>* Alternative 2: chapter 3, items 1+3+7 och 8.</li> </ul>
- indicates an error message .....	<ul style="list-style-type: none"> <li>* Identify error message, see page 26, and carry out recommended actions.</li> </ul>
Discordant sounds are heard .....	<ul style="list-style-type: none"> <li>* Do not use Turny. Contact your installer immediately.</li> </ul>

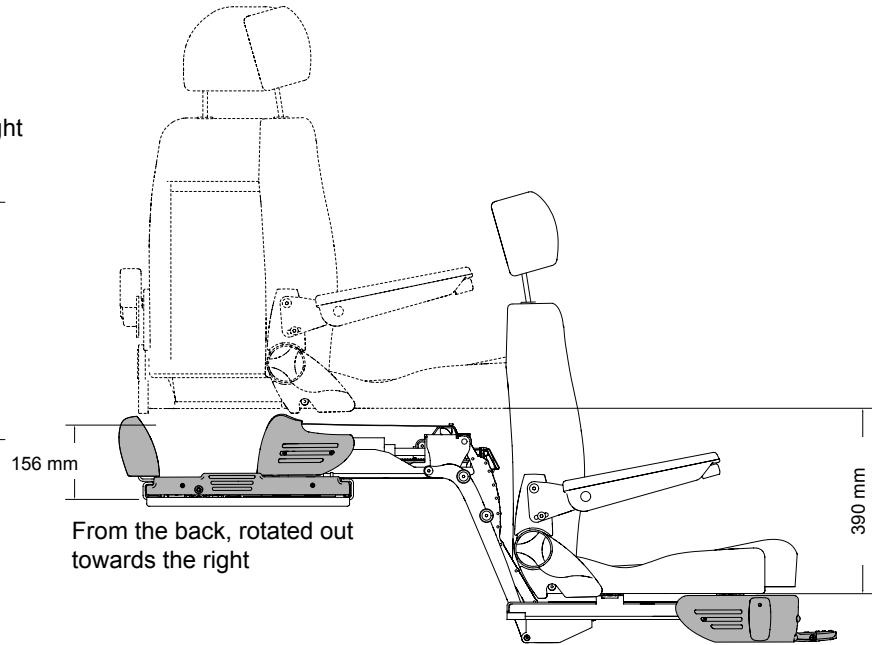
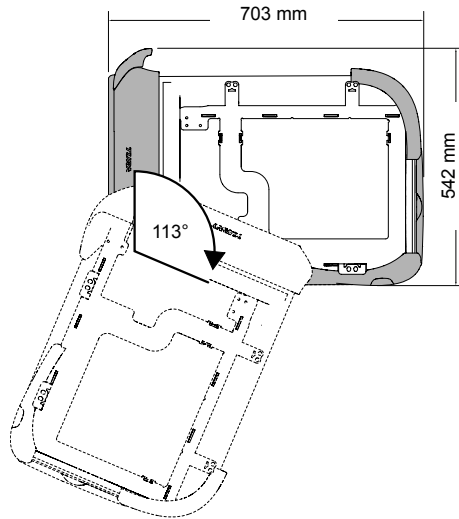
Please contact your installation engineer if you require further assistance or need to ask questions about warranty.  
We recommend keeping the following information to hand at all times:

Model.....	Contact details for the installation engineer.....
Serial number .....	.....
Year of manufacture .....	.....
Date of purchase .....	.....

# 6 Technical specification

## 6.1 Technical data

From above, rotated out towards the right


From the back, rotated out towards the right

Length with footrest retracted .....	703 mm	Weight .....	75 kg
Width .....	542 mm	Max user weight* .....	167 kg
Installation height, up to chair attachment .....	156 mm	Max power consumption.....	12 A
Max rotation angle .....	113°	Power consumption in standby mode.....	≈ 55 mA
Max electric lengthwise adjustment.....	200 mm	Fuse.....	15 A
Lifting height .....	390 mm		

\* 167 kg applies if the chair weights max 13 kg. In the case of heavier chairs and/or accessories, seek the advice of your installer.

## 6.2 Components

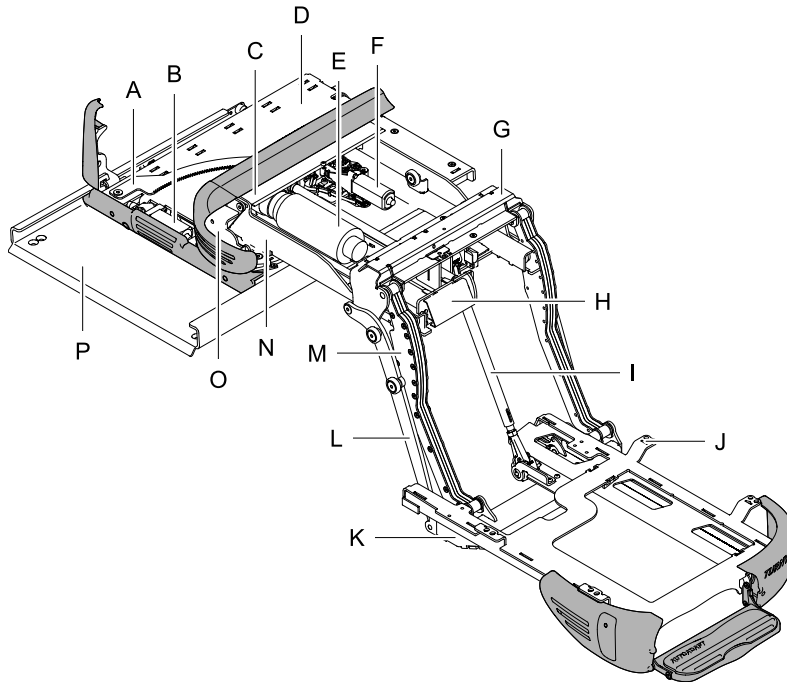---

A .....	Gear ring for rotation motor	I.....	Drive shaft
B .....	Lengthwise adjustment motor	J.....	Chair plate
C .....	Protective curtain	K.....	Outer slide rails
D .....	Base plate	L.....	Push-rod
E .....	Lifting motor	M.....	Supporting arms
F .....	Rotation motor	N.....	CE label
G .....	Inner slide rails	O.....	Collision safety catch behind cover
H.....	Over recline switch	P.....	Adapter plate

# 7 Introduction for users

Autoadapt would like you, as a user, to feel secure and comfortable with your new product. For that reason, your installer has the task of going through the following chapter with you:

- 1 Safety**
- 2 Operation**
- 3 Emergency-mode operation**

If any questions about the product should come up later on, you are of course very welcome to contact your installer again.

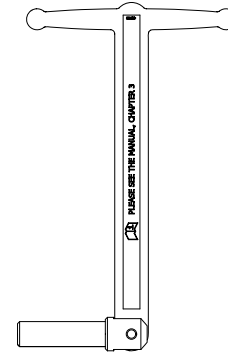
The above points have been gone through.

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.....

# Service

Recommended frequency	Service performed by authorized dealer
<p>At ≈ 4 operations/day of use or less.. is service after one year of use.</p> <p>At more than 4 operations/day of ..... use is service based on average use. Please contact Autoadapt support for service intervals.</p>	<p>* For service schedule, please visit; dealerweb.autoadapt.com</p> <p>* For service schedule, please visit; dealerweb.autoadapt.com</p>

▫ In order to improve our product, please send an e-mail to [support@autoadapt.se](mailto:support@autoadapt.se) with information about additional work done that is not specified in the service schedule.

Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)	Date	Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)	Date
	Additional work done ▫ ----- ----- ----- ----- ----- ----- -----		Additional work done ▫ ----- ----- ----- ----- ----- ----- -----
<b>1</b> (stamp/signature of the dealer)		<b>2</b> (stamp/signature of the dealer)	

<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>3</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>	<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>4</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>
<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>5</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>	<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>6</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>
<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>7</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>	<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>8</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>

<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>9</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>	<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>10</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>
<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>11</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>	<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>12</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>
<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>13</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>	<p>Service performed by (in accordance with service schedule at dealerweb.autoadapt.com)</p> <p><b>14</b> (stamp/signature of the dealer)</p>	<p>Date</p> <hr/> <p>Additional work done ☐</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p> <p>-----</p>

A global provider of vehicle adaptation solutions for persons with reduced or limited mobility

### **Autoadapt AB**

Åkerivägen 7  
S-443 61 Stenkullen  
Sweden  
Phone: +46 (0)302 254 00  
E-mail: [contact@autoadapt.se](mailto:contact@autoadapt.se)

### **Autoadapt UK Ltd**

Unit 1, Windsor Industrial Estate  
Rupert Street  
Aston, Birmingham B7 4PR, UK  
Phone: +44 (0)121 33 35 170  
E-mail: [contact@autoadapt.co.uk](mailto:contact@autoadapt.co.uk)

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© 2013 Autoadapt AB