


# Carony Go

## Charge the battery

EN

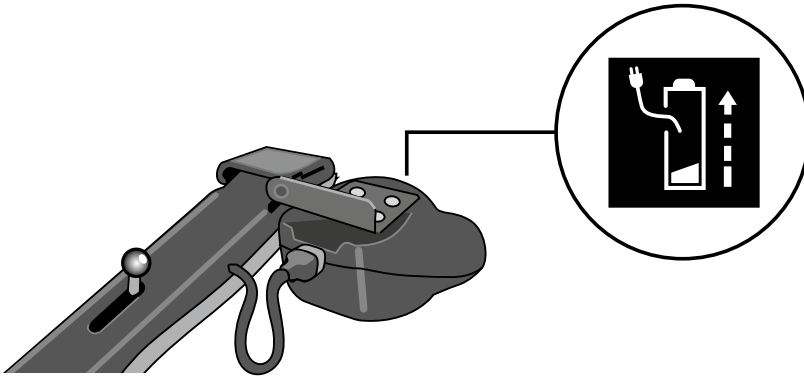

Before using Carony Go for the first time, the batteries must be charged.  
When Carony Go is not in use always connect to battery charger.

Red flashing.....	Battery failure
Yellow shining.....	The battery is loading
Green shining.....	The battery is full loaded
Yellow flashing.....	The charger does not detect the battery
Yellow/green flashing.....	The charging voltage is below 1,4 V/cell
Yellow/green alternately.....	Intern temperature >60°C
Green flashing.....	High charging voltage or loading failure

