**CROSSIT**

**Safety** *Säkerhet, Sikkerhet, Sicherheit, Sécurité, Seguridad, Sicurezza, Segurança, Безопасность* 6

**Mounting** *Montering, Monterung, Montage, Montaje, Montaje, Assemblaggio, Montagem, Монтаж* 8

**Usage** *Användning, Bruk, Benutzung, Utilisation, Uso, Utilizzo, Utilização, Использование* 12

**Options** *Tillval, Opsjoner, Optionen, Options, Opciones, Opzioni, Opções, Варианты* 16

## **A universal belt solution from Careva Systems AB**

We are pleased that you have chosen a Crossit belt from Careva Systems AB. Crossit is a positioning belt designed with the help of many involved parents of children with disabilities, drivers in the community transport as well as occupational therapists and professional designers. The product is universal and fits both adults and children in different transportation situations.

The Crossit belt can be fitted in all types of seats with an opening between the seat base and back. The product can be used in combination with a booster seat or a forward-facing child car seat.

If you have questions or feedback about your Crossit belt, do not hesitate to contact us or your nearest dealer.

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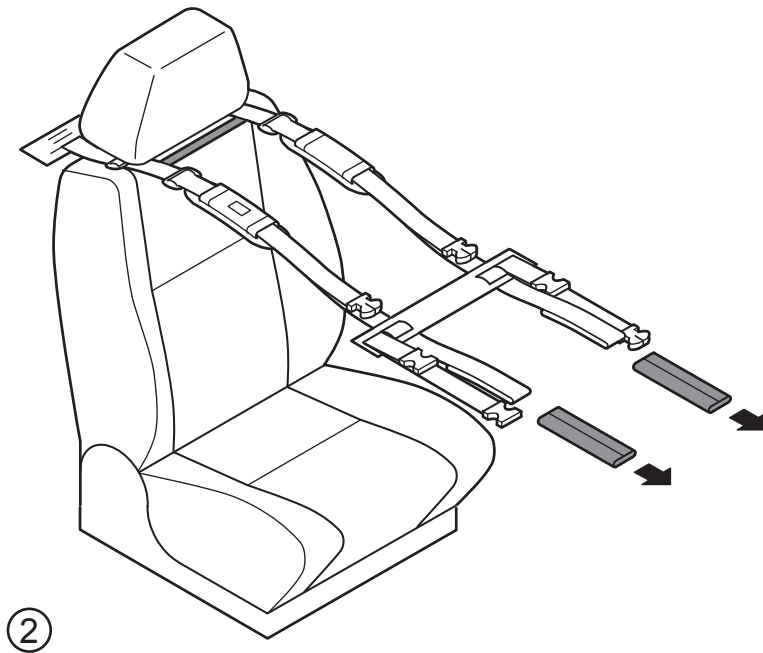
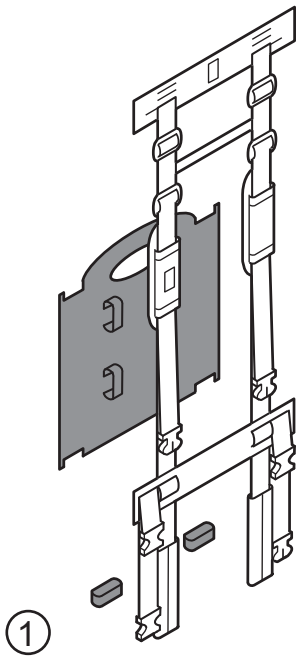
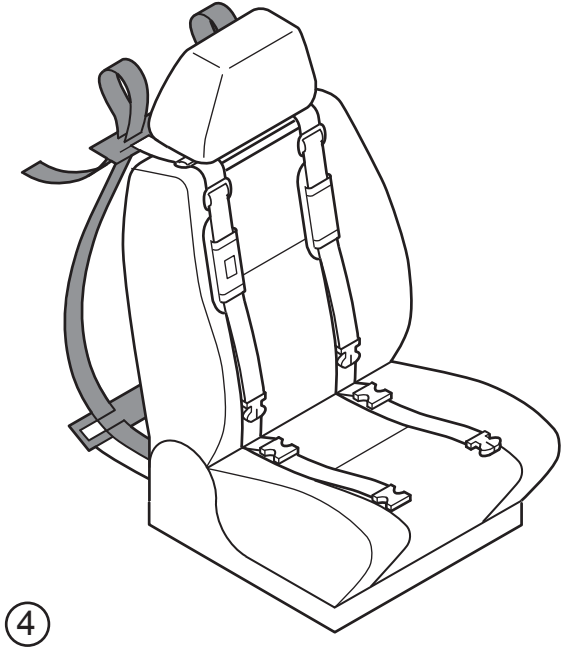
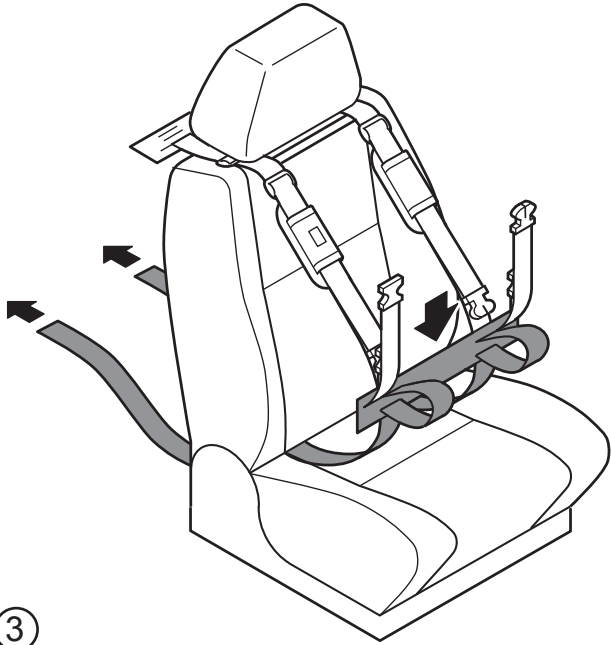
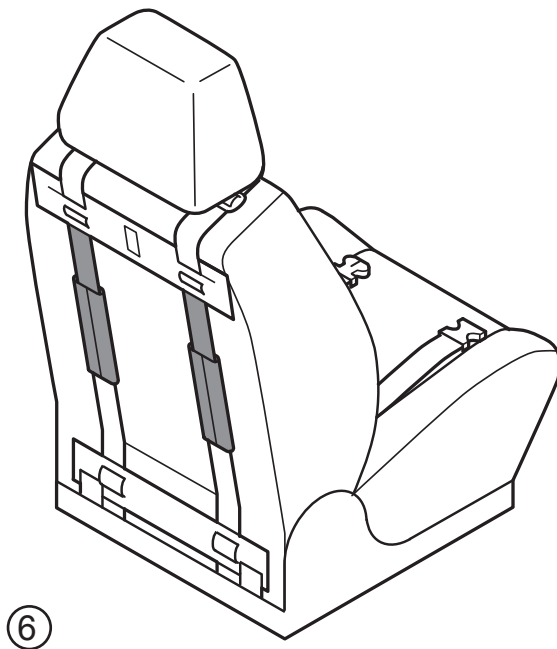
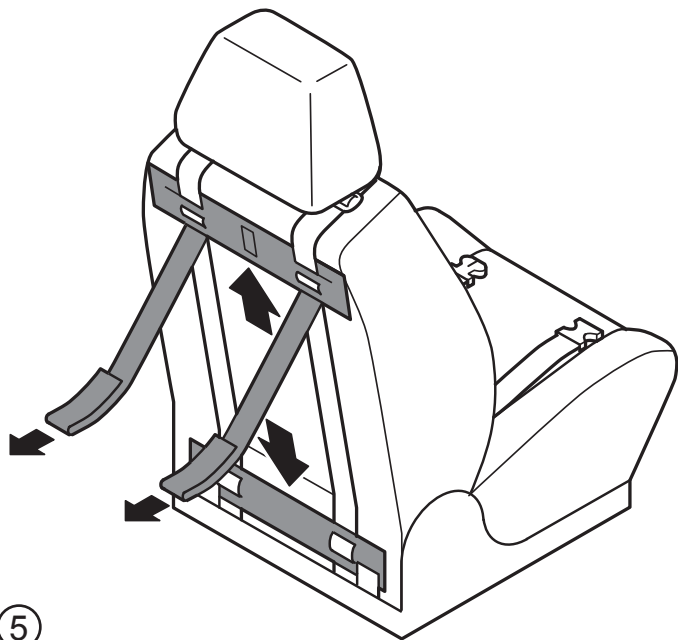
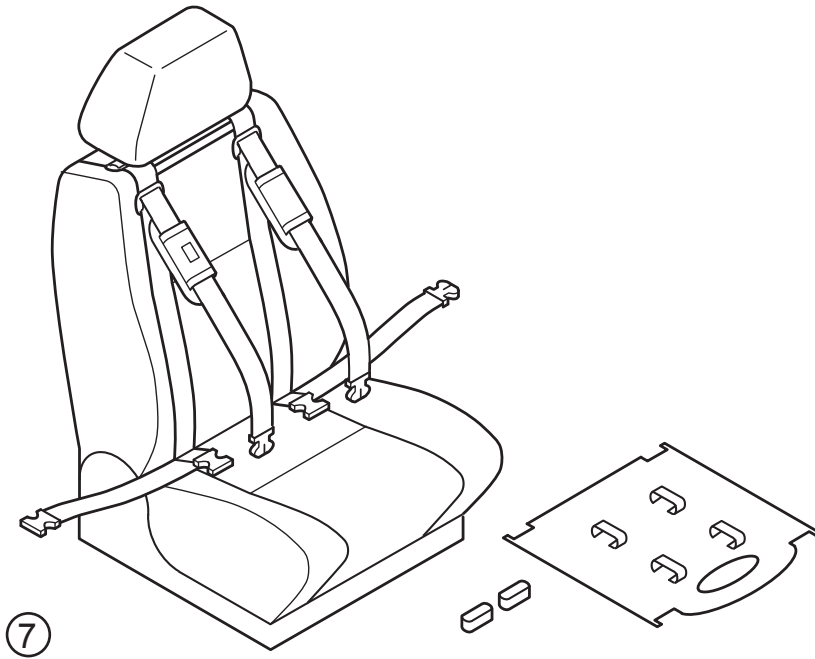
**① Make sure to cross the belt over the chest.  
If the cross is off-centre, use alternative B.**

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**② Adjust to the height of the shoulders.**

Justera i höjd med axlarna.  
Juster i høyde med skuldrene.  
In Schulterhöhe einstellen.  
Réglez à la hauteur des épaules.  
Ajustarlo a la altura de los hombros.  
Regolare all'altezza delle spalle.  
Ajuste de acordo com a altura dos ombros.  
Отрегулируйте по высоте плеч.

**③ Place the waist belt across the hips.**

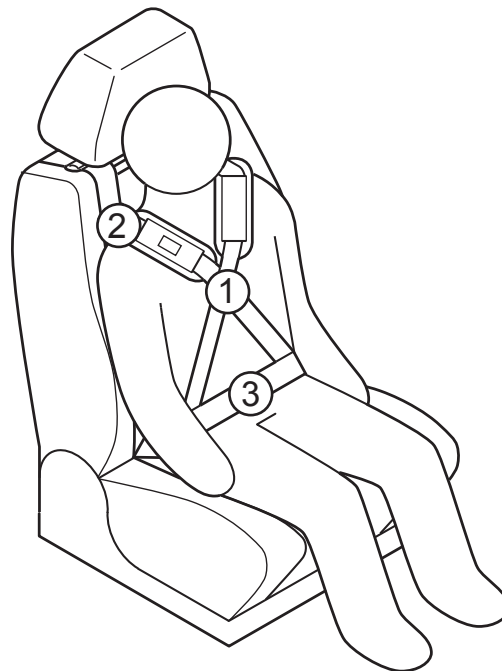
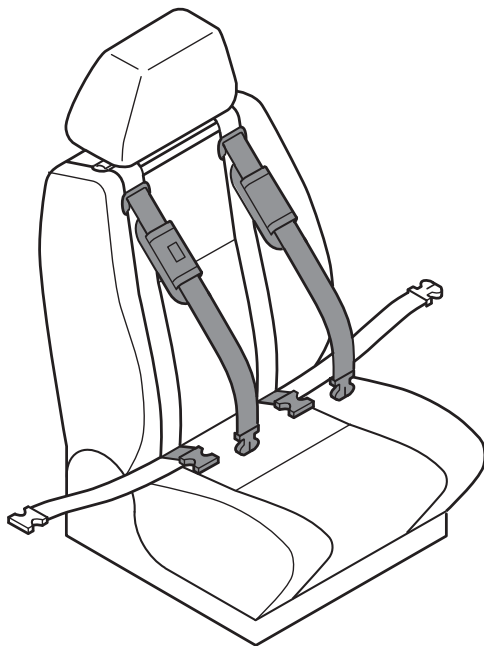
Placera midjebältet över höften.  
Plasser midjebeltet over hoftene.  
Den Beckengurt legen Sie über den Hüftbereich.  
Placez la ceinture de taille sur la hanche.  
Coloque el cinturón de la cintura sobre las caderas.  
Posizionare la cintura ventrale sul bacino.  
Posicione o cinto abdominal sobre os quadris.  
Проденьте поясной ремень по бедрам.

**④ Trim long straps as needed.**

Klipp av långa band vid behov.  
Klipp av lange bånd ved behov.  
Ggf. lange Bänder abschneiden.  
Si nécessaire, coupez les sangles longues.  
Si es necesario, corte el exceso de cinta.  
Accorciare le cinghie se necessario.  
Apare o comprimento das correias conforme a necessidade.  
При необходимости укоротите (отрежьте) длинный пояс.

*Alternativ A, Alternativ A, Alternative A, L'alternative A,  
La alternativa A, L'alternativa A, Alternativa A, Вариант A*

**① Make sure that the chest buckle is placed on the chest.**

Se till att bröstlåset hamnar över bröstbenet.  
Pass på at brystlåsen kommer over brystbenet.  
Das Gurtschloss muss auf dem Brustbein liegen.  
Veillez à ce que la boucle de fermeture de poitrine se place sur la poitrine.  
Asegúrese de que el cierre del pecho quede sobre el esternón.  
Assicurarsi che la chiusura pettorale sia posizionata sullo sterno.  
Certifique-se de que a fivela de peito esteja posicionada sobre o peito.  
Убедитесь, что грудной замок располагается на груди.

**② Adjust to the height of the shoulders.**

Justera i höjd med axlarna.  
Juster i høyde med skuldrene.  
In Schulterhöhe einstellen.  
Réglez à la hauteur des épaules.  
Ajustarlo a la altura de los hombros.  
Regolare all'altezza delle spalle.  
Ajuste de acordo com a altura dos ombros.  
Отрегулируйте по высоте плеч.

**③ Place the waist belt across the hips.**

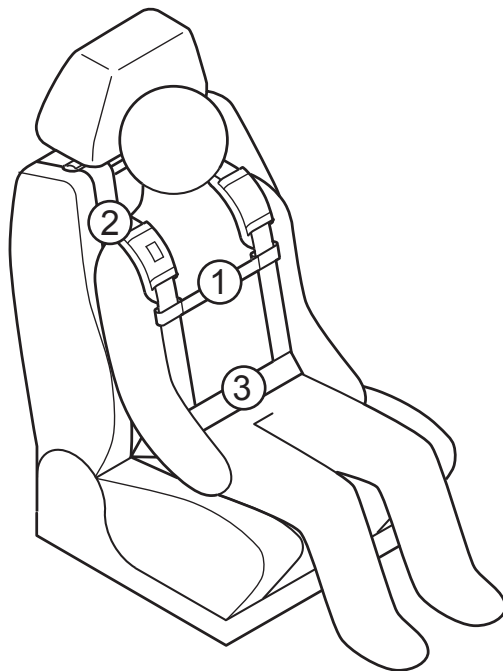
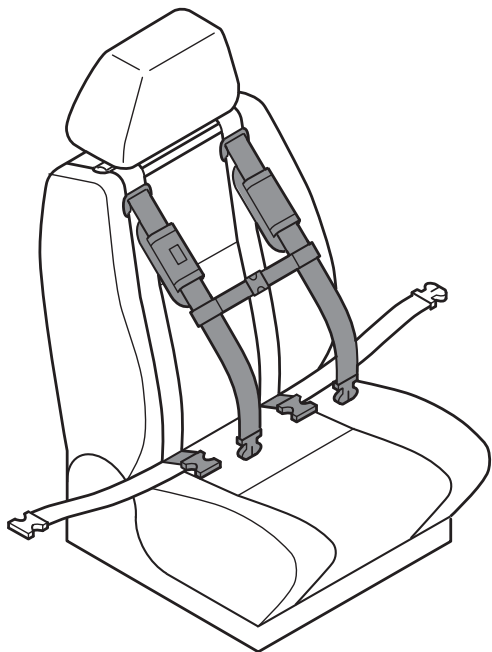
Placera midjebältet över höften.  
Plasser midjebeltet over hoftene.  
Den Beckengurt legen Sie über den Hüftbereich.  
Placez la ceinture de taille sur la hanche.  
Coloque el cinturón de la cintura sobre las caderas.  
Posizionare la cintura ventrale sul bacino.  
Posicione o cinto abdominal sobre os quadris.  
Проденьте поясной ремень по бедрам.

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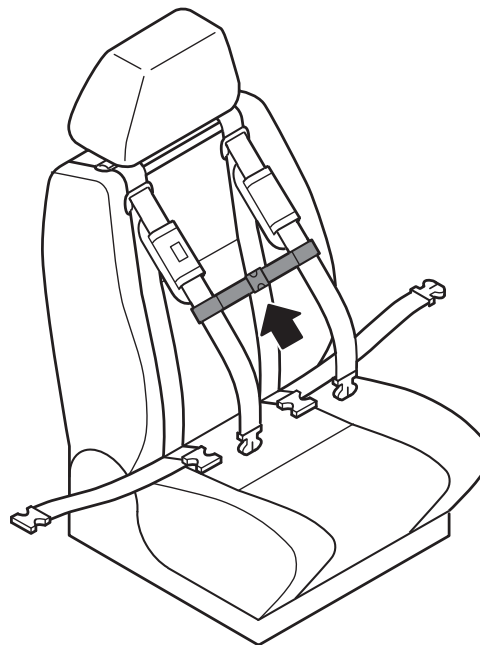
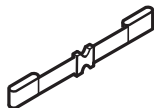
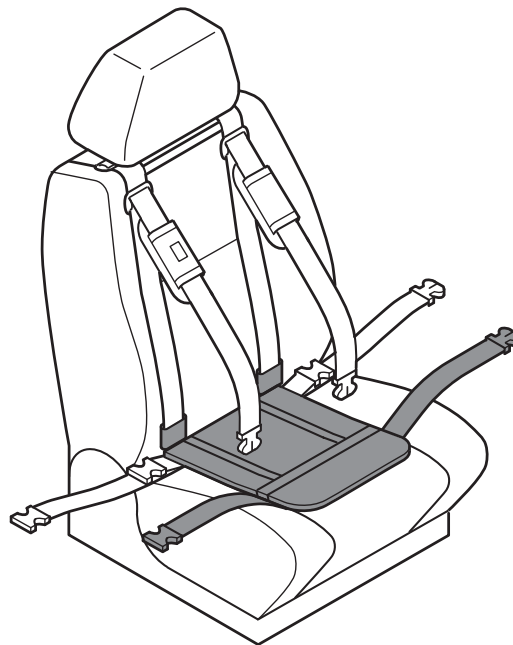
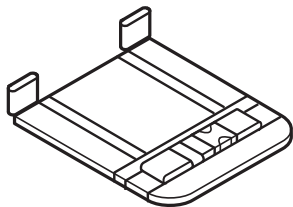
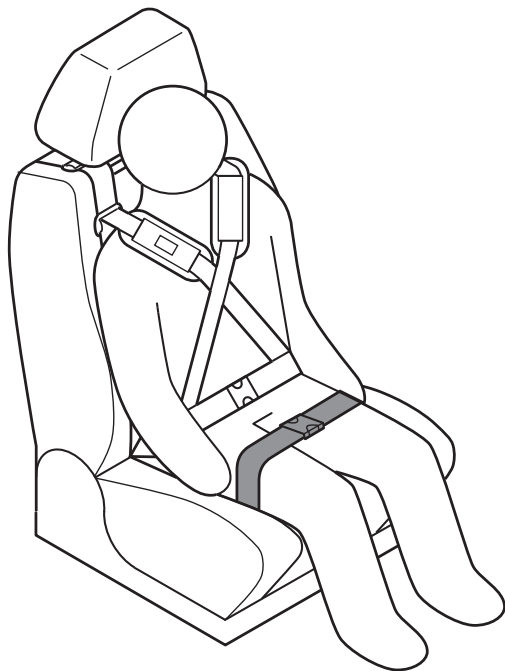
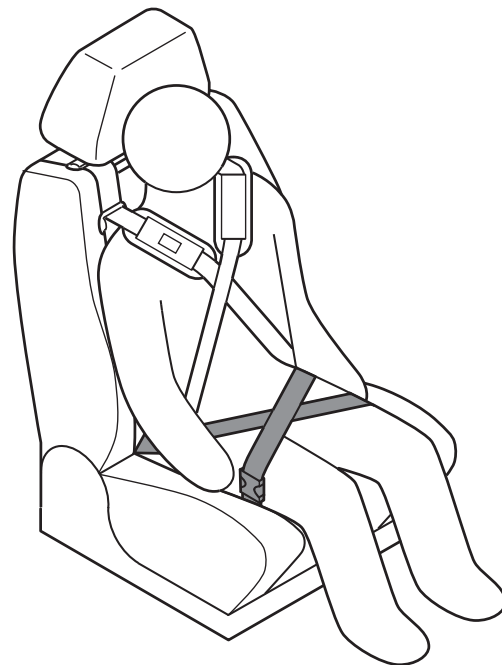
Illustrations, descriptions and specifications in this manual are based on current product information.  
Careva Systems AB reserves the right for alterations without previous notice.

Illustrations by Emma Selfjord

© 2011 Careva Systems AB

# CAREVA SYSTEMS AB

[www.careva.se](http://www.careva.se)

[www.autoadapt.com](http://www.autoadapt.com)

[www.carehainland.fi](http://www.carehainland.fi)

handicare

[www.handicare.no](http://www.handicare.no)

[www.onhealthcare.ie](http://www.onhealthcare.ie)

[www.t-rv.de](http://www.t-rv.de)