# Tilda

Seat tilter

SV	4	Bruksanvisning
EN	10	User manual
DE	16	Gebrauchsanweisung
FR	22	Instructions d'utilisation
ES	28	Instrucciones de uso
IT	34	Istruzioni per l'uso

SV

EN

DE

FR

ES

IT

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Thank you for choosing a

# Tilda from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit [www.braunability.eu](http://www.braunability.eu).

Once again, thank you for placing your confidence in our products!

---

**Safety**

---

I2 Safety measures

**Operation**

---

I3

**EN**

Index

“Product certificate” at the end of the manual.

## Safety

### Safety measures

*Make sure that no fingers are under the seat when rising it from tilted position.*

*Make sure that there are no foreign objects or gravel in the mechanics or between the plates.*

*During drive the seat may not be tilted to more than pos. I (lowest tilt position).*

*When combined with Carony the Anti-tip protection must be used.*

*Watch your head when passing through the doorway.*

## Operation

Tilda provides three tilt positions. The maximum tilt position is 15° but can be limited to 11° or 7°. Clean with a vacuum cleaner. Can also be wiped gently with a moist towel and some mild cleaning detergent, when needed. Tilda is designed to function maintenance-free. Check irregularities such as loosening screws or unusual noise during use.

Maximum user weight.....	150 kg
Maximum load on footrests.....	70 kg
Weight incl.footrest.....	8,3 kg
Width.....	520 mm

- The product documentation must always be available during inspection of the vehicle.
- Always contact your dealer for disassembly.

Please contact your installation engineer if you require further assistance or need to ask questions about warranty. We recommend keeping the following information to hand at all times /

Contact details for the installation engineer /

Model /

Serial number /

Year of manufacture /

Date of purchase /

Notes /

---

---

---

---


---

---

---

---

FIG 1

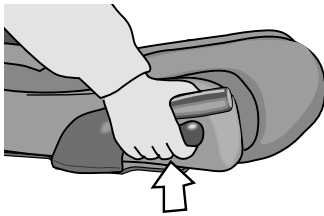### Tilting the seat

Release the tilt function by pressing both handles towards each other and pull them backwards (fig. 1).

Release the lower handle (lock handle) to allow the lock to fix inside one of the three tilt position catches.


POS 0


POS 1

POS 2

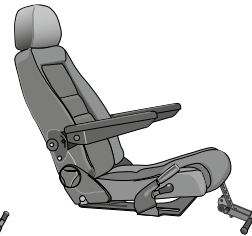POS 3
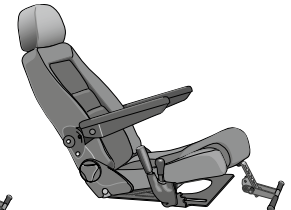
### Rising the seat

Press the two handles towards each other and push them forwards until the seat is risen to an upright position (fig. 1).

Let go of the lower handle (lock handle) to allow the seat to be fixed in the desired upright position. If it feels heavy just put your hand on top of the backrest and push.

### In snug conditions

In some vehicles with a high and wide centre console it might be necessary to slide the seat forward or to limit the tilt angle before swivelling outwards (in Turn-out applications).

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## Declares that the product /

Tilda, model left/right

## Conforms to following directives, standards and regulations /

2006/42/EEC  
2007/46 EEC  
2007/47/EEC,  
LVFS 2013:11  
ECE R-17 rev 5 (incl supplement 08)  
ECE R- 21  
FMVSS 302

BraunAbility Europe AB, Stenkullen  
2019-04-02

Jan M Jensen  
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005  
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.  
BraunAbility Europe AB reserves the right to make alterations without previous notice.  
© 2019 BraunAbility Europe AB

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)