Turny Low Vehicle

Swivel seat

SV	4	Bruksanvisning
EN	24	User manual
DE	44	Gebrauchsanweisung
FR	64	Instructions d'utilisation
ES	84	Instrucciones de uso
IT	104	Istruzioni per l'uso

SV

EN

DE

FR

ES

IT

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A large, empty rectangular frame with a thin black border, centered on a white background.

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Signatur, installatör

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
Thank you for choosing a

Turny Low Vehicle from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit www.braunability.eu.

Once again, thank you for placing your confidence in our products!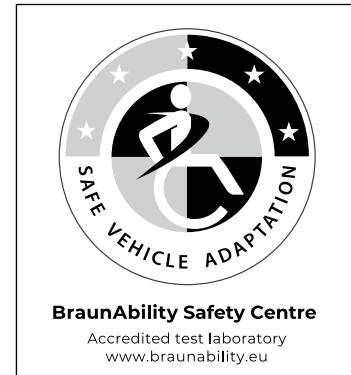

Operation

- 26 Presentation
- 27 Hand held control
- 28 Operation Turny Low Vehicle

Emergency-mode operation

- 30 Using the hand held control
- 33 Using the emergency operation kit

Care instructions

36

Troubleshooter

37

Introduction for users

42

“Product certificate” at the end of the manual.

Operation Presentation

Turny Low Vehicle has two basic functions: swivel the seat in/out and forwards/backwards movement for comfort. Note that it is not possible to swivel the seat when the door is closed, nor is it possible to adjust comfort forwards/backwards once swivelling out has begun.

The path that Turny Low Vehicle follows is preprogrammed by your installation technician. In the case of faults or deviations from the preprogrammed path, the seat will stop and cannot be manoeuvred normally. In this event, Turny Low Vehicle loses its position and must be reset through an emergency operation. Fault message no. 23, Position lost, is shown on the hand held control. Should this occur, follow the "***Emergency Operation by Hand Control***" instructions. Once this is completed, Turny Low Vehicle will function as normal again. Contact your installation technician if emergency operation must be performed often.

In the case of certain electrical faults, emergency operation with the hand held control will not work. One example is when the hand held control does not react to key presses and the display is extinguished despite the door being open. Another example is when fault message no. 2 Cable break in spiral connector to hand held control is displayed. Should this occur, follow the "***Emergency Operation with Emergency Operation Kit***" instructions. Always contact your installation technician after a manual emergency operation to remedy the fault before using Turny Low Vehicle again.

Hand held control


Optional function buttons light up green.

Hold the selected button down. When the seat reaches its preprogrammed end position, it will stop automatically. To stop movement, release the button.

Turny Low Vehicle's location in the vehicle on the right or left side determines the functions of buttons 2 and 4.

During emergency operation or programming, all the lights will flash RED to indicate that Turny Low Vehicle is not following its preprogrammed path and that certain safety functions are disengaged.

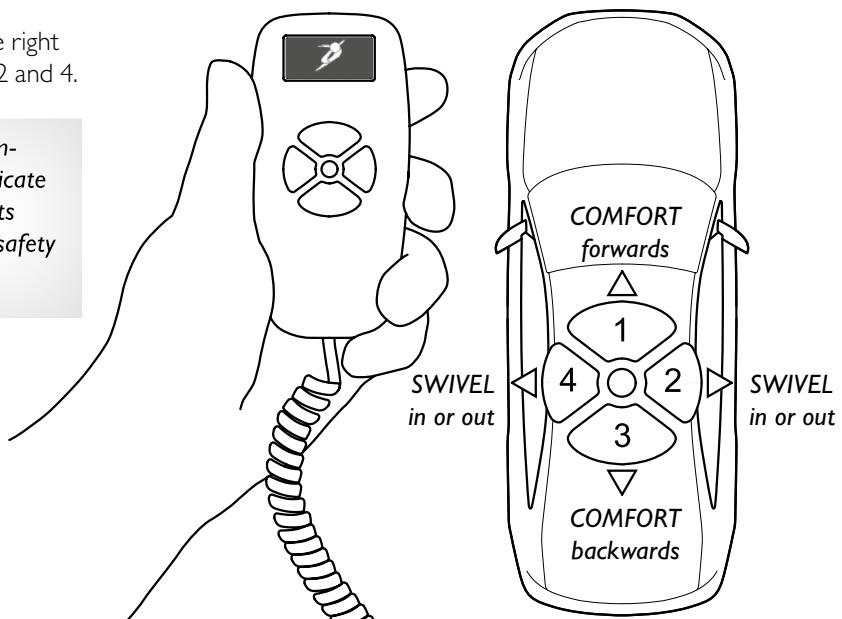Operating Turny Low Vehicle

A door icon is displayed on the right of the screen when the door is open.

Make sure the vehicle is standing level and that the park brake is on.

Make sure that nothing is at risk of catching or being pinched. Note that there is a risk of pinching despite the built-in safety functions.

Recline the backrest to minimize the risk of bumping your head in the door opening. If possible, use the footrest to facilitate the operation.


If Turny Low Vehicle is not operated full cycle, it will sooner or later lose its position and evoke error code 23. It is of no great significance if this happens now and then but it should be avoided completely if possible.


Swivel in and out

Turny Low Vehicle will follow its preprogrammed path.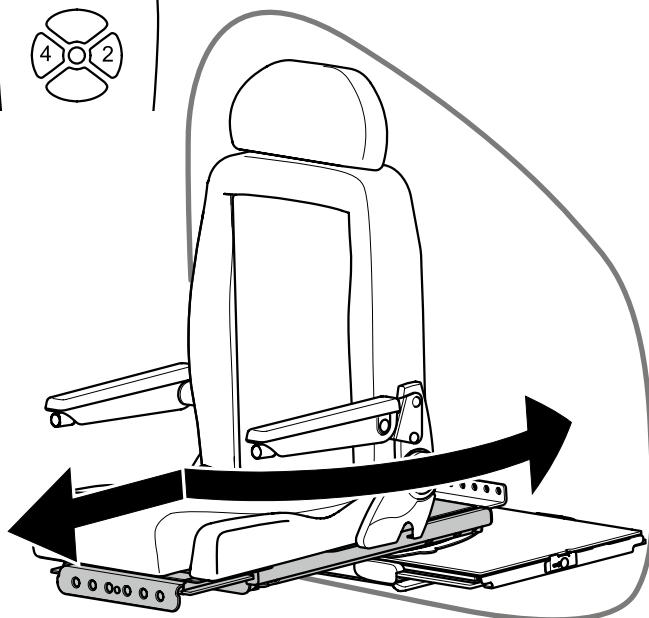

Adjusting comfort

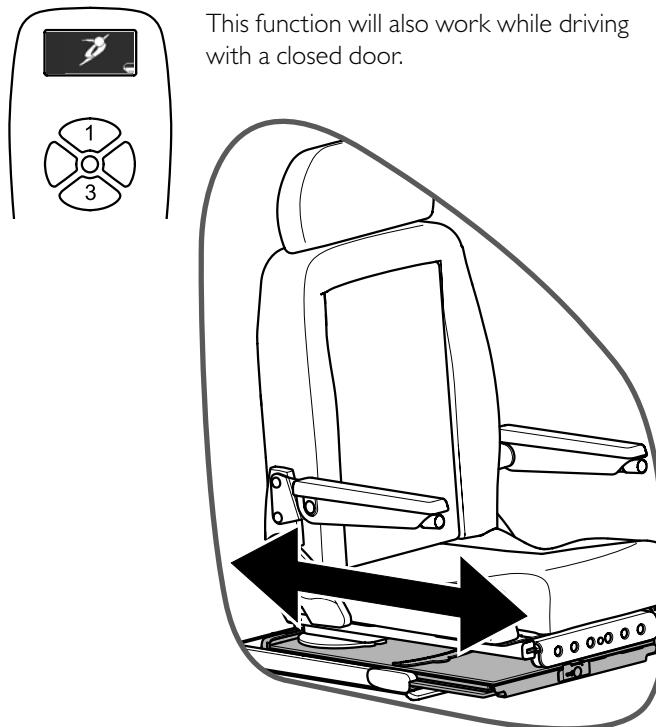
This function will also work while driving with a closed door.

Emergency-mode operation

Using the hand held control

- 1 Activate the emergency operating menu

Hold down the yellow button until the screen **Enter Code** is shown. Then enter the code 1, 1, 2

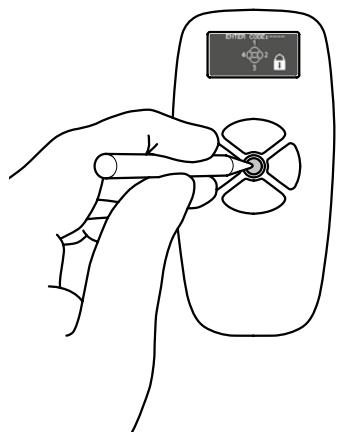
- 2 Swivel the seat in


During emergency operation, all the lights will flash RED to indicate that Turny Low Vehicle is not following its preprogrammed path and that certain safety functions are disengaged.

Select the desired movement. Both directions of the desired movement must first be lit before the in movement takes place. The seat may require moving forwards slightly to achieve this for lengthwise and extension. Then switch between movements and directions to manoeuvre in the seat.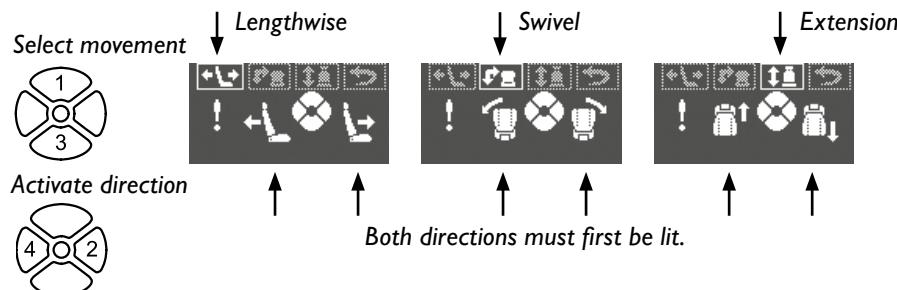



Lengthwise and extension - operate the seat backwards until the symbols marked below become grey. The seat will stop automatically and a click will be heard. If necessary, the system can be overridden in this position to force the seat further back. Hold in button 4 for four seconds. A countdown will start on the display and the seat will continue backwards when it is finished. The seat will stop automatically when it has reached its end position, a click will be heard and the symbol is fully extinguished.

Swivel - swivel in the seat until the marked symbol below is fully extinguished. The seat will stop automatically when it has reached its end position and a click is heard.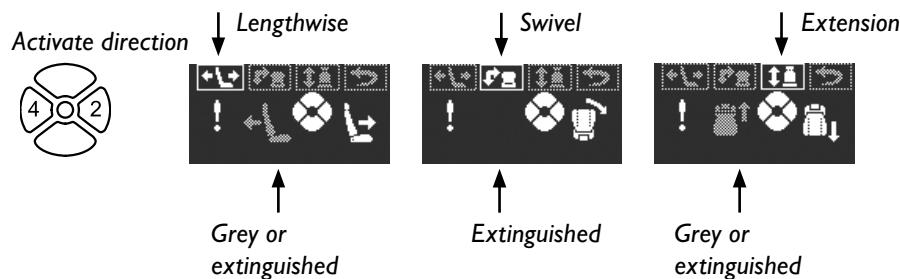

3 Exit the emergency operation menu

When the above symbols for lengthwise and extension have become grey or extinguished and the swivel symbol extinguished, the “**exit symbol**” will come on and the emergency operation menu can be exited.

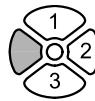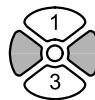- ↓ *Exit*
- 1  
Press 1 or 3 and select **Exit**.
Press 2 to confirm **Exit**. Turny Low Vehicle is now back in its normal operating mode.
- 2  
If this screen appears the criteria to exit the system have not been achieved.
Go back and do 2 again.
Press 1 or 3 to select the main menu at the top.



Using the emergency operation kit

Move the seat backwards, so that it is not extended, and swivel it back into the car.

Begin by releasing the swivelling mechanism if these manoeuvres must be combined and cannot be done separately. Adjust lengthwise as necessary. For swivelling and lengthwise movement, see overleaf.



EXTENSION

Insert the socket in the designated hole. Crank the ratchet clockwise to move the seat backwards.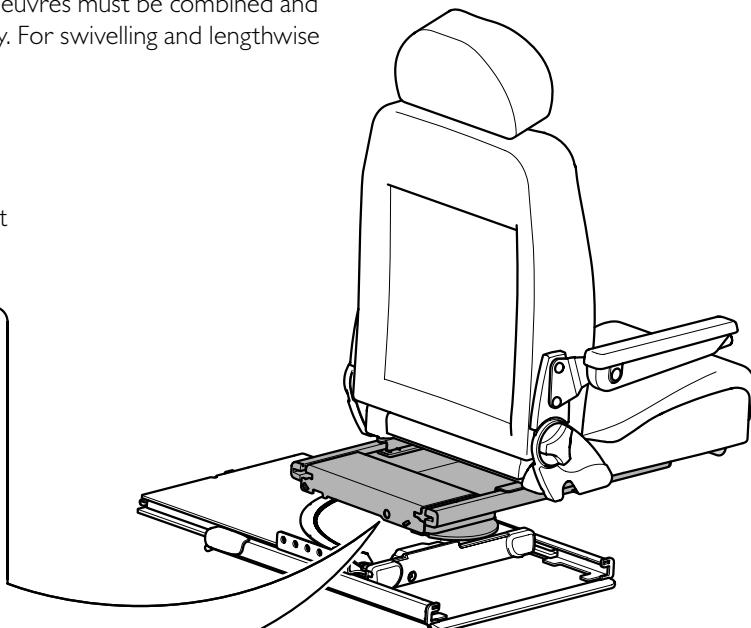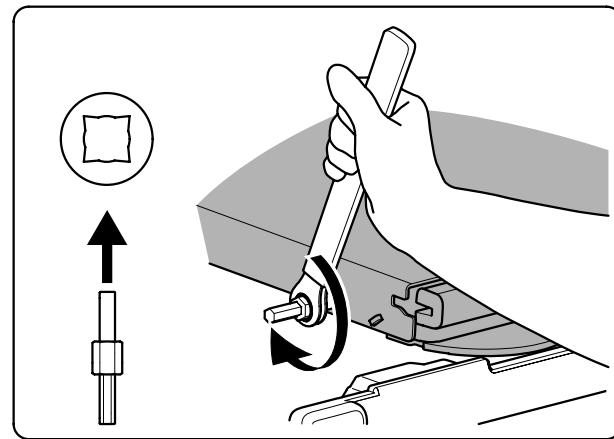



SWIVEL

Pull out the yellow lock plate at the front of the base plate to release the turning mechanism. Turn the seat all the way in and then slide the lock plate back to lock the seat in position.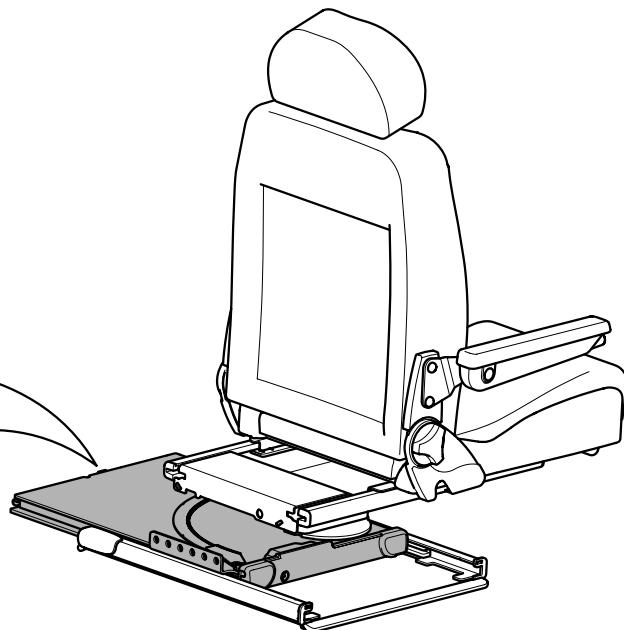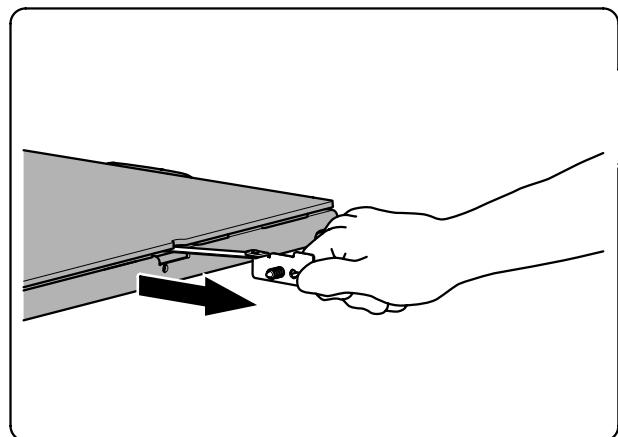

LENGTHWISE ADJUSTMENT

Insert the socket in the hole in the lower rear cover. Crank the ratchet anticlockwise to move the seat backwards.

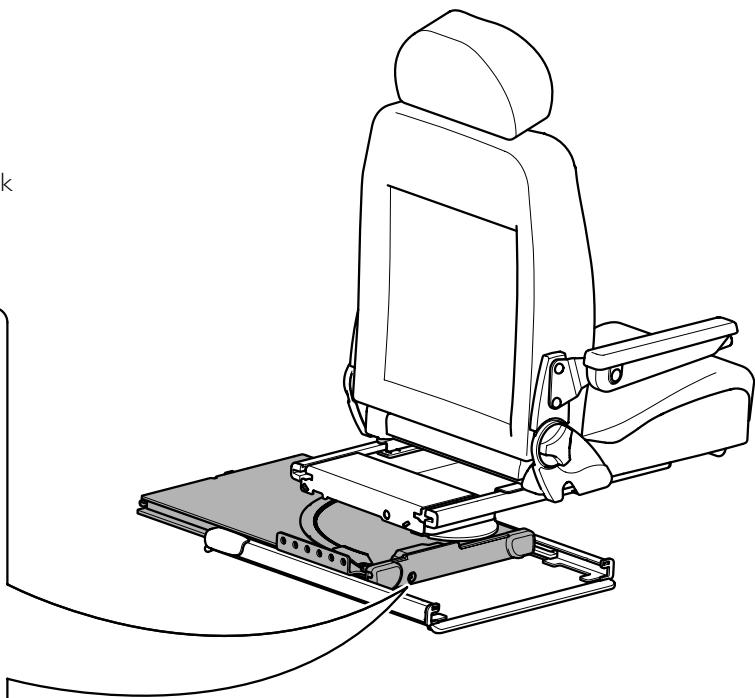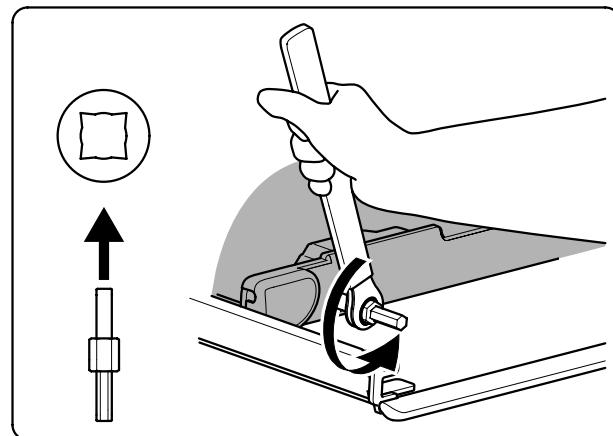Care instructions

As necessary.....

Carefully vacuum clean Turny Low Vehicle. Use no liquids.

At ≈ 4 operations/day of use or less, is service after one year of use.

Contact your installer for servicing information.

At more than 4 operations/day or less, is service based on average use. Please contact BraunAbility support for service intervals.

Contact your installer for servicing information.

- Always contact your dealer for disassembly.

Any maintenance, modifications or repairs to the Turny Low Vehicle may only be performed by a qualified installation technician.

Product data

Weight of product, without seat and accessories 65 kg
Fuse 15A

Troubleshooter

Hand held control /

- does not light up.....

- The hand held control may be in "***sleep mode***", i.e. resting. Open the door.
- Check that the wiring to the hand held control is connected and undamaged.
- Check that the fuse is intact and that the car battery is charged.
- Follow the "***Emergency Operation***" using the Emergency Kit instructions.

- shows warning/fault message.....

- Identify the message on the following pages and perform the recommended measures.
- Follow the "***Emergency Operation by Hand Control***" instructions.
- Follow the "***Emergency Operation***" using the Emergency Kit instructions.

Unusual noises heard during operation.....

- Do not used Turny Low Vehicle. Contact your installation technician for an overhaul.

EN

Troubleshooting

Warnings/fault messages

Warning, Incomplete seat swivel-in. Turny Low Vehicle is not in a crash-safe position. The symbol is laterally inverse on a left-side Turny Low Vehicle.

Low battery charge.

1. Cable break in the spiral connector to the hand held control.
2. Communication problem between the control card and the hand-held control.

1. Swivel motor blocked.
2. Swivel motor pulse generator error.

1. Extender motor blocked
2. Extender motor pulse generator error

1. Lengthwise adjustment motor blocked.
2. Lengthwise adjustment motor pulse generator error.

Recommended action

Swivel the seat all the way in.

Charge the vehicle battery.

Replace the cable.

1. Check that there are no obstructions.
2. Check the connector to ensure that no cables are loose.
3. Follow the emergency operation instructions.

Warnings/fault messages

Swivel motor pinch protection activated.

Extender motor pinch protection activated.

Lengthwise adjustment motor pinch protection activated.

Cable break or limit switch for the swivel motor out of service.

Cable break or limit switch for extender motor out of operation.

Cable break or limit switch for length adjustment motor out of operation.

Recommended action

If obstructed, Turny Low Vehicle will stop and back a little. Remove the obstruction and try again.

Follow the emergency operation instructions.

Warnings/fault messages

Swivel motor overloaded.

Extender motor overload

Lengthwise adjustment motor overloaded

Swivel-in motor switch activated in wrong position.

Extender adjustment motor switch activated in wrong position.

Lengthwise adjustment motor switch activated in wrong position.

Turny Low Vehicle has lost its position.

Tip! Ensure programming has been completed correctly, refer to the “**Program path of movement**” heading in Chapter 11.2. (*Installation manual*).

Recommended action

Make sure there are no obstructions and try again with an empty seat.

Follow the ***Emergency Operation with hand-held control*** instructions to reset Turny Low Vehicle's start position.

Follow the instruction ***Emergency Operation with hand-held control*** to reset Turny Low Vehicle's start position.

Please contact your installation engineer if you require further assistance or need to ask questions about warranty. We recommend keeping the following information to hand at all times /

Contact details for the installation engineer /

Model /

Serial number /

Year of manufacture /

Date of purchase /

Notes /

Introduction for users

BraunAbility Europe AB would like you, as a user, to feel secure and comfortable with your new product. For that reason, it is your installer's duty to go through the following with you /

Safety
Operation
Emergency-mode operation

If any questions about the product should come up later on, you are of course very welcome to contact your installation engineer again.

The above chapters have been gone through /

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Declares that the product /

Turny Low Vehicle, model right/left

Conforms to following directives, standards and regulations /

2006/42/EEC
2007/46 EEC
2007/47/EEC,
LVFS 2013:11
ECE R-10
ECE R-14
ECE R-16
ECE R-17 rev 5 (incl supplement 08)
ECE R- 21
ISO 3795

BraunAbility Europe AB, Stenkullen
2019-04-02

Jan M Jensen
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.

BraunAbility Europe AB reserves the right to make alterations without previous notice.

© 2019 BraunAbility Europe AB

BraunAbility Europe AB
Åkerivägen 7
443 61 Stenkullen
Sweden

Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu