


## How to set up the BraunAbility Remote app

1. Download the BraunAbility Remote app from Google Play or App Store.
2. Activate Bluetooth on your smart-phone.
3. Press the Teach-in button (A) quickly three times in a row and follow the setup wizard in the app.
4. If the Teach-in button on the receiver isn't accessible, run the lift until it is.

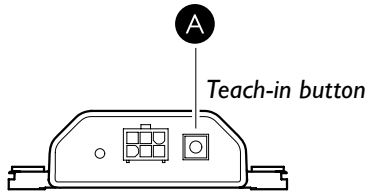


*Before using this device, download and read the user manual available at: [www.braunability.eu](http://www.braunability.eu)*

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)


427358-4 2020/12

 **BraunAbility®**

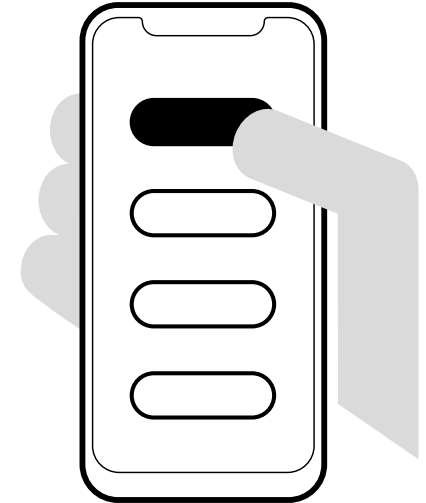

## BRAUNABILITY REMOTE

Quick guide for Q-series,  
Vista, Millennium and Century

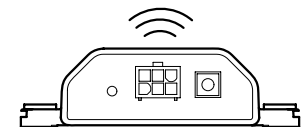

## How to install the BraunAbility Remote receiver


### Q-series

1. Disconnect the standard hand held control and **replace the standard socket with the adapter cable socket**. Remove the standard cable socket by pushing out the 4 clips from the inside. Reuse the clips when fitting the adapter cable socket.
2. Connect the old socket to the adapter cable inside the pump module and the hand held control on the outside.
3. Connect the adapter cable to the receiver.
4. Mount the receiver according to the illustration using the supplied adhesive pads and fasten using a drill (5 mm) and **one plastic clip**.
5. Use the supplied cable tie to tidy up the excess cable.
6. Switch from A to B on the main control board inside the pump module of the lift.

### Vista, Millennium, Century

1. Choose placement, **inside, on or near** the pump module. Ensure that the **Teach-in button** on the receiver is easy to access. Mount the receiver in step 4.
2. Connect the adapter cable.  
**A, Inside pump module:** See step 1 and 2 for Q-series.  
**B, Outside pump module:** Disconnect the standard hand held control. Connect the supplied adapter cable between the pump module and the standard hand held control.
3. Connect the adapter cable to the receiver.
4. Mount the receiver according to the illustration using the supplied adhesive pads. In some cases it might be necessary to drill (5 mm) and use the supplied clips.
5. Use the supplied cable tie to tidy up the excess cable.

### Q-series / A, Inside pump module

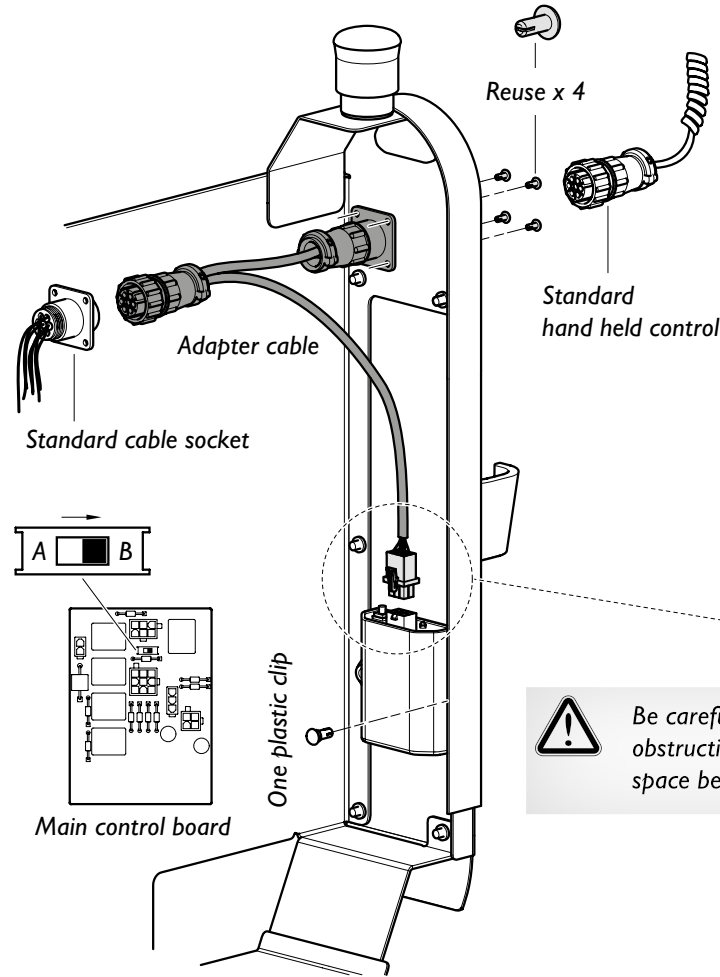

### B, Outside pump module

