# Caroslide

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Thank you for choosing a

# Caroslide from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit [www.braunability.eu](http://www.braunability.eu).

Once again, thank you for placing your confidence in our products!

**Installation**

---

- 16 Rebuilding the Caroslide for left side operation
- 18 Mounting underneath the Turnout / Turnin
- 20 Mounting on top of Turnout / Turnin with a BEV-seat

**Operation**

---

22

EN

Index

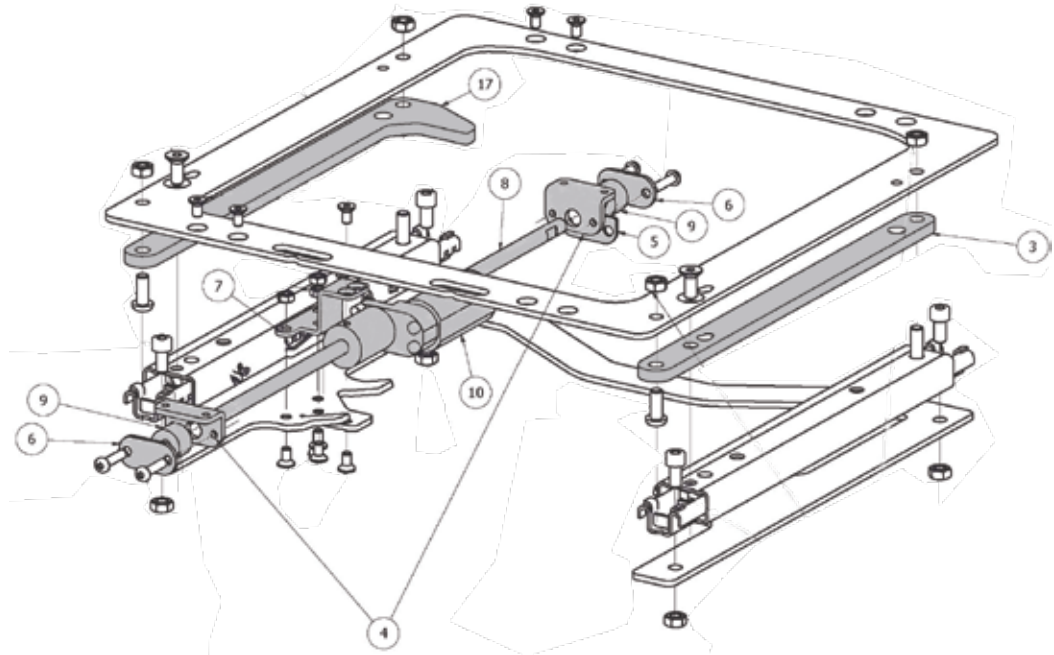
“Product certificate” at the end of the manual.

# Installation

## Rebuilding the Caroslide for left side operation

If the Turnout is in a left side version the Caroslide must be rebuilt for left side operation. Following parts must be moved and mirror-inverted, marked in grey /

- 3. Spacer
- 4. Bracket
- 5. Rotation lock
- 6. Locking plate
- 7. Motor bracket
- 8. Threaded axle
- 9. Nuts, 2 pcs.
- 10. Motor
- 17. Stop spacer



**When rebuilding for left side operation following instructions are very important /**

Tighten the three screws for the motor bracket to the lower plate by using normal torque. Then tighten the two motor screws by using a light torque. Wiggle the motor towards both sides within the play admitted by the two screws. Fix the motor in the middle of the play movement. Tighten the screws by using normal torque.

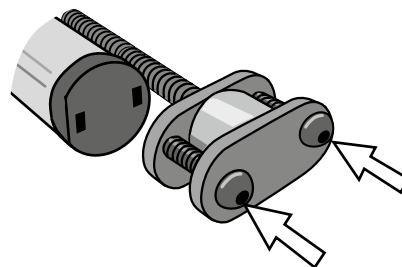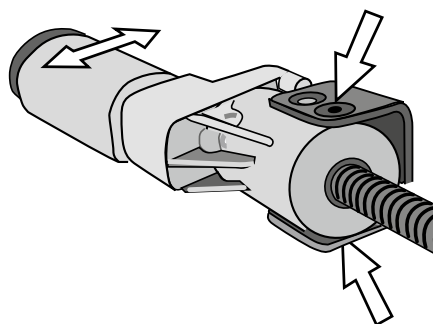Drive the motor to an end position. Loosen the locking plates to provide a position for the threaded axle that is free from strain. Make sure trapezoid nut is tight to the plate. Tighten the screws alternately using normal torque. Repeat the procedure at the other end position.

## Mounting underneath the Turnout / Turnin

Modify, alternatively remove the rear cover of the Turnout.

Countersink the holes in the mounting bracket to use for mounting the bracket legs.

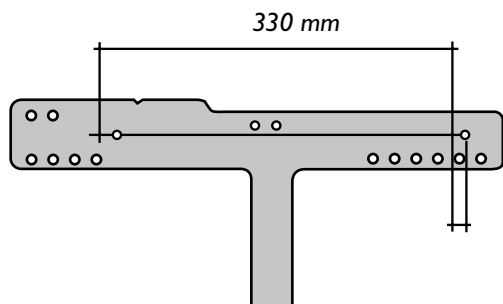
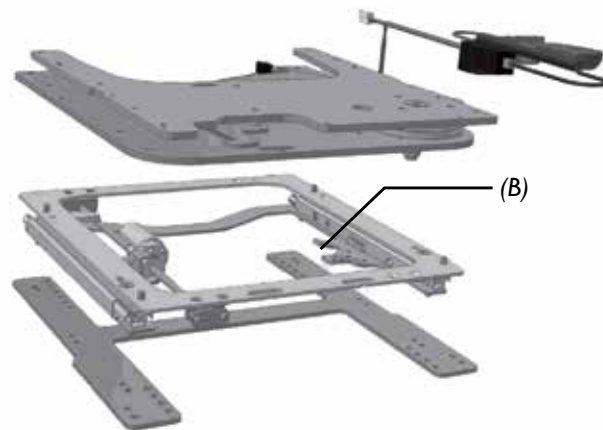
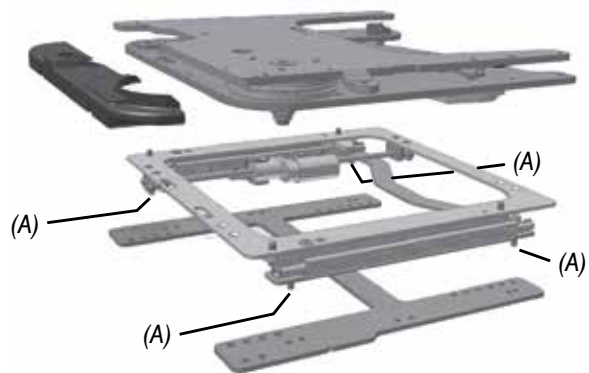
Mount the Caroslide between the Turnout and its mounting bracket by first removing the four M8 nuts (A) from the underside of Caroslide.

Then drive back and forth to be able to reach the screws. Fix the four M8 screws in the (M8) drilled 330 mm cc holes on the Turnout mounting bracket. Drive Caroslide backwards. Remove the upper M8 nuts.

If it is a Turnout 2E bend down the stopper (B) on the opposite side from where the Caroslide motor is placed, to allow free passage for the Turnout motor. Alternatively, place a 5 mm spacer between Turnout 2E and Caroslide.

Screw the four remaining M8 screws in the Turnout.

The wiring diagram is found on page last page.

## Mounting on top of Turnout / Turnin with a BEV-seat

Remove the four bottom M8 nuts (A). Then drive back and forth to be able to reach the screws.

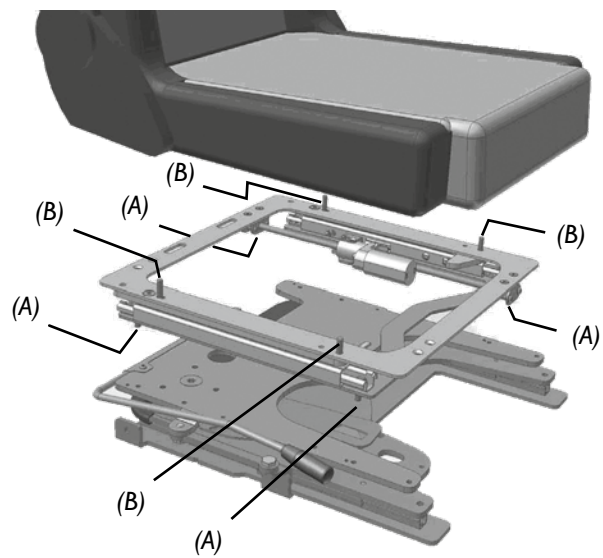
Screw the four screws, either in the rear drill pattern or in the front drill pattern, depending on what is required. Do NOT use the front drill pattern in vehicles with a narrow door opening.

Using the front drill pattern would require a door opening of at least 650 mm, to be able to swivel inside the vehicle with the rails in a max forward/out position when a motor failure has occurred.

Drive Caroslide backwards. Remove the four upper M8 nuts (B), exchange the four M8 screws for the enclosed M8x35 screws. Screw the BEV-seat onto the Caroslide.


A Recaro Specialist L seat can also be used, but in that case two holes must be drilled in the rails and M6 screws must be used (not included).

The wiring diagram is found on page last page.

## Operation

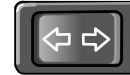Caroslide adjusts the seat electrically lengthwise. This increases comfort and makes it easier to get in and out of the vehicle. Keep foreign objects away from the rails. Clean regularly by using a vacuum cleaner. Caroslide is designed to function maintenance-free. Check irregularities such as loosening screws or un-usual noise during use.

|                                   |        |
|-----------------------------------|--------|
| Max user weight.....              | 150 kg |
| (100 kg Mounted under the Turnin) |        |
| Width.....                        | 445 mm |
| Length.....                       | 450 mm |
| Height.....                       | 41 mm  |
| Stroke.....                       | 240 mm |
| Fuse.....                         | 7,5A   |

- The product documentation must always be available during inspection of the vehicle.
- Always contact your dealer for disassembly.

Please contact your installation engineer if you require further assistance or need to ask questions about warranty. We recommend keeping the following information to hand at all times /

Contact details for the installation engineer /

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
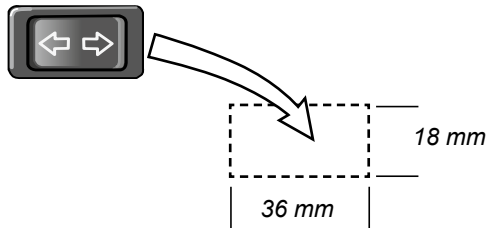
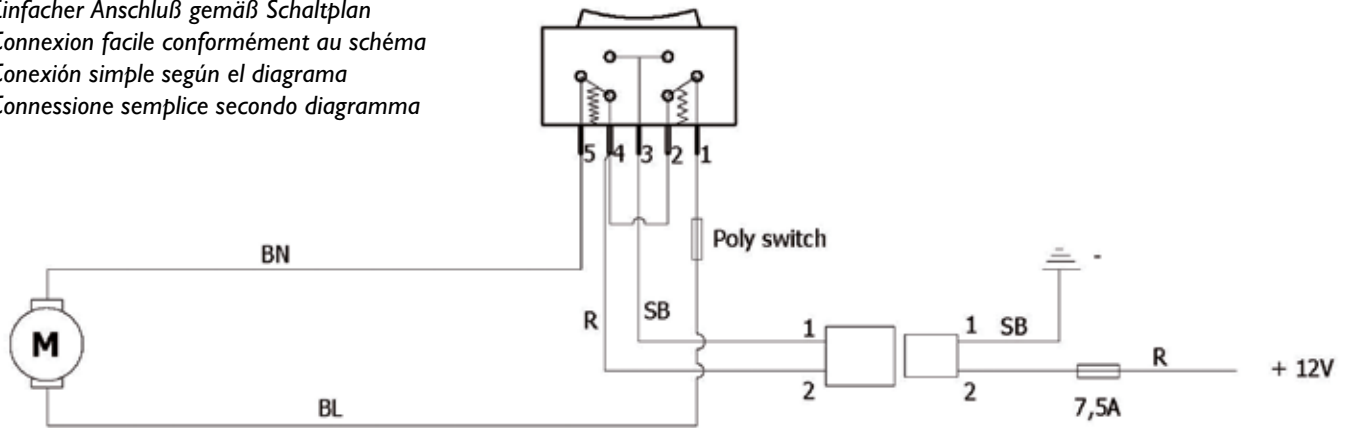
## Declares that the product /

Caroslide, model left/right

## Conforms to following directives, standards and regulations /

2006/42/EEC  
2007/46 EEC  
2007/47/EEC  
LVFS 2013:11  
ECE R-10  
ECE R-17 rev 5 (incl supplement 08)  
ECE R- 21  
FMVSS 302  
ISO 7637-2  
ISO 11452-2  
ISO 11452-4

BraunAbility Europe AB, Stenkullen  
2019-04-02

Jan M Jensen  
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005  
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.  
BraunAbility Europe AB reserves the right to make alterations without previous notice.  
© 2019 BraunAbility Europe AB

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)